AESCULAP® DENTAL

HAUPTKATALOG
MAIN CATALOG

AESCULAP® DENTAL

COMPETENCE IN DENTISTRY

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with deepening precision and using less and less invasive techniques.

Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

AESCLAP®'s products are developed strictly by the close contact to our customers. This applies as much to new and innovative areas such as microsurgical instruments for implantology, periodontics and endo-

dontics, as to standard dental instruments in general. Throughout the world, AESCULAP® products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available, optimized hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. AESCULAP®'s range of services can help to achieve such improvements.

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Good functionality

- Precise manufacturing and high class materials lead to high quality and reliability
- The working tips are matched to the requirements of dentistry

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Comfortable ergonomics

- Shape and thickness ensure low-fatigue work
- Lightweight handles

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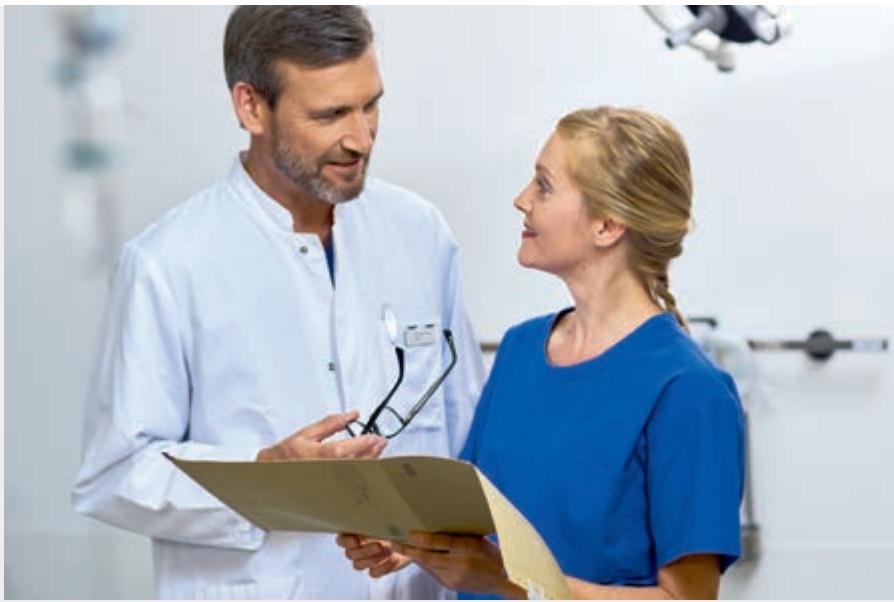
Aesthetics

- The winner of the **iF DESIGN AWARD 2015** Category Medicine / Healthcare

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AESCULAP ACADEMY – Forum for contemporary medicine.

The Aesculap Academy is one of the leading medical forums for everyone who is professionally, passionately and ambitiously committed to people's health. To those medical professionals we offer top quality knowledge transfer based on globally recognized quality criteria using innovative methods and technologies.

The Aesculap Academy has its roots in the company B. Braun, which has been protecting and improving people's health for more than 175 years. With our courses, hands-on trainings and symposia we help to honor our parent company's promise of SHARING EXPERTISE.

The courses at the Aesculap Academy offer participants who want to continue to learn in an inspiring environment knowledge transfer and teaching that are adapted to real life: lifelong learning, true-to-life training situations and realistic content for a better life for patients and medical staff.

Aesculap Akademie GmbH
Am Aesculap-Platz
78532 Tuttlingen
Phone +49 7461 95-2001
www.aesculap-academy.com

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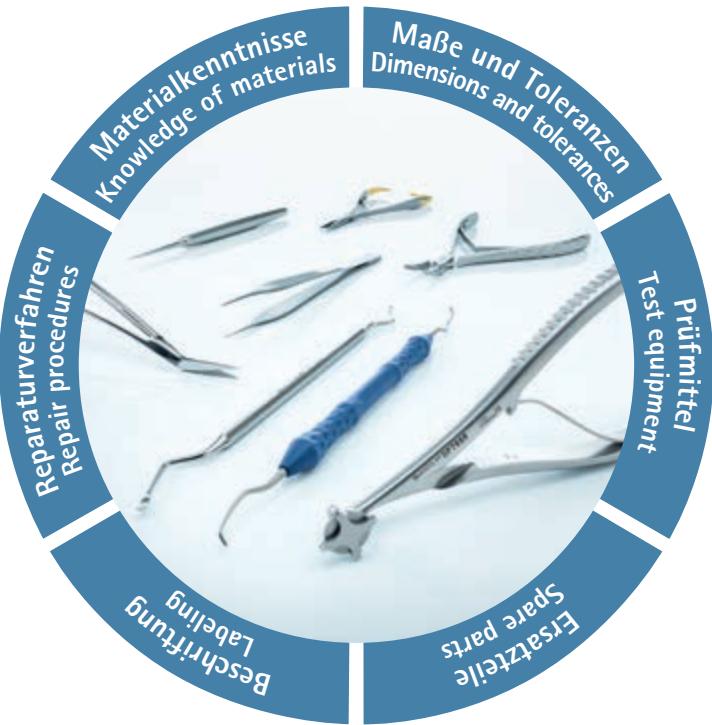
Repair procedures

- High-quality repair procedures
- Work process conforming to the manufacture of the new product
- Access to current original drawings and modification information

Dimensions and tolerances

- Clearly-defined repair limits in accordance with AESCULAP® quality standards
- Minimum size specifications for the reliable use of an instrument set
- Preservation of model consistency even after multiple repairs

Spare parts

- Exclusive use of AESCULAP® original spare parts
- Safeguarding of functionality and durability
- Prevention of corrosion and changes in the material

Knowledge of materials

- Processing methods are adapted to the composition of the material
- Reworking of surfaces complies with valid procedure for new products
- Exclusive use of validated auxiliary materials

Test equipment

- Complete control after the repair
- Guarantee of functionality by testing all important features
- Use of the same test steps and parameters used for new products

Labeling

- Restoration of original AESCULAP® labeling
- Application of ATS repair labels with month and year of repair

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Planning and Consulting

In an age of managed care, quality assurance circles and rapid progress in the field of minimally invasive techniques, the future has already begun today. This future vision invariably also means investment in human resources and technology.

Limits on financial resources in the health-care sector and complex documentation and settlement procedures mean that decisions on such investments are becoming increasingly important, but at the same time more and more difficult.

Investment and future – two concepts that have now become more important than ever before. For only those who make the

right investments today will be in the position to reap the benefits of those investments tomorrow.

Having devoted particular attention to the demand for competent and fair advice, AESCULAP® has continually expanded this area of service. Workable planning for hospital surgical departments depends partly on sound analyses of the actual situation in and around the operating theatre.

Plans to expand, strengthen or restructure facilities can only be addressed properly if you are, for example, aware of the current status of your instruments.

Especially, if new long-term objectives are envisaged, there is no alternative to a detailed appraisal of the status quo. This is where our AESCULAP® planning and advice team can help. We analyse the quality, condition and further usability of your present set of instruments quickly, thoroughly, competently and independently. We then work with you to develop facility criteria indicating the most efficient way to put your plans into practice.

Although every hospital and dental practice has its own individual problem profile, there are standard situations applicable to the planning of operating theatre instrument trays. AESCULAP® can call upon decades of experience for creating a useful tool to provide you with fast, practical solutions to problems – a surgical instrument 'library', that you can fall back on if and when the specific need arises.

We have given the utmost priority to ensure that our planning and advice enables you to take your decisions on a basis of maximum certainty.

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AESCULAP® „Surface“

For mid luminous conditions like
diagnostics, scaling, etc.

AESCULAP® „Rhodium“

For higher luminous conditions
like surgery.

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Suitable for mechanical
cleaning in the assembled
state.

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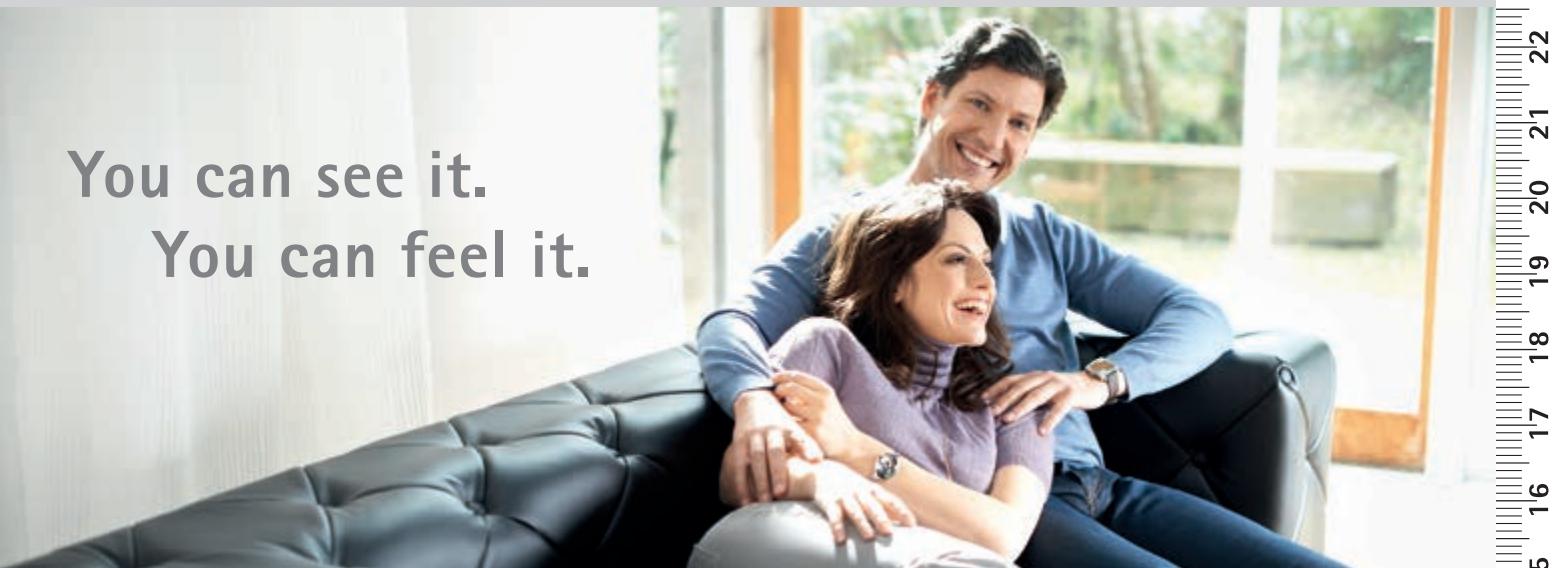
Make yourself comfortable.

You'll never want to put them down again – the new instruments in AESCULAP®'s Ergoprobe range make working that much easier, more relaxed and less tiring. The secret lies in the extremely lightweight, ergonomic handles with surfaces specially designed to have a particularly pleasant feel.

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Good Functionality

- Dental probe working ends made of Phynox steel
 - highly gracile | elastic | dimensionally stable
- Periodontal probes are coated with TiAl-nitride
 - accurate marking | smooth transitions

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Comfortable ergonomics

- Shape and thickness ensure low-fatigue work
- Extremely lightweight handles

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Aesthetics

- The winner of the iF DESIGN AWARD 2015 Category Medicine/Healthcare

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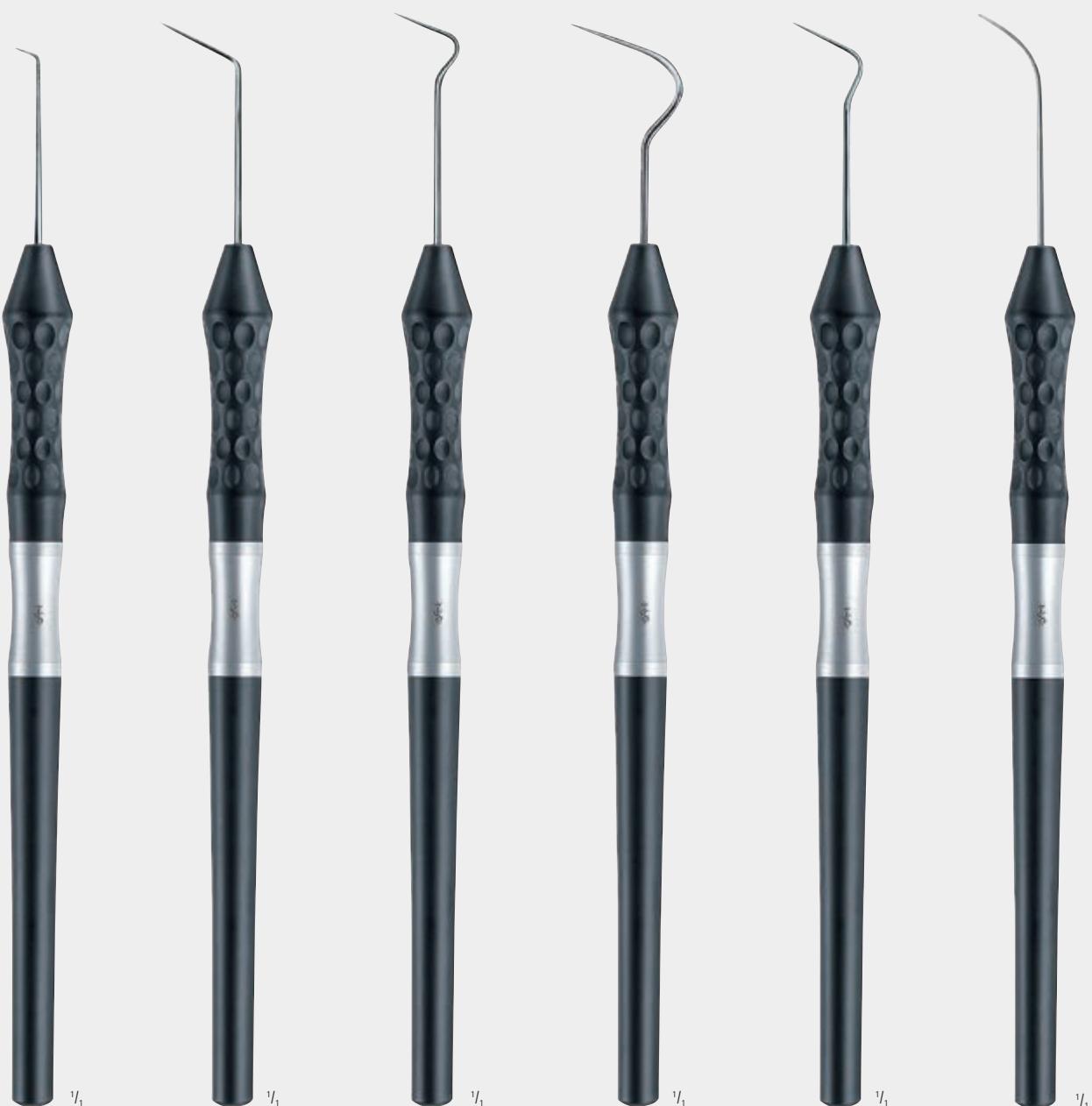
Due to a lack of measurement pressure standardisation, clinical measurements taken using different probing techniques often provide contradictory results. Objective evaluation of periodontal status is only possible when using pressure-sensitive periodontal probes. When assessing periodontal disease, probe measurements are subject to numerous instrumental and subjective influences, which can often result in underestimation of actual probing depths. When too little pressure is used, the main limiting factor on measurement probes is soft-tissue morphology rather than proportions of coronal, supra-alveolar fibre structures. Thanks to their defined spring pressure of 0.2 N, the DB764R and DB765R periodontal probes allow standardised probing in follow-up care of periodontal diseases, and especially in cases of inflammatory gingival changes in long-term studies.

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Periodontal probe with defined probing force of 0.2 N (20 g)

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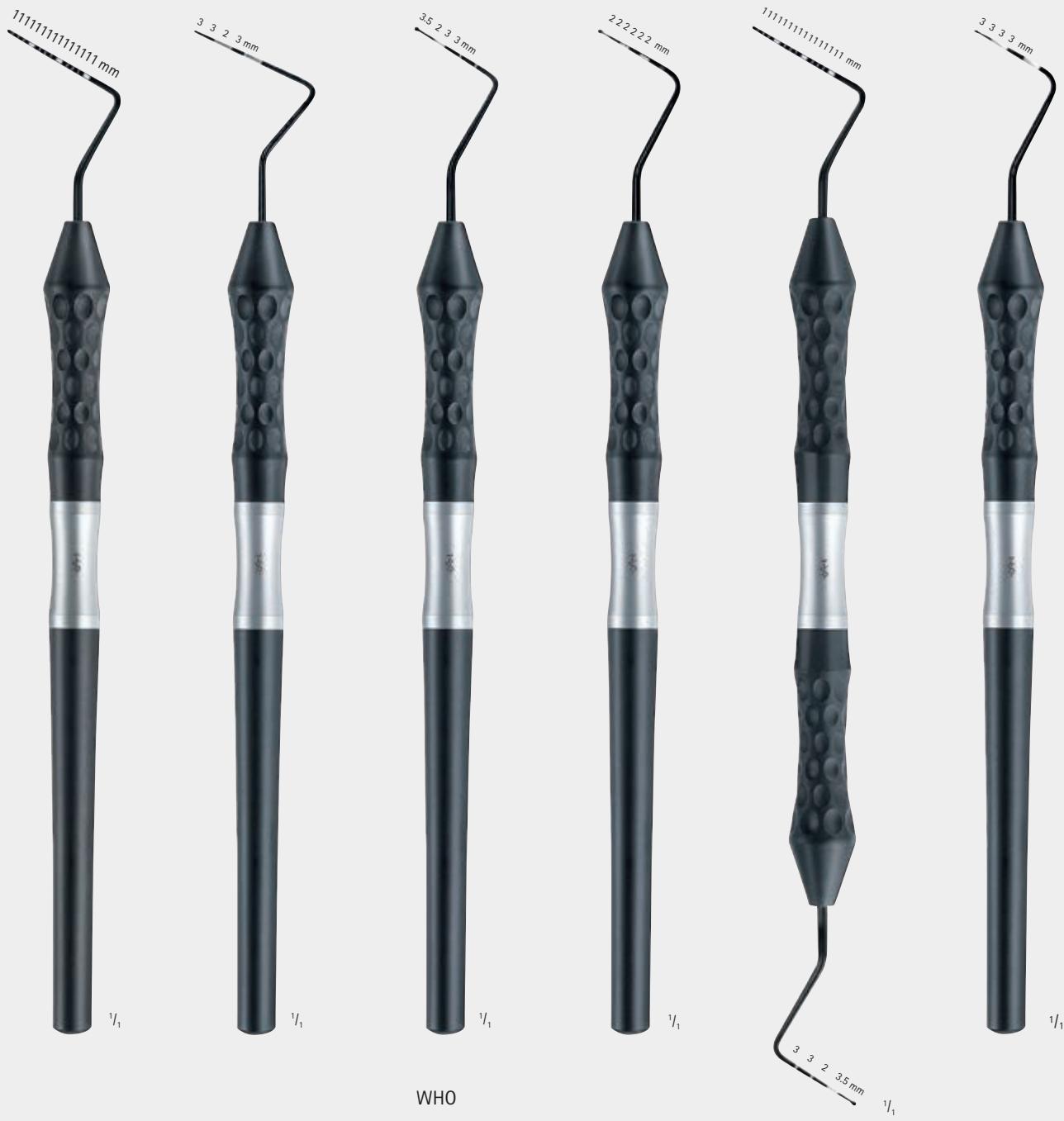
Colour coding system, coated with TiAl-nitride
 ■ exact marking and smoothing

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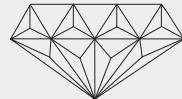
Special tooth tweezers LONDON-COLLEGE, mod.
Particularly strong, distortion-free design, serrated.

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DUROGRIP®-Tooth forceps with tungsten carbide insert for secure grasping of matrices and interdental wedges.

DA246R

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Dental forceps

150 mm, 6"

DA254R

150 mm, 6"

DIAGNOSTIK
DIAGNOSTICS

PARODONTOLOGIE
PERIODONTICS

The Winner of the
iF DESIGN AWARD 2015
Category Medicine / Healthcare.

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Colour coding system, coated with TiAl-nitride

- exact marking and smooth transition

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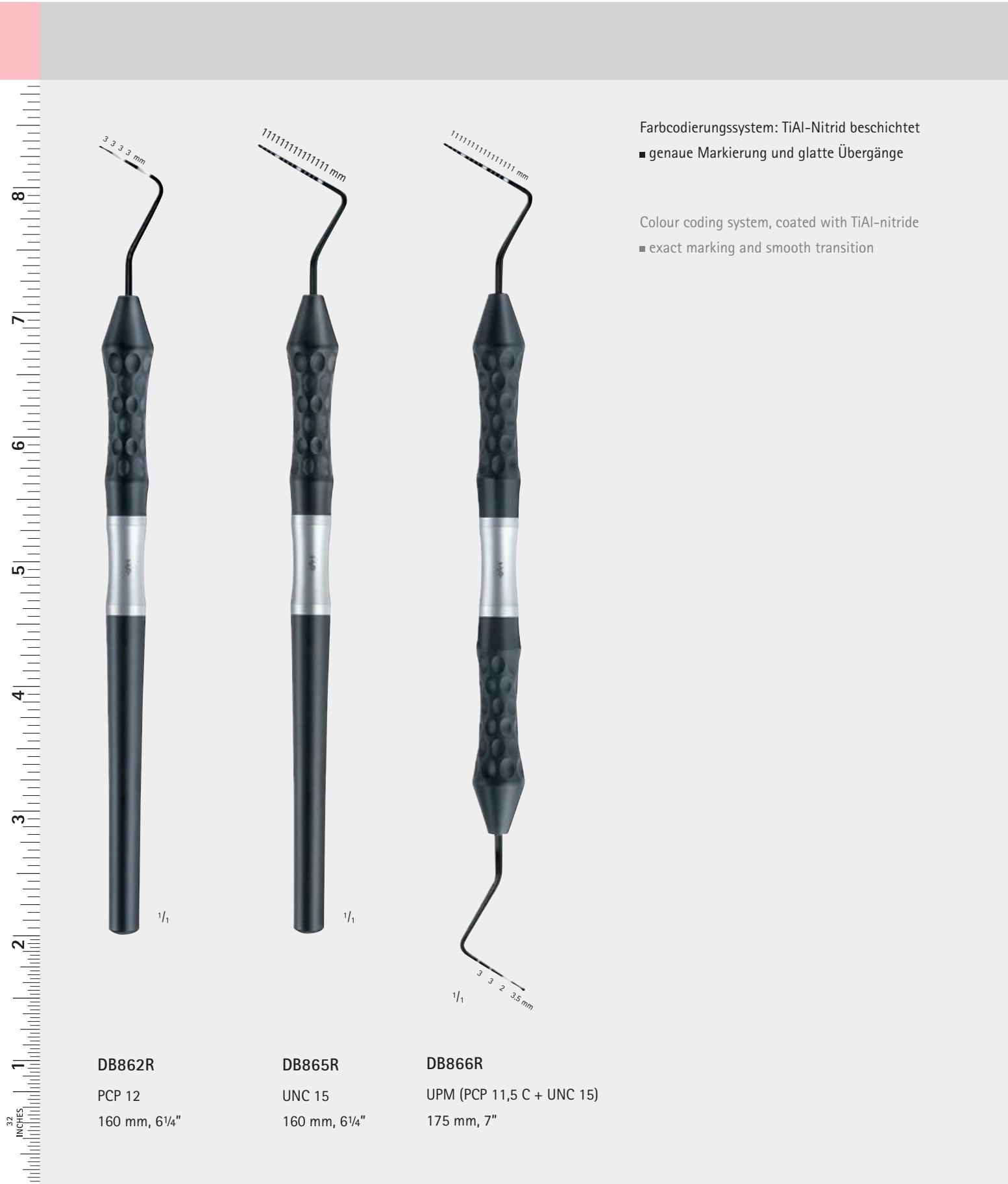
- Anatomical "high-tech" shape
- Extra light and "handy" allows more fatigue-free working
- Colour-coded handles for a quicker and clearer orientation
- Suitable for steam sterilization 134°C (Autoclave) and hot air sterilization up to 240°C

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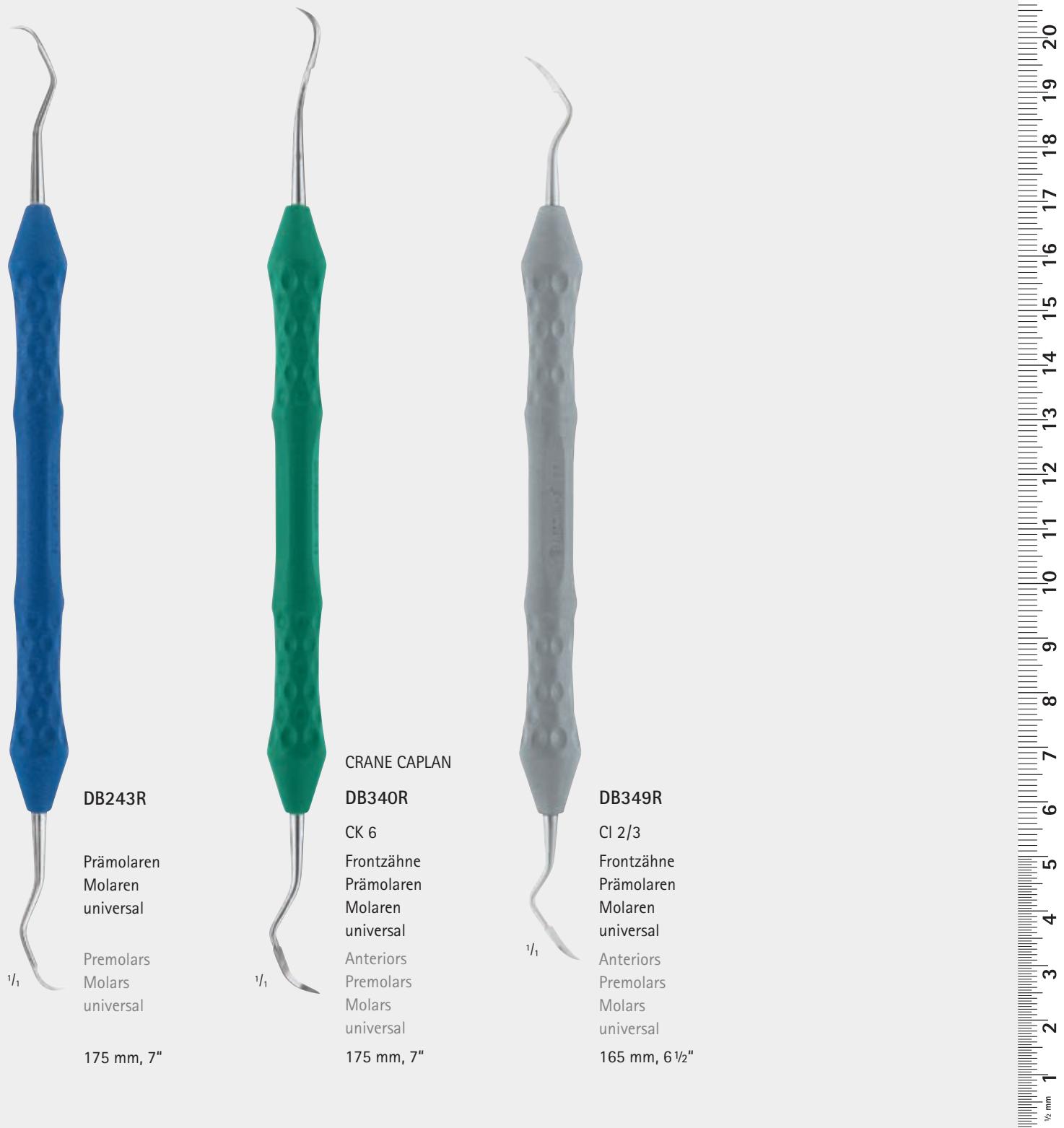
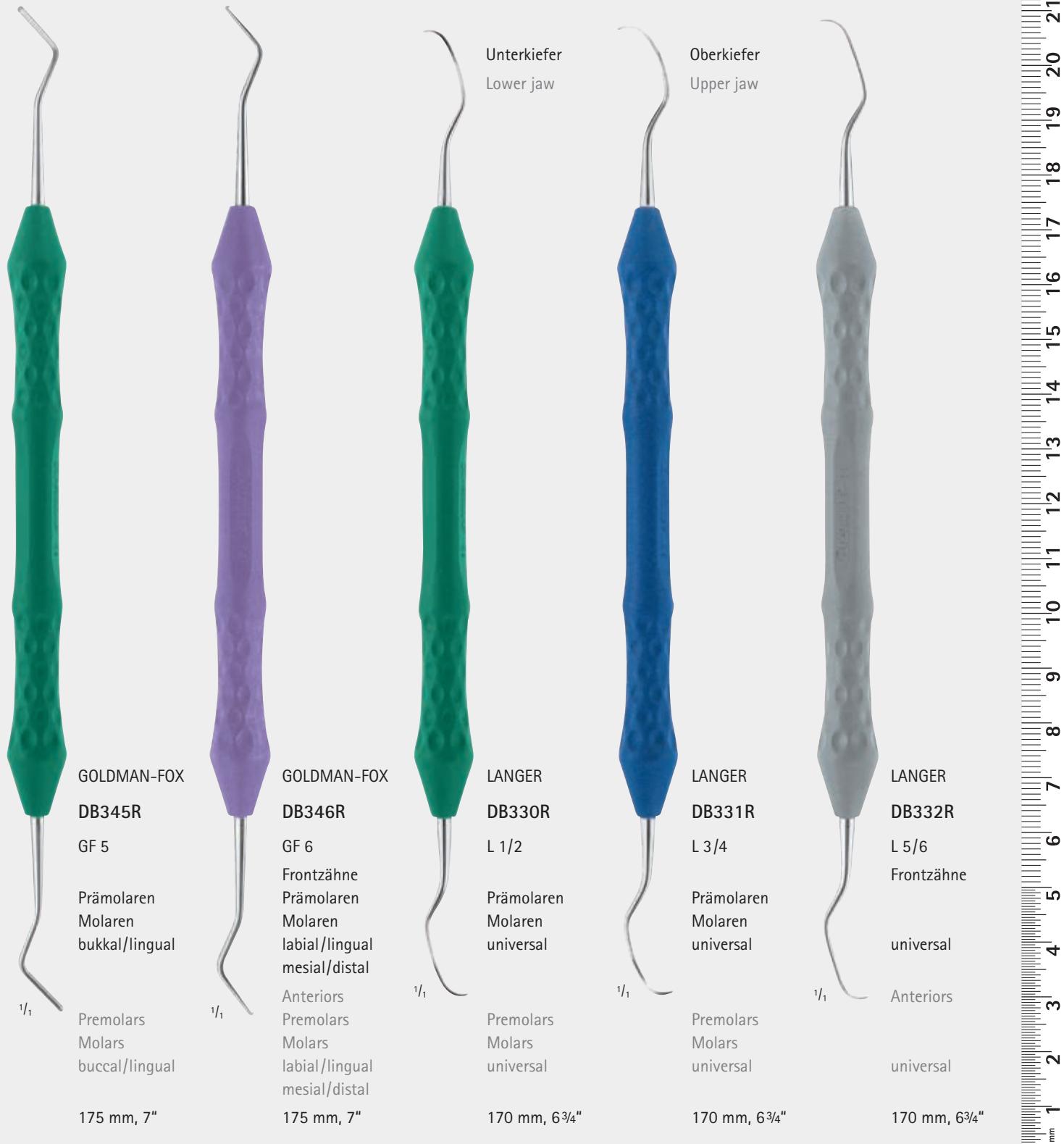
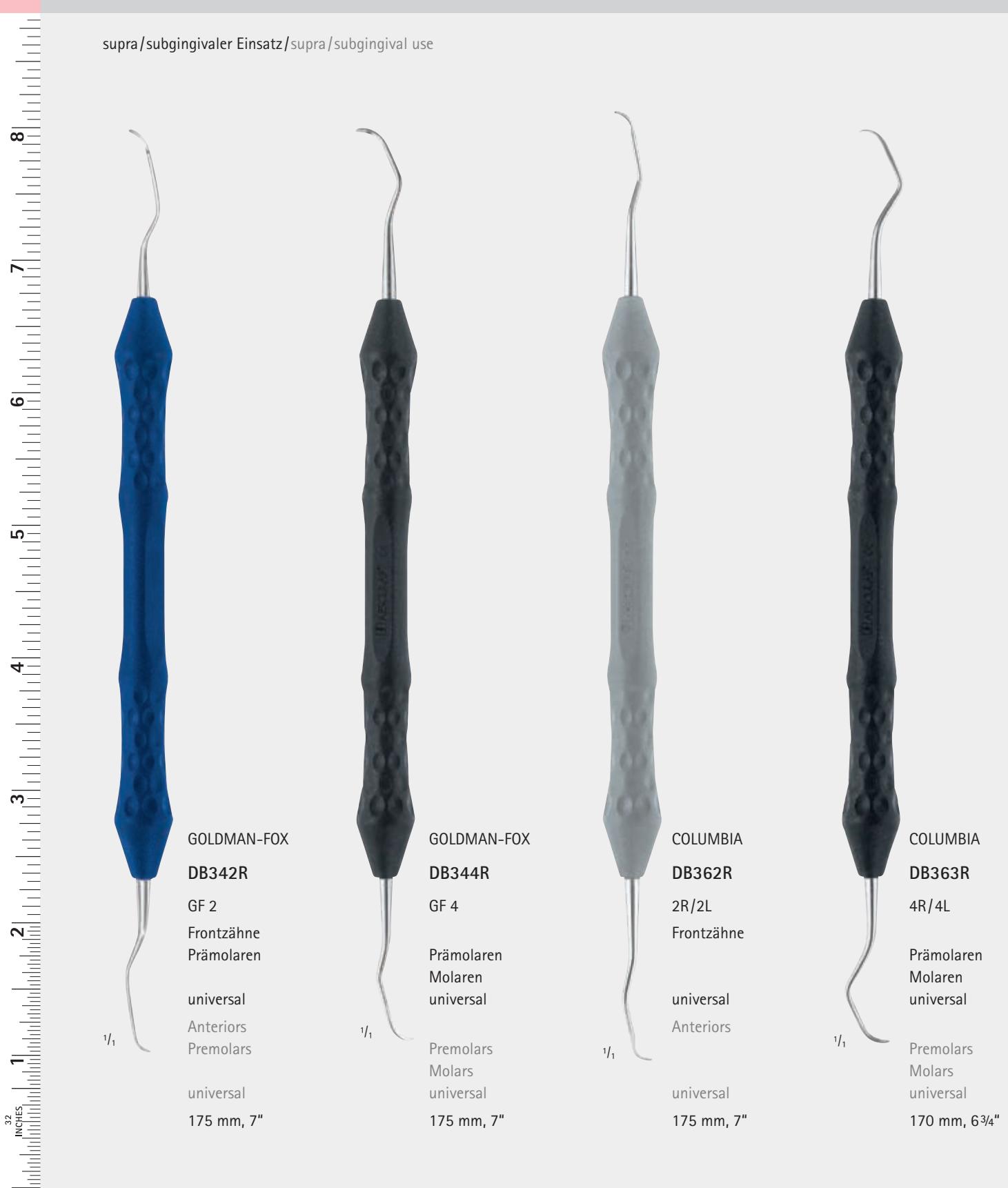
The combination of high rigidity and filigree-fine working ends means that stubborn plaque in particular can be removed thoroughly and efficiently with rigid curettes. It is no longer necessary to carry out time-consuming repeat scaling with flexible curettes. Flexible curettes should always be used to complete root planing.

The Gracey Figures 12/13 and 11/14 prove to be particularly time-saving, since using Figure 12/13 all mesio-distal surfaces in the side tooth area, e.g. in the first quadrant, can be completely treated in vestibular direction in one step without having to change the instrument. It is only with the change to palatal direction that Figure 11/14 is used. The second quadrant begins of course with Figure 11/14 for mesio-distal in vestibular direction, etc.

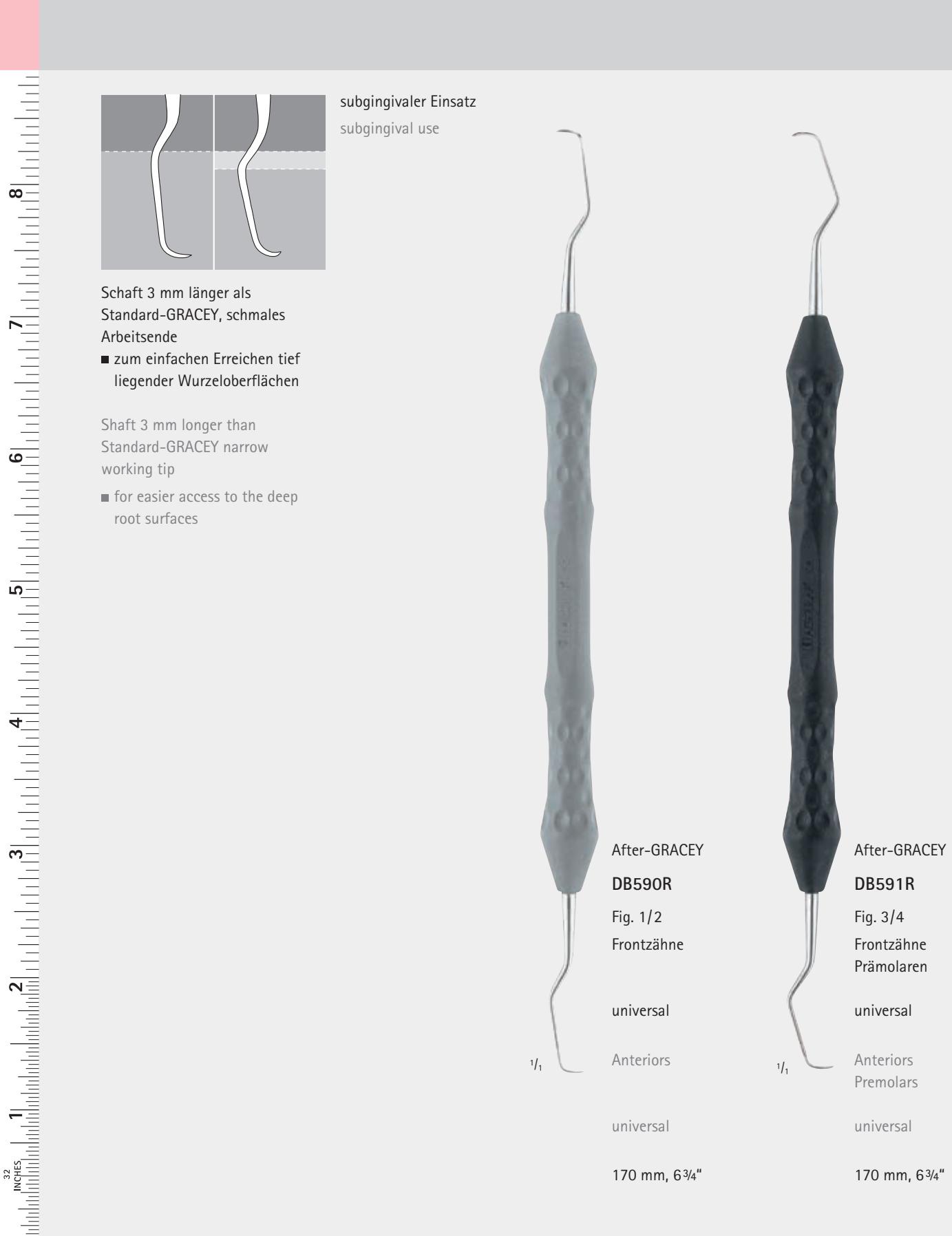
SPEZIELLE KOMBINATION VON GRACEY-FIGUREN, RIGID GRACEY-KÜRETEN
 SPECIAL COMBINATION OF GRACEY-FIGURES, RIGID GRACEY-CURETTES

1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22
 $\frac{1}{2}$ mm

PARODONTOLOGIE

PERIODONTICS

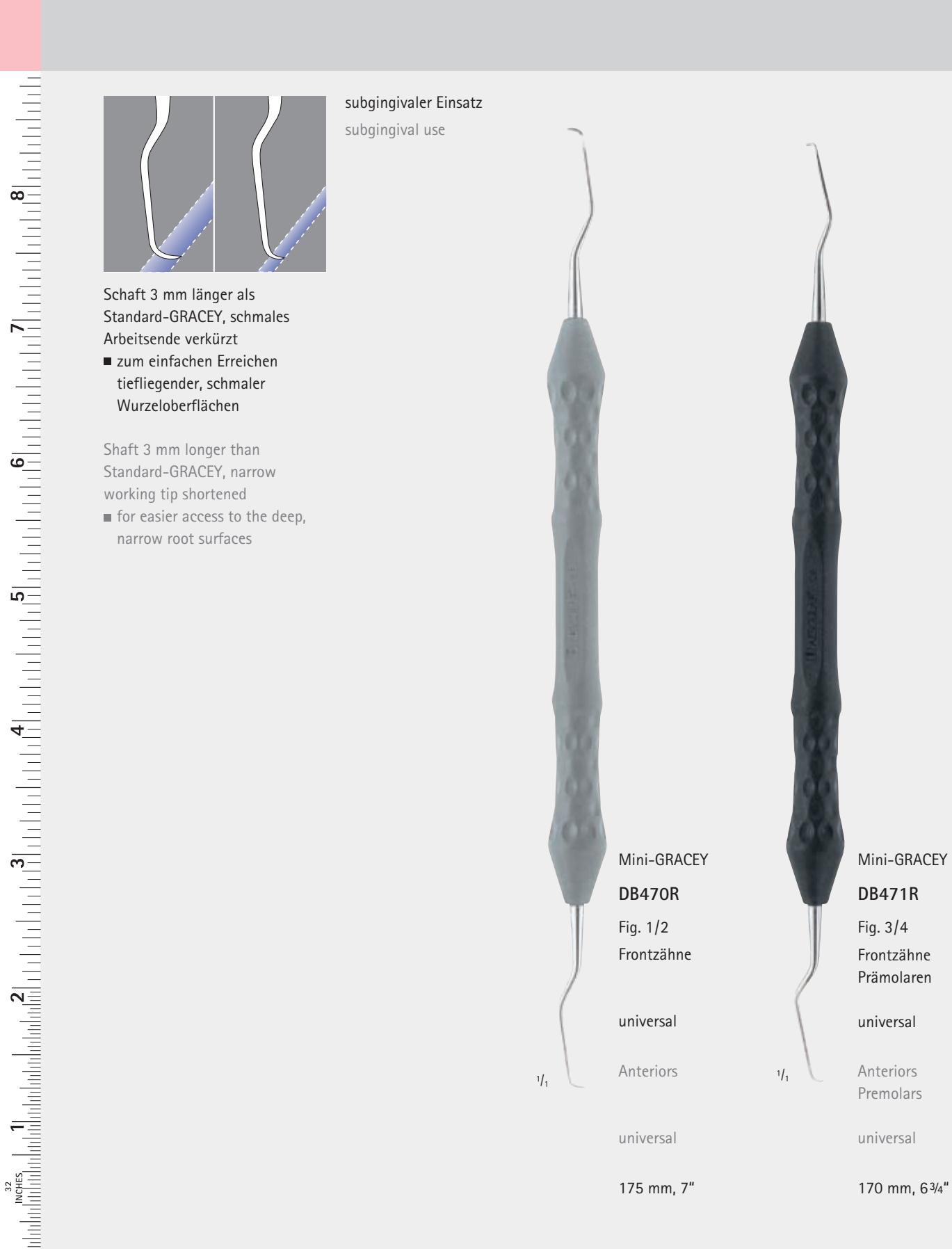
AFTER-GRACEY-KÜRETEN
AFTER-GRACEY-CURETTES

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Titanium curettes for implant cleaning

- Manually sharpenable

To ensure the long-term success of an implant, consistent aftercare is very important. To eliminate plaque on the implant-abutment interface, titanium curettes can be preferable.

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Force-controlled periodontal probe

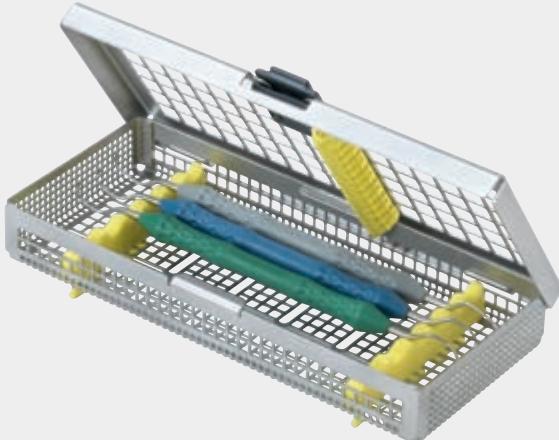
- Defined probing force of 0.2 N (20 g)
 - Low implant damage risk
- TiAl-nitride coating
 - Exact marking and smooth transition
 - No abrasion

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3rd European Consensus Conference
(EuCC) Cologne 2008 on Periimplantitis:
Diagnosis & procedure: ...careful probing
of pocket on four sides (0.2 N probe)...

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Recall Set consisting of
DB180T, DB181T, DB182T
and JG387R

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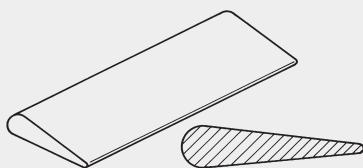
Plastic identification labels, white,
with 10-digit barcode for praxiDoc
or DIOS software applications,
labeling as specified

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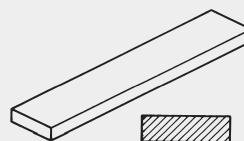
India Corund grinding stone, beige
for shaping and pregrinding of very
blunt instruments

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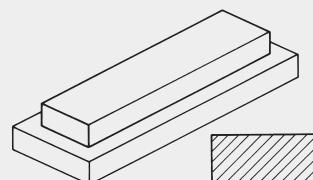
Original Arkansas grinding stone,
grey-white, for repeated grinding on
scalers and curettes as well as for
finishing

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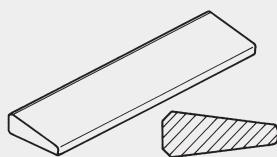
Super Arkansas grinding stone, white,
for repeated grinding on scalers and
curettes as well as for finishing

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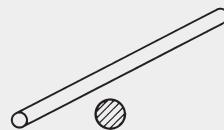
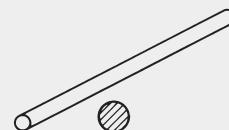
100 mm lang, Ø 6 mm

India Corund grinding stone, round
file, brown beige, for deburring the
working ends

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Plexiglas test rods, for checking
cutting effect of scalers and curettes.

Sales Unit: PAK = Package of 6 pieces

* JG598

Steriliti® AESCULAP®-Schleifstein-Öl

Inhalt: 50 ccm

Steriliti® AESCULAP® oil for grinding
stones

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Items marked * are necessary for the
basic equipment

STERILISIERBARE SCHLEIFSTEINE, ZUBEHÖR
STERILIZABLE GRINDING STONES, ACCESSORIES

PARODONTOLOGIE

PERIODONTICS

BB511

BB515-C

BB515

BB542

doppelschneidend
double cutting

vakuum-vergütet
heat-treated in vacuum

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Scissors

Screw connection enables the regrinding of the scissors.

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New micro instrument handles

As well as its optically appealing design the new surface we have created for the handles of our successful micro instruments offers principally hygienic advantages. The substantially improved grip allows working with minor physical strain more fatigue-free working.

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Special hint

Since suture material is rather tough, the cutting capacity of scissors is quickly reduced which is noticeable as soon as tissues are cut afterwards.

Our solution

To cut sutures with special ligature scissors. The ligature scissor FD252R has additional rounded edges to prevent jamming of the suture.

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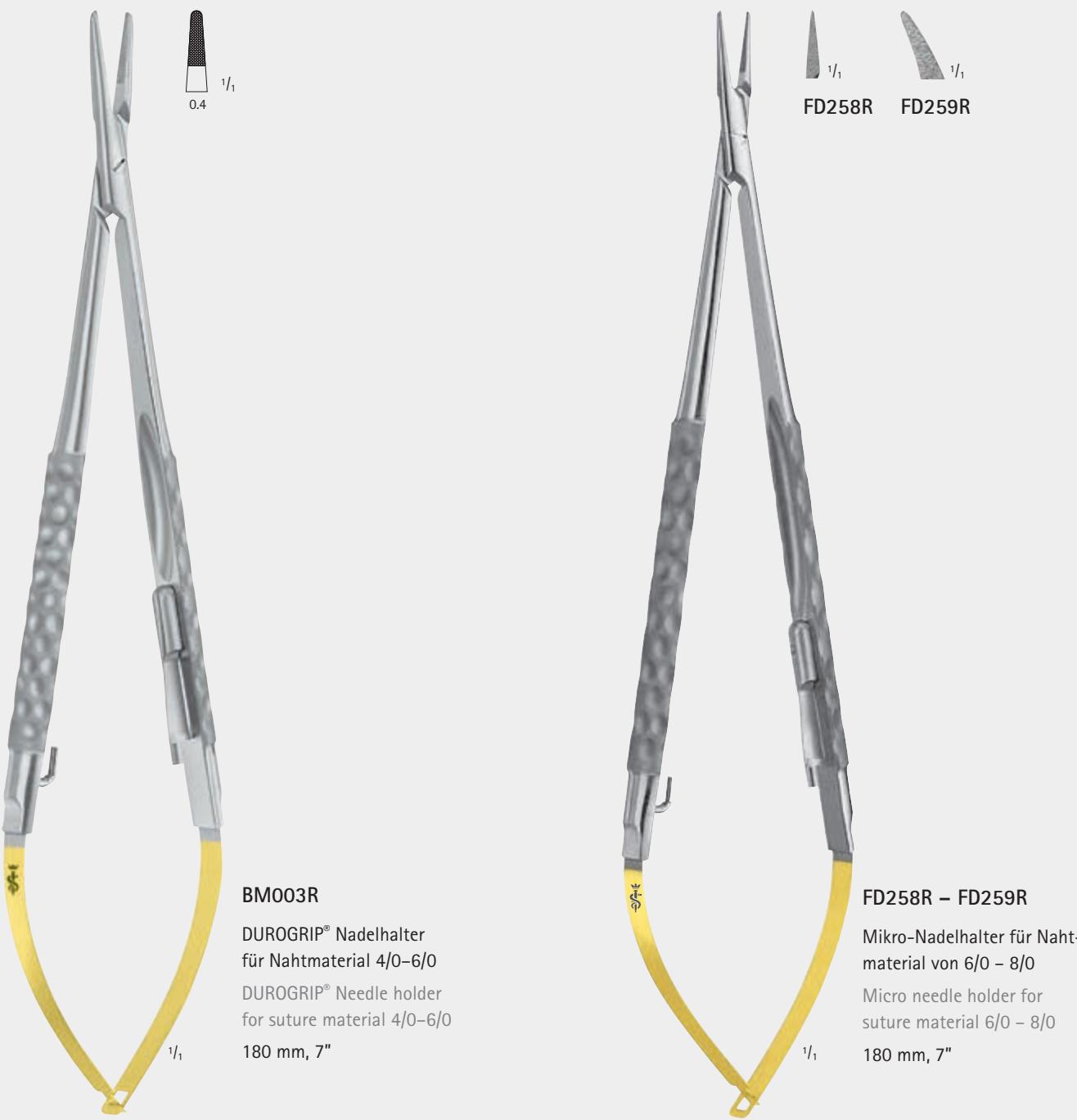
Rounded plate-shaped working tips allow more movement, and significantly increase the radius of mobilisation in frontal and lateral sub-periostal preparation.

The sharpened edge of the instrument tips also allows sulcular as well as tunnel technique preparation (as with the envelope technique).

The design and angle of the working tips significantly facilitates preparation in convex areas.

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Original Arkansas grinding stone,
for grinding on tunnelling instruments

75 x 25 x 7 mm

JG598

Sterilit®i AESCULAP®-
Schleifstein-Öl
Inhalt: 50 ccm

Sterilit®i AESCULAP®
oil for grinding stones
Contents: 50 ccm

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The **straight** tunnelling instrument is mainly used in the upper jaw, in the frontal and lateral region. This allows large areas of the gingiva to be undermined using both tunnel and sulcular techniques.

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The **angled** tunnelling instrument is primarily used for preparing lateral, buccal regions. It can also be used distally, without tearing the mucosal tissue and also for the lower jaw.

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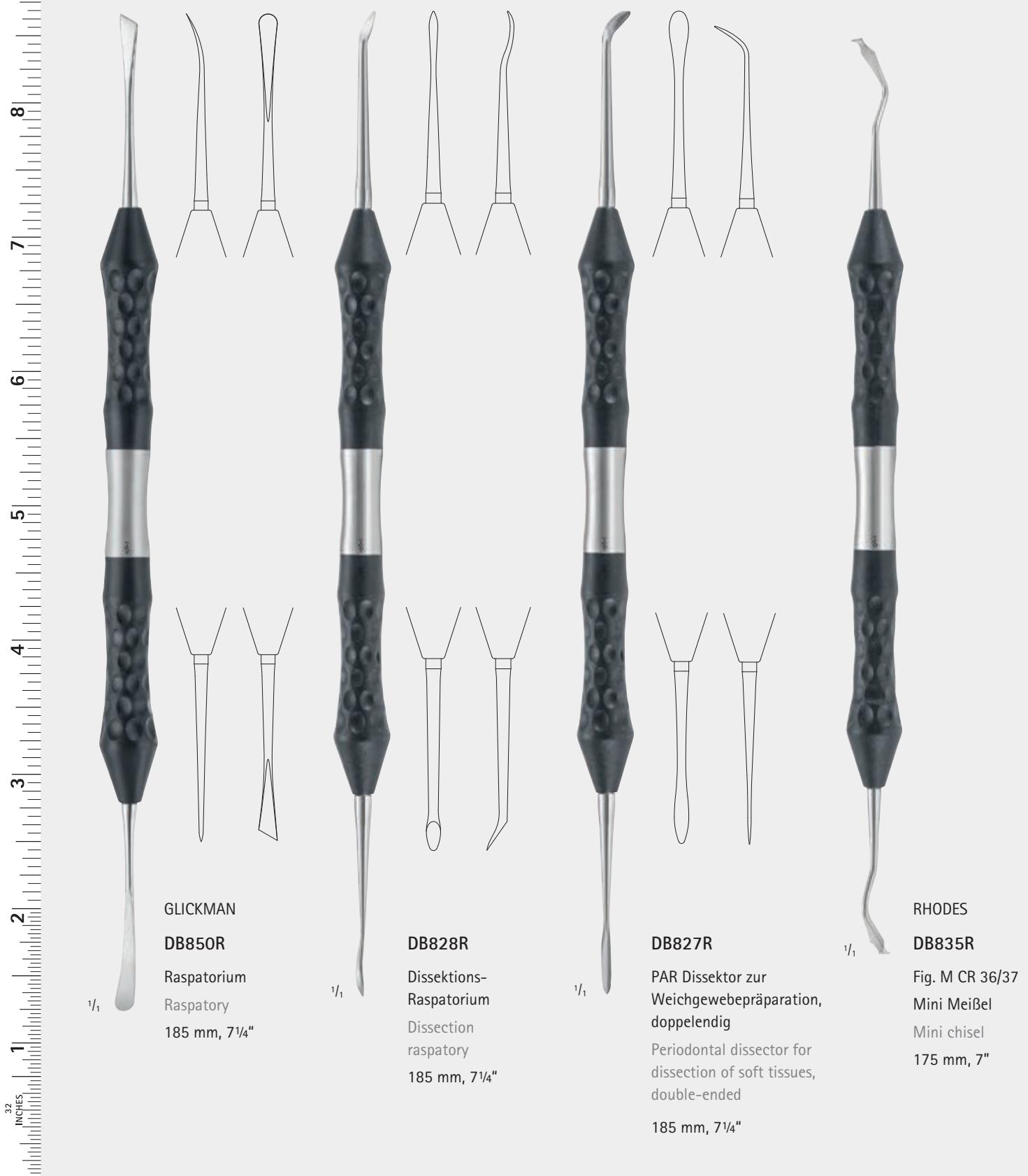
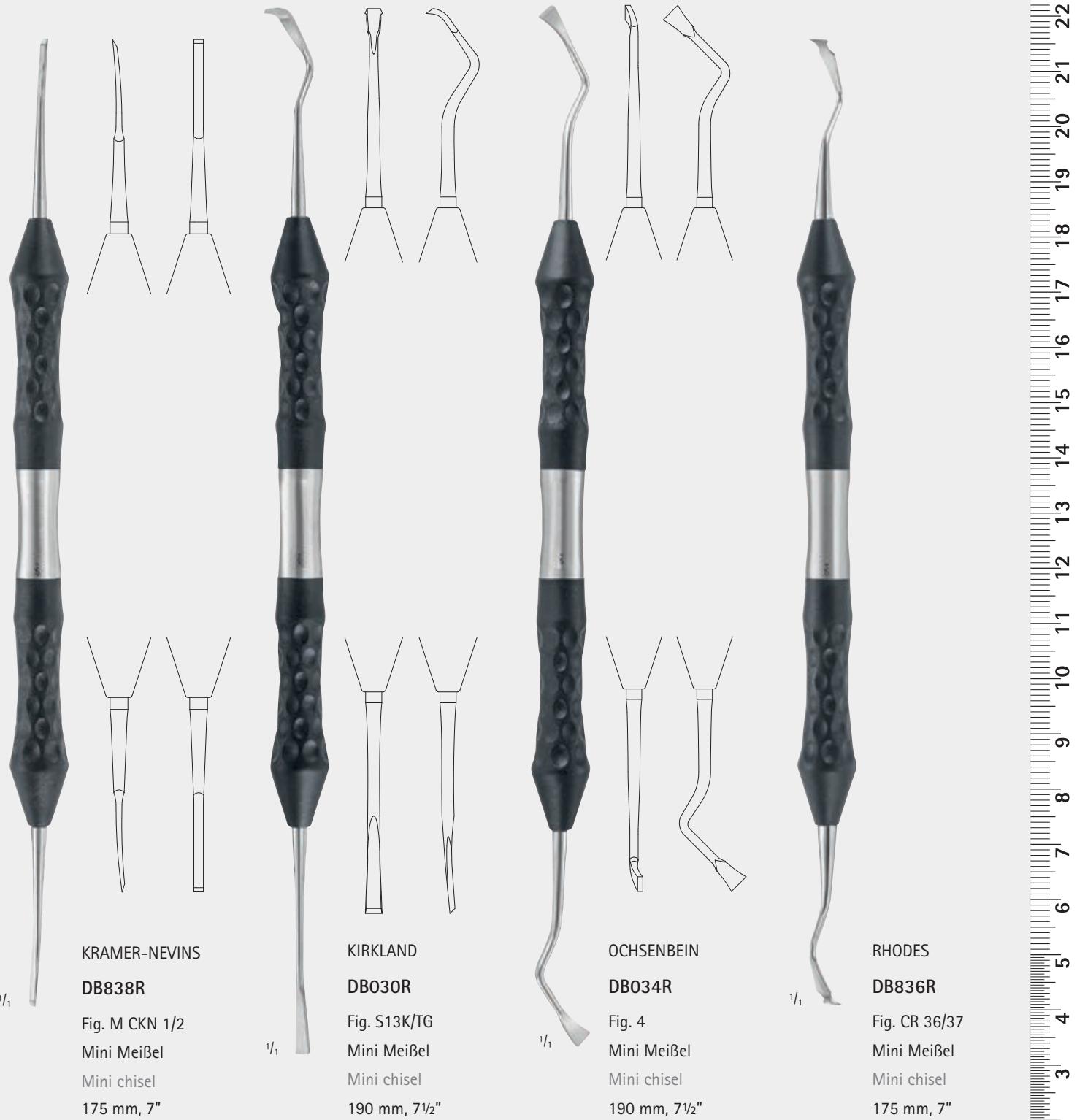
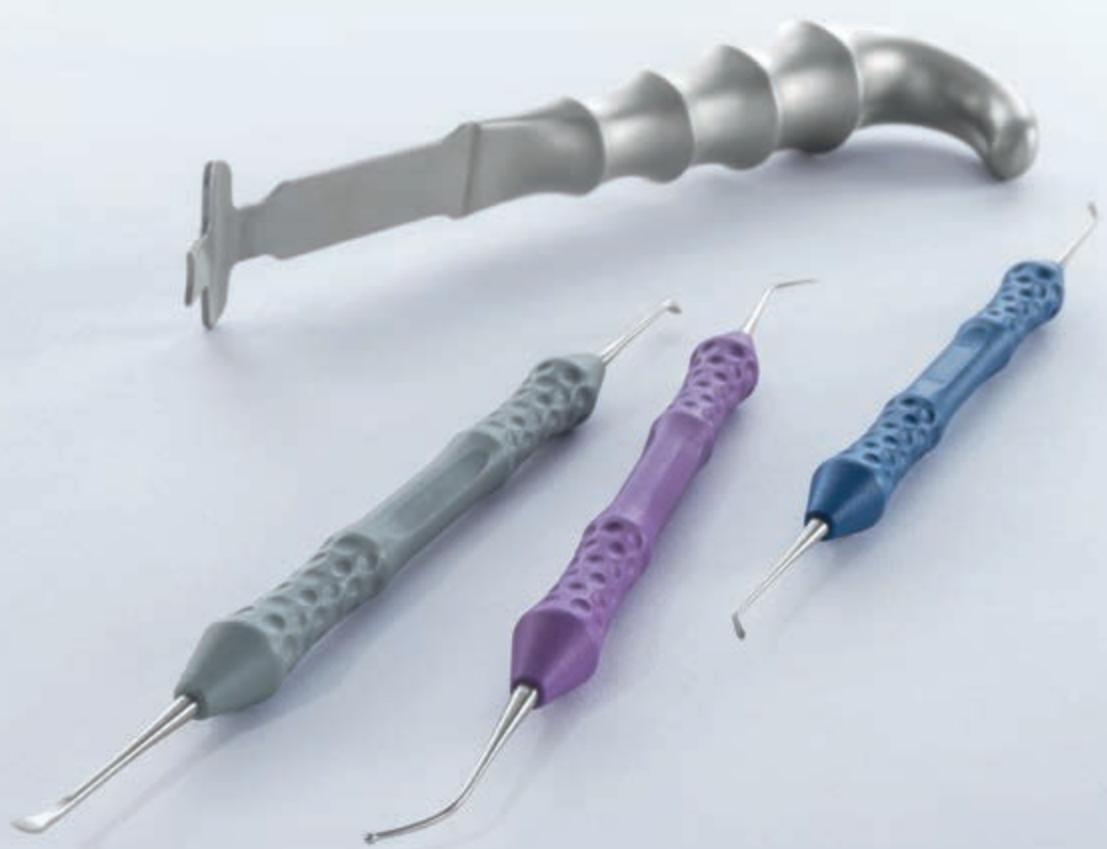
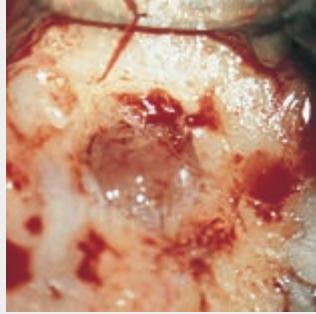
Elevation of the flap starting from the relieving incision with the disk-shaped elevator (DB407R).

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Retraction of the flap using the retractor.

The apical lesion is visible in the center of the picture.

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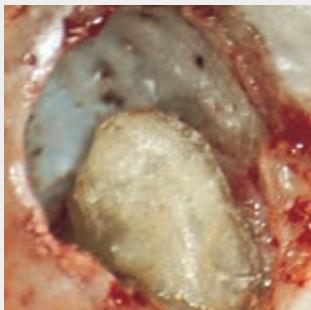
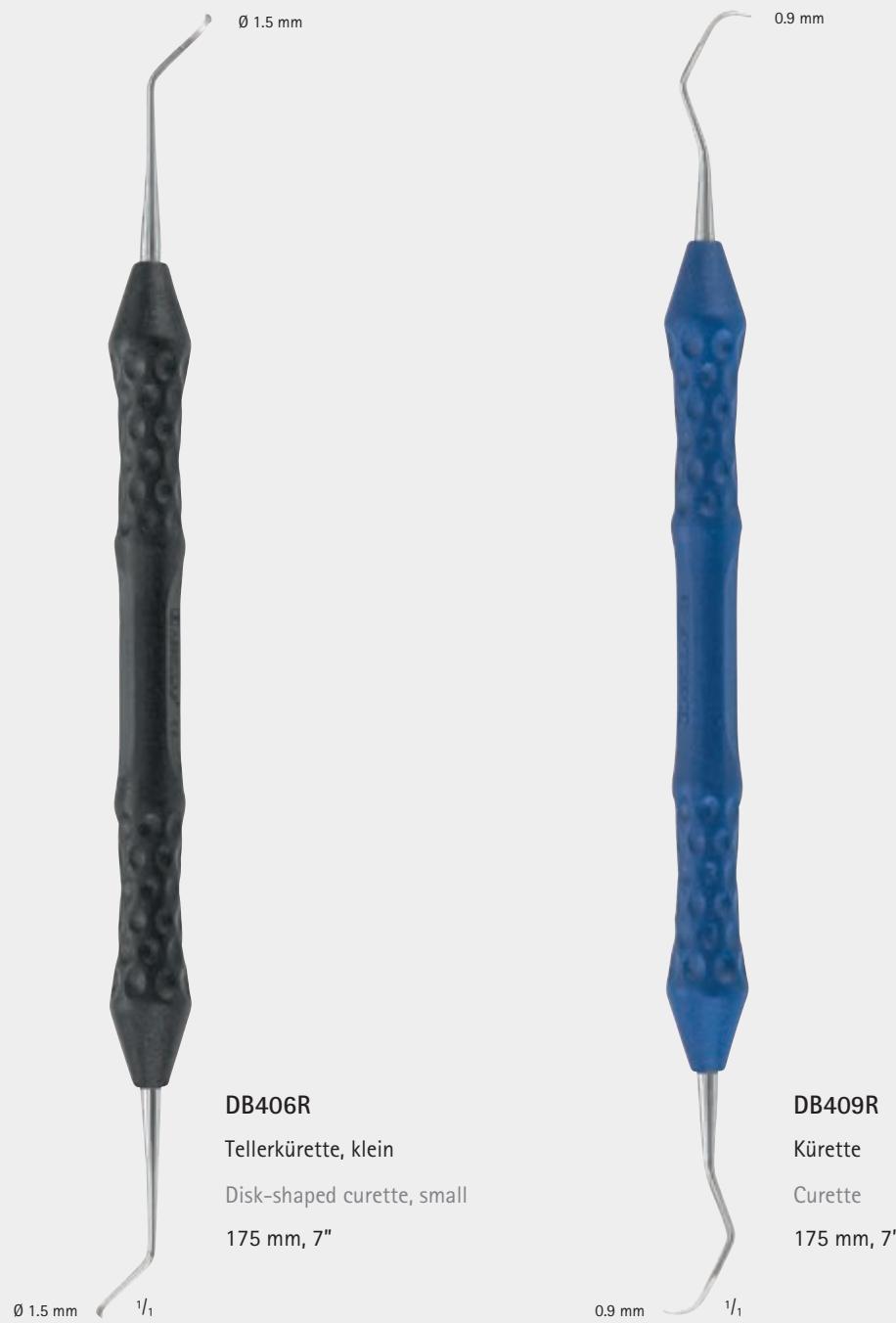
The apical lesion removed using the following:
disk-shaped curettes (DB406R resp. DB408R, the spoon-shaped curette (DB409R)).

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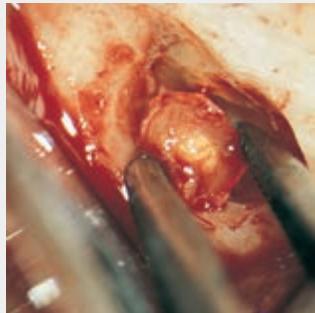
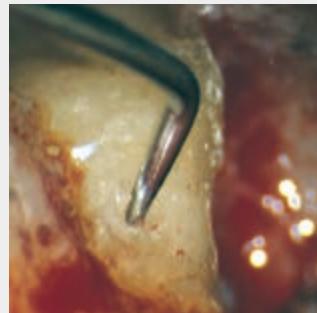
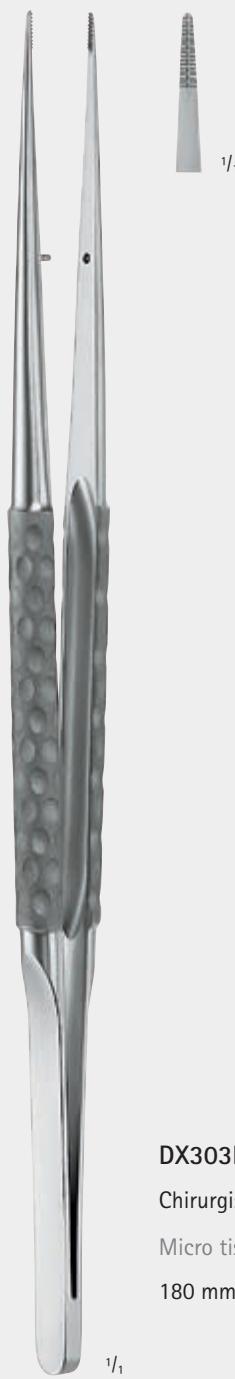
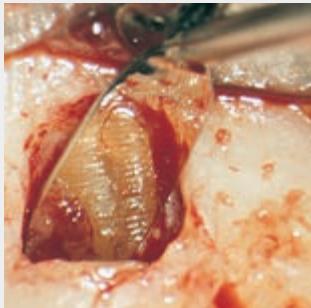
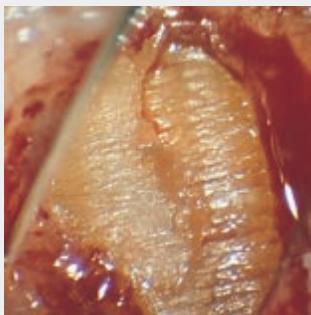
Use of the oval micro mirror (DA045R resp. DA046R). Fracture line barely detectable in low magnification (12x).

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Large magnification (25x) of the above illustration. The fracture line is visible in the center of the illustration.

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Use of the round micro mirror (DA043R resp. DA044R). View of prepared retrograde cavity with the post in the root canal.

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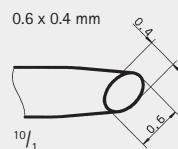
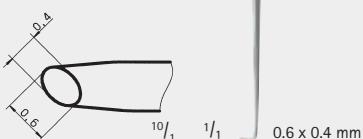
Oval retrograde cavity, seen after preparation of the isthmus between the root canals. The oval shape plugger (DE442R) is used to condense the retrograde filling material.

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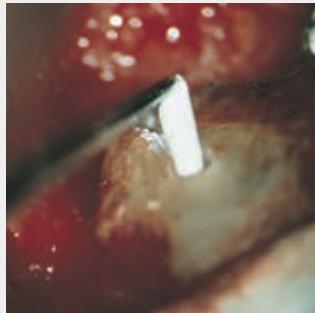
32 INCHES

A vertical ruler scale on the left side of the page, ranging from 1 to 8 inches. Each inch is divided into 32 smaller segments, though only the first few are explicitly labeled.

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Application of the retrograde
filling material into the cavity
with the micro spatula
(DE440R resp. DE441R).

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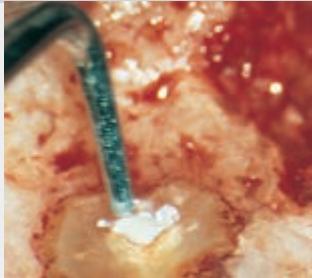
Ball burnisher/spatula,
Micro spatula, angulated

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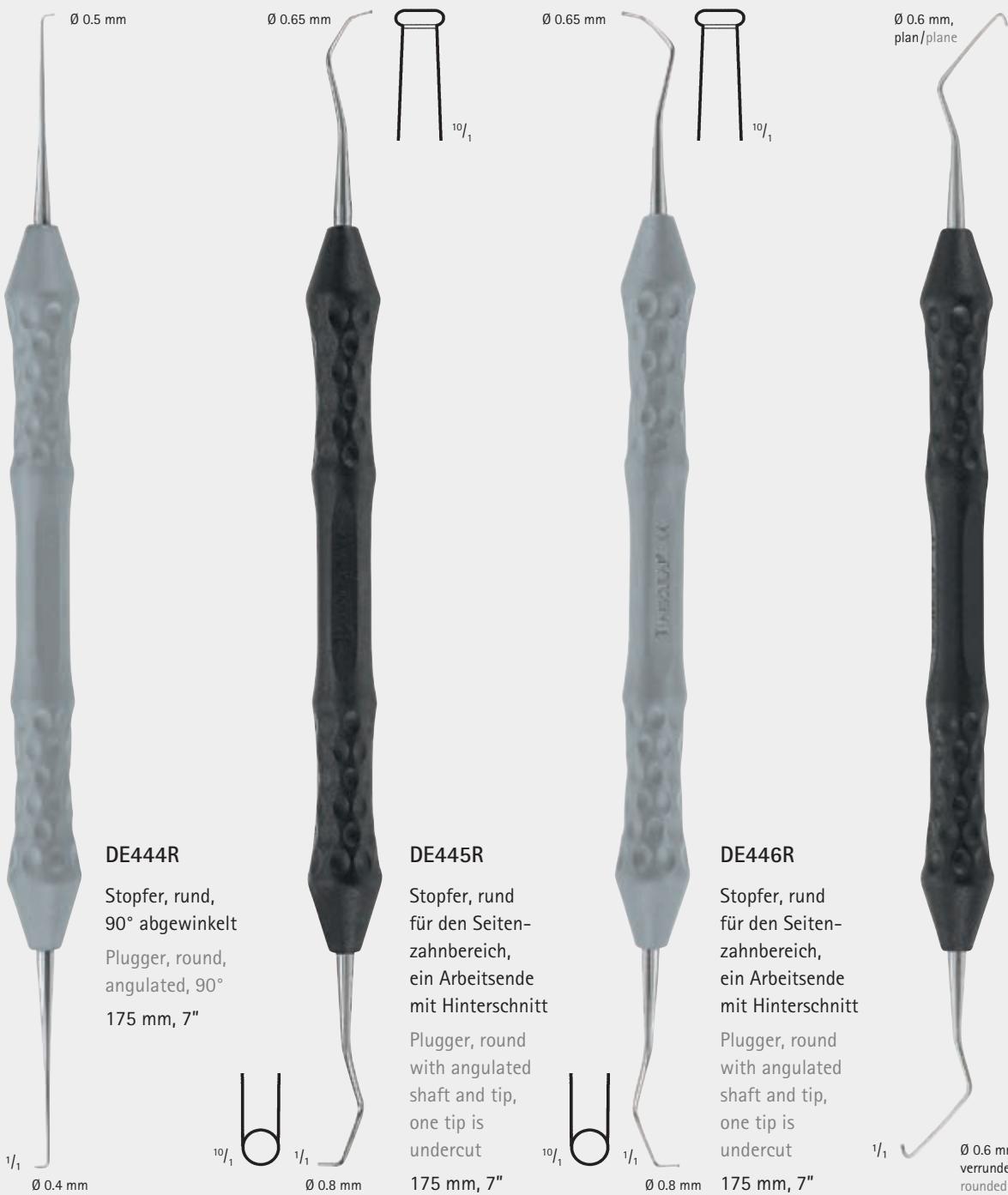
Condensation of the filling
material with the round plunger
(DE444R resp. DE445R or
DE443R) (in round canals).

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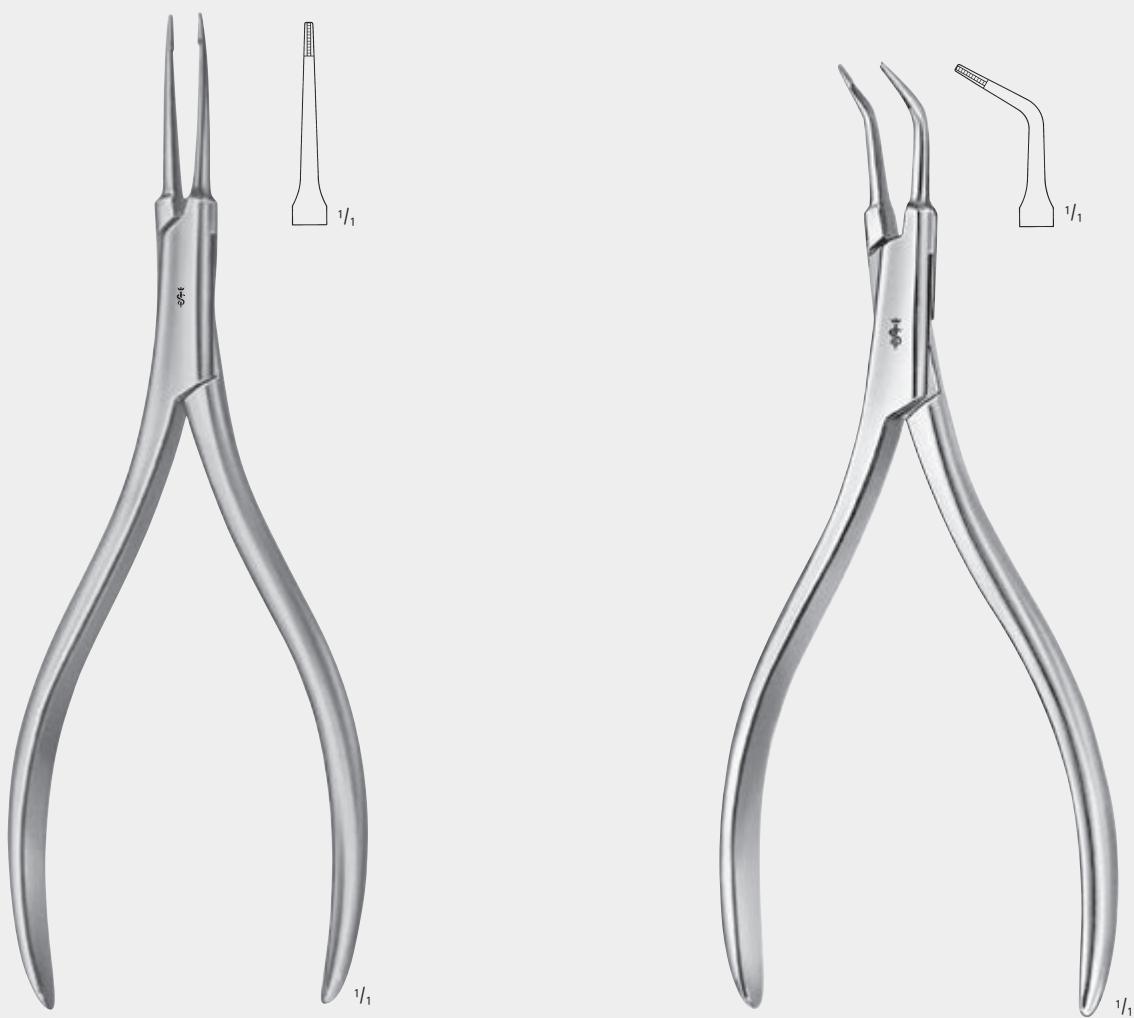
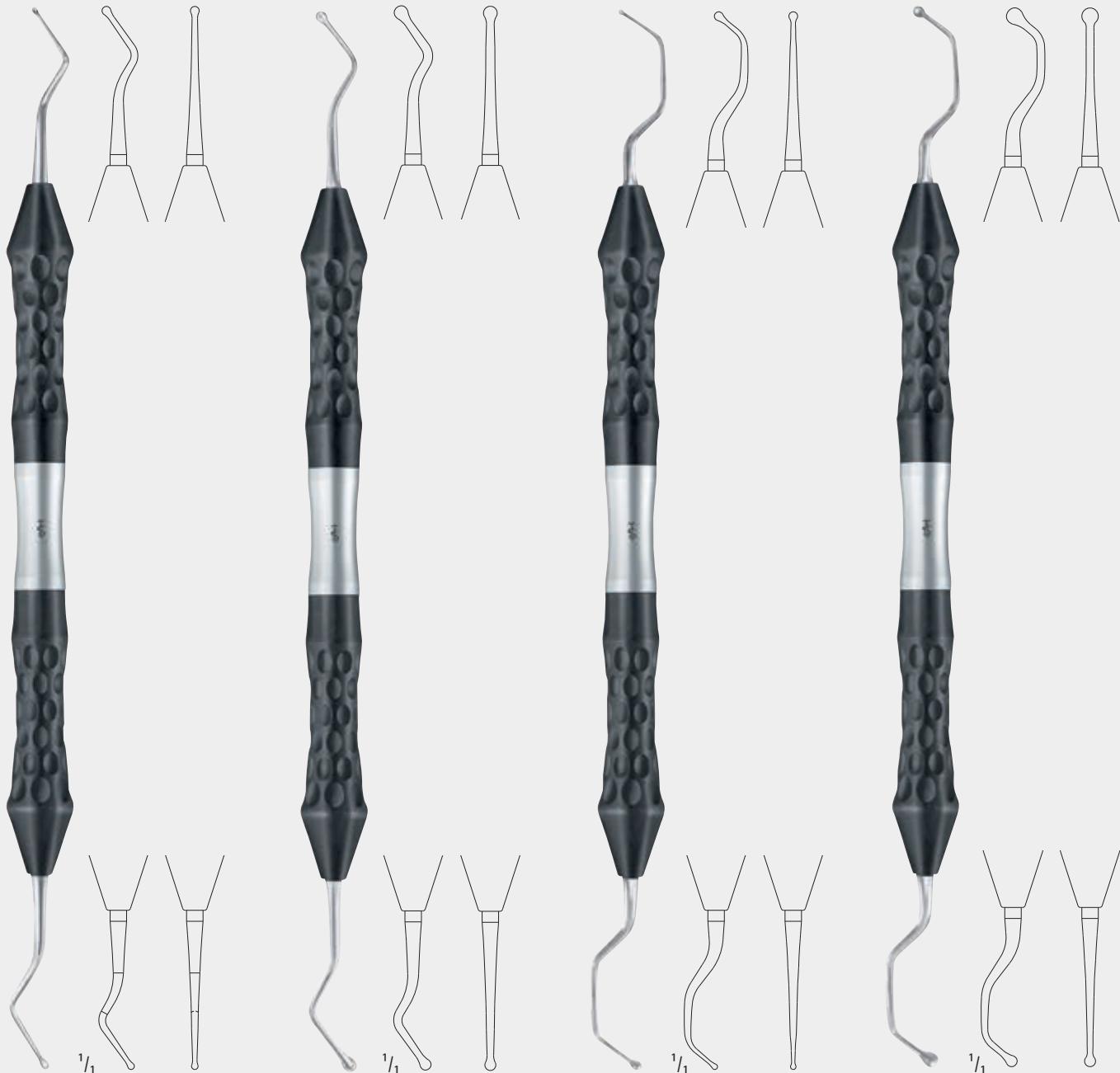
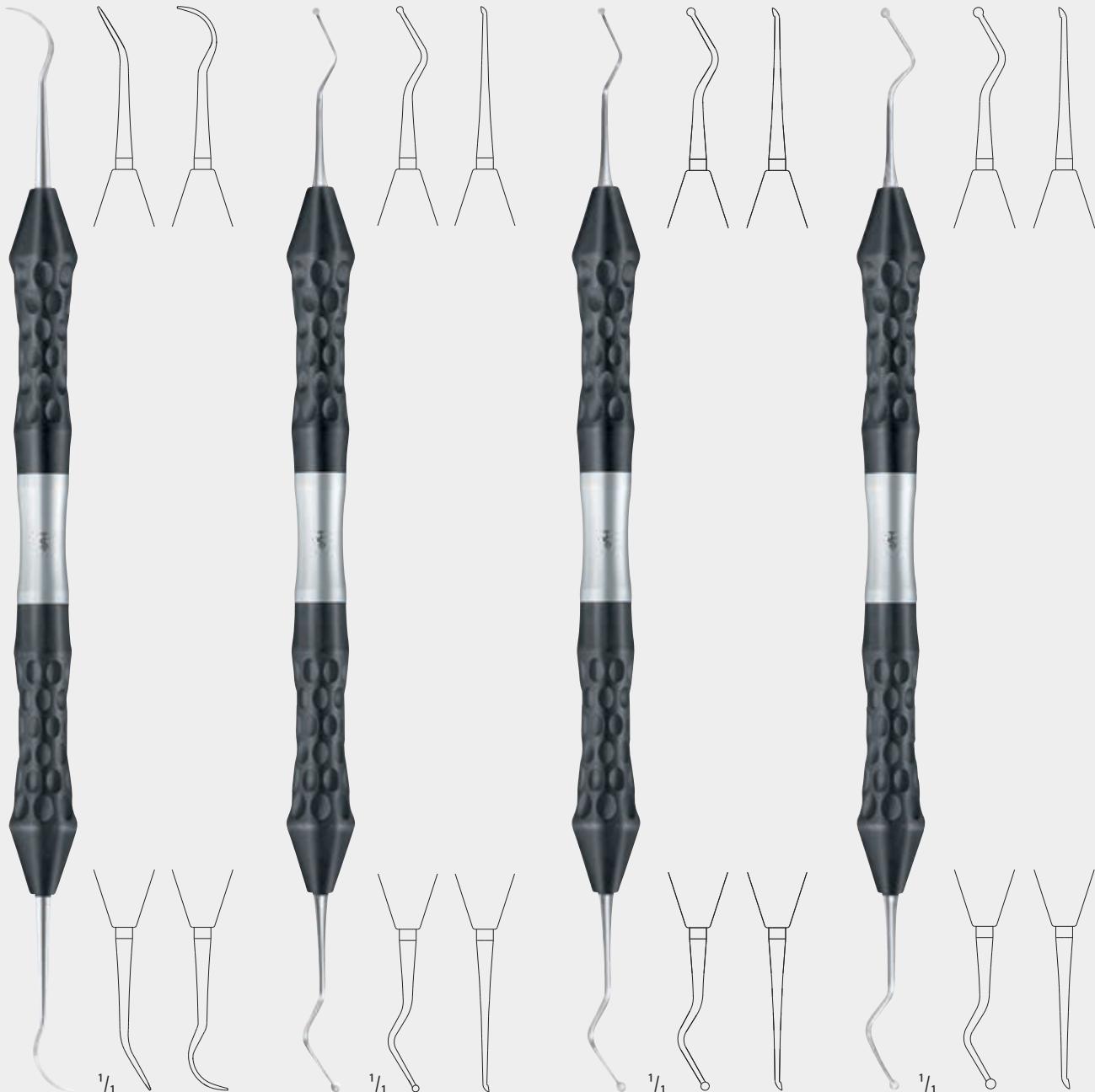
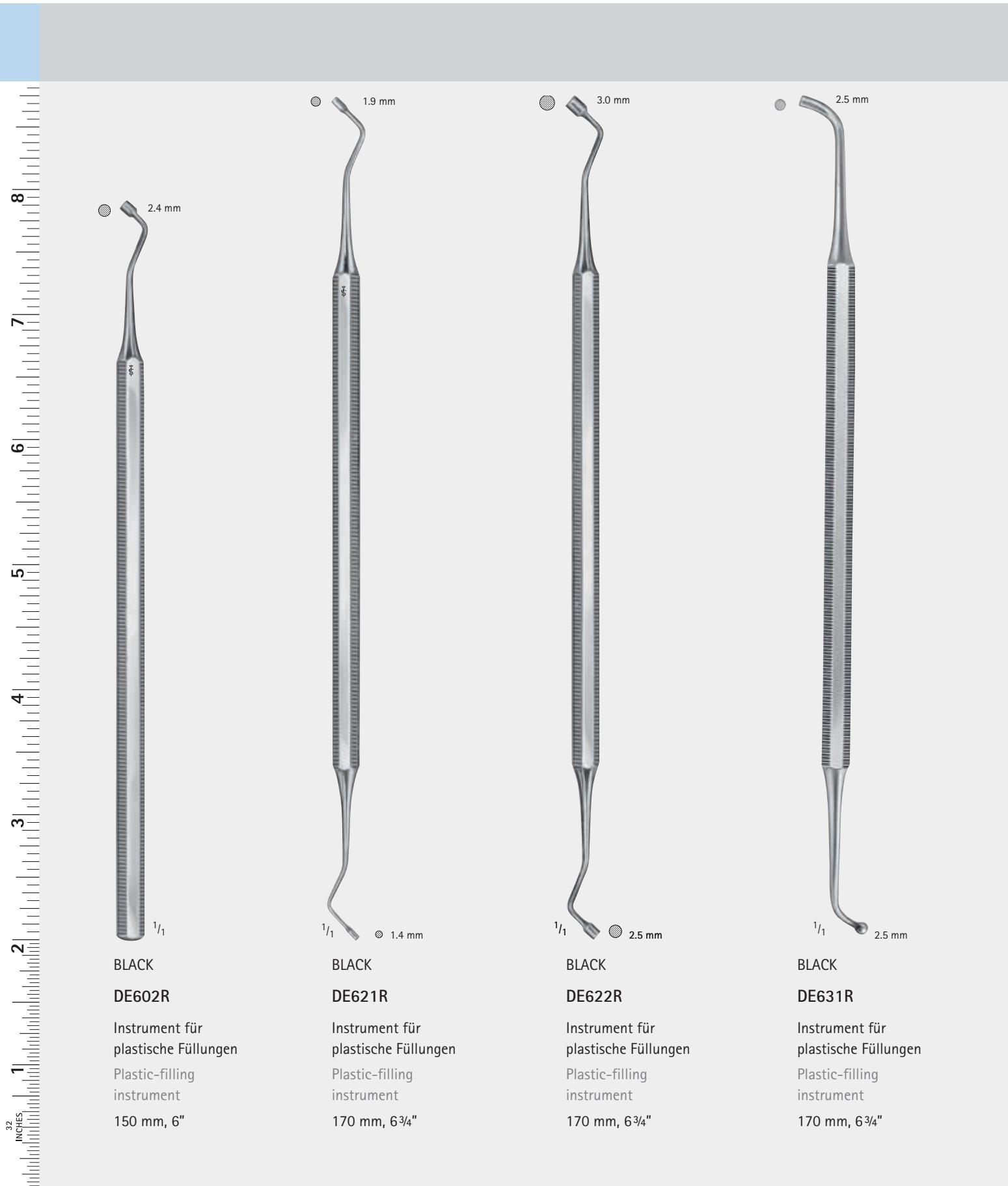
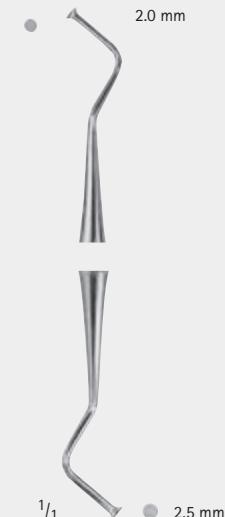
For applying lower filling material
and protective varnish out of calcium
hydroxide and glass ionomers.

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For applying medicaments, lining of
cavities and condensing of amalgam
170 mm, 6 3/4"

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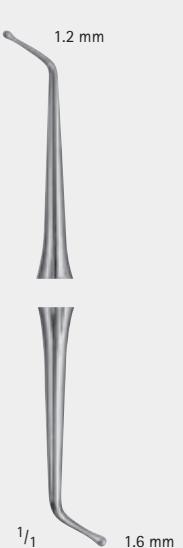
For applying and plugging
of lower filling material and
plastic filling material

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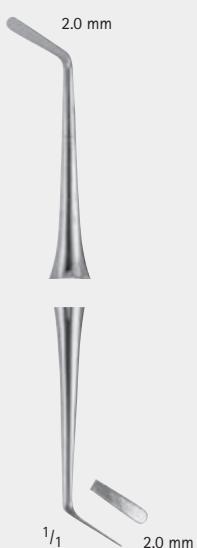
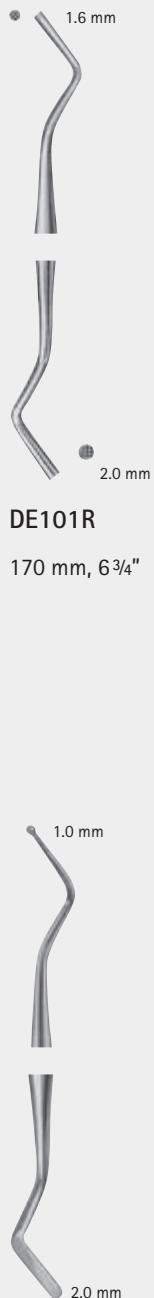
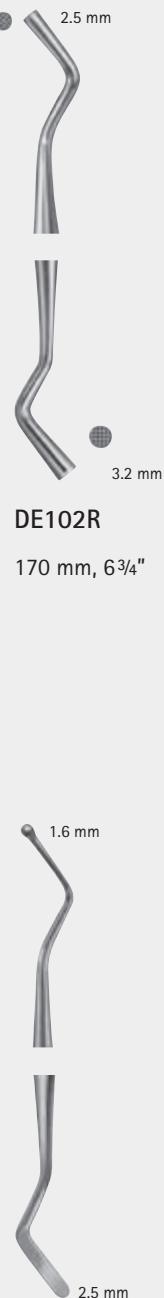
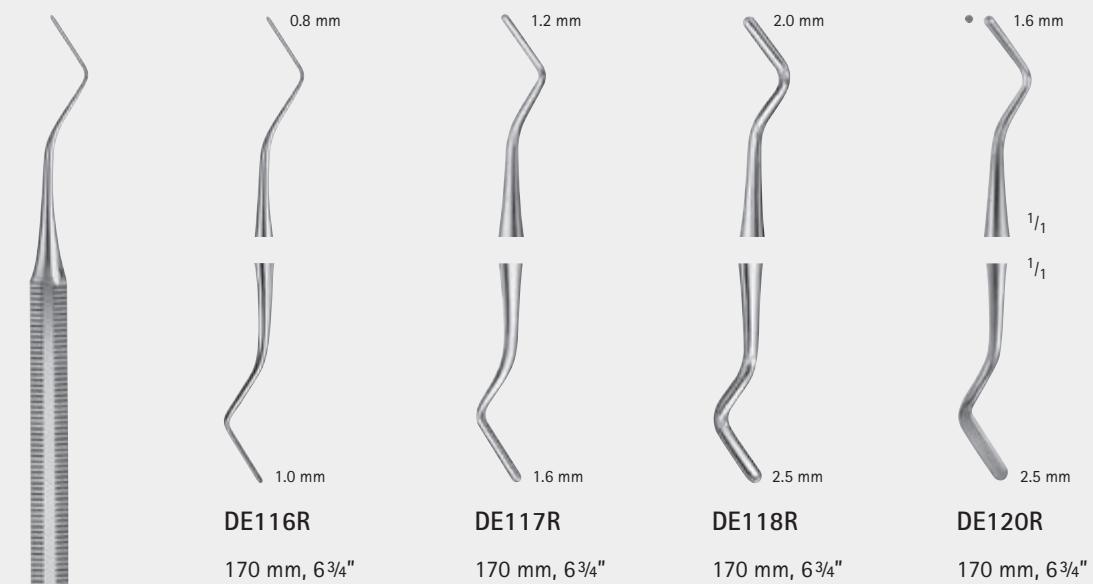
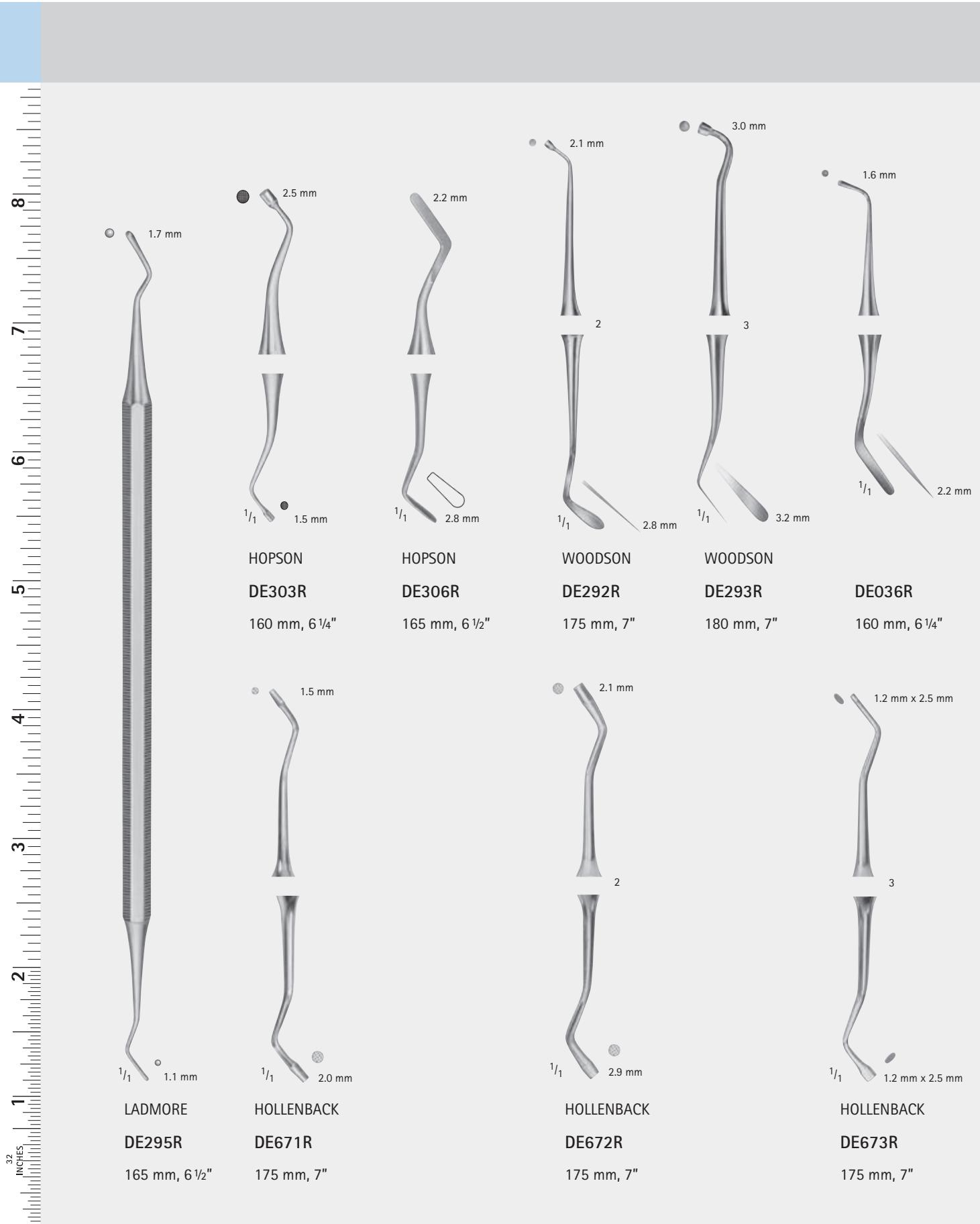
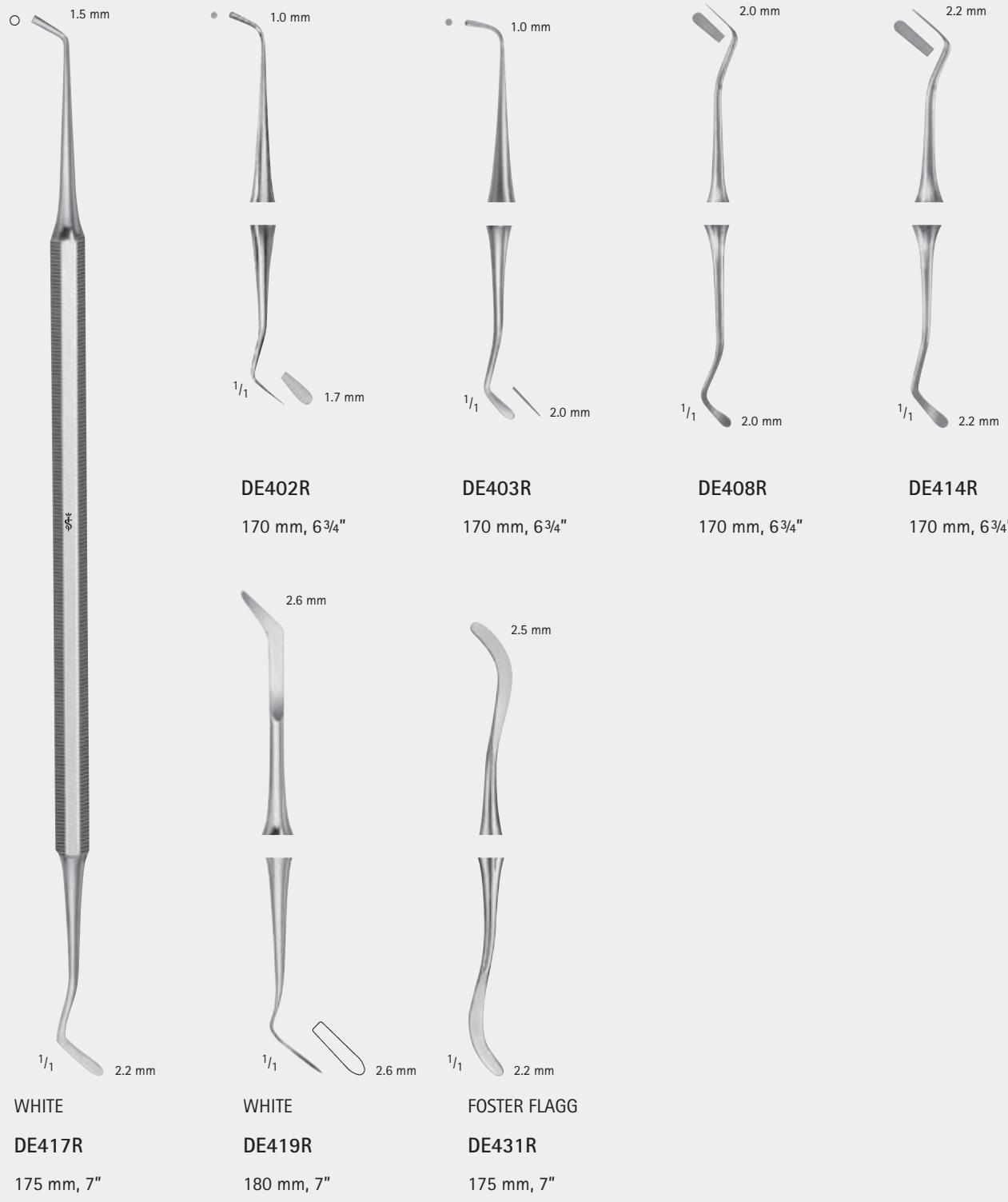
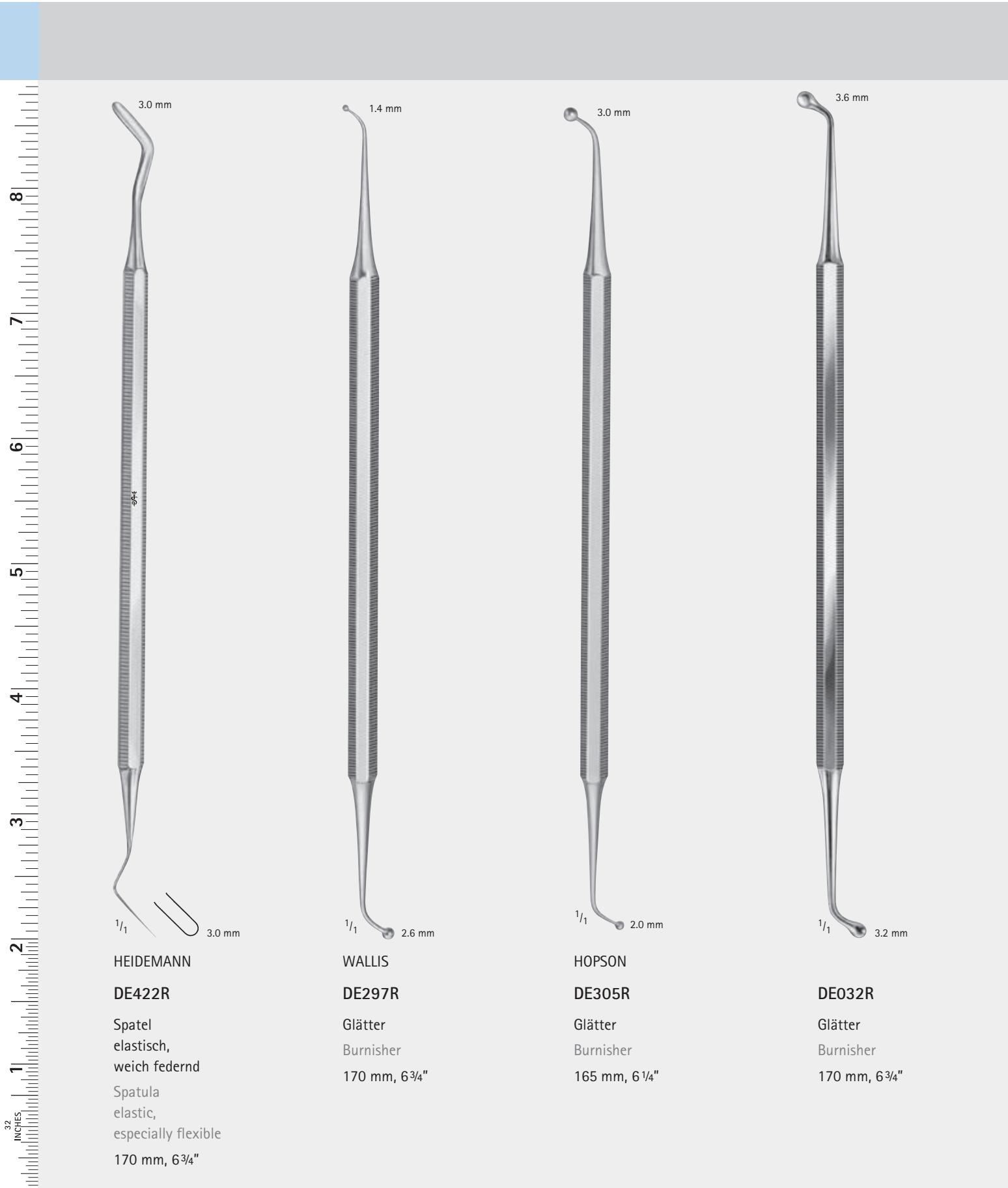
32 INCHES

WHITE
DF028R
Glätter
Burnisher
150 mm, 6"

HOLLENBACK
DE676R
Modellier Instrument
Finishing instrument
170 mm, 6 3/4"

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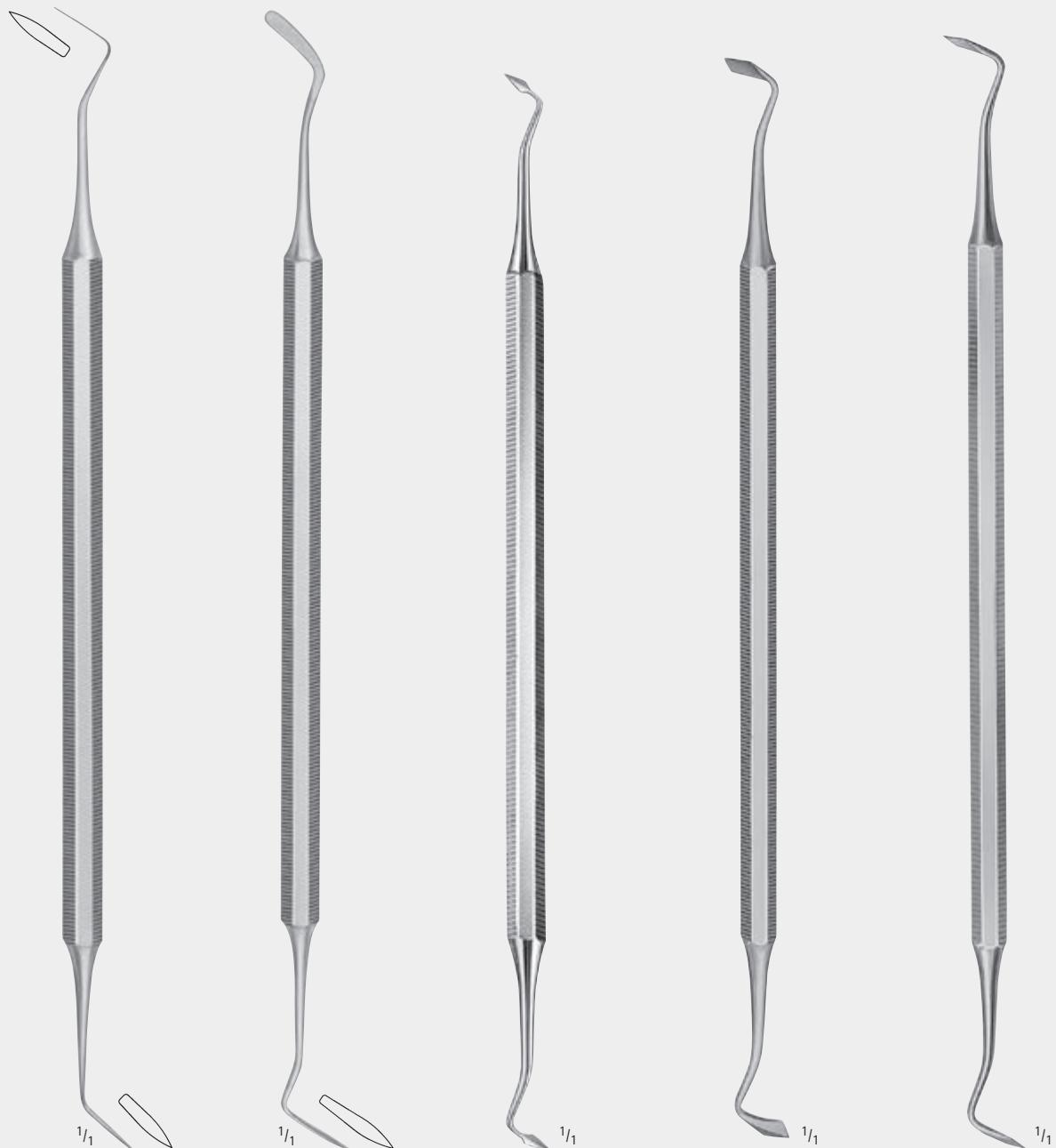
WARD
DE425R
 Modellier Instrument
 Finishing instrument
 170 mm, 6 3/4"

WARD
DE426R
 Modellier Instrument
 Finishing instrument
 170 mm, 6 3/4"

DF060R
 Modellier Instrument
 Finishing instrument
 160 mm, 6 1/4"

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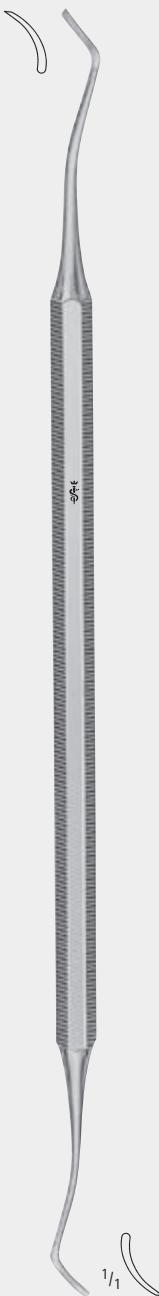
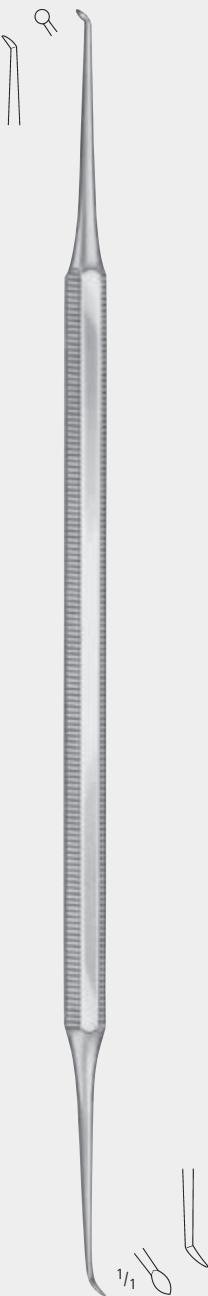
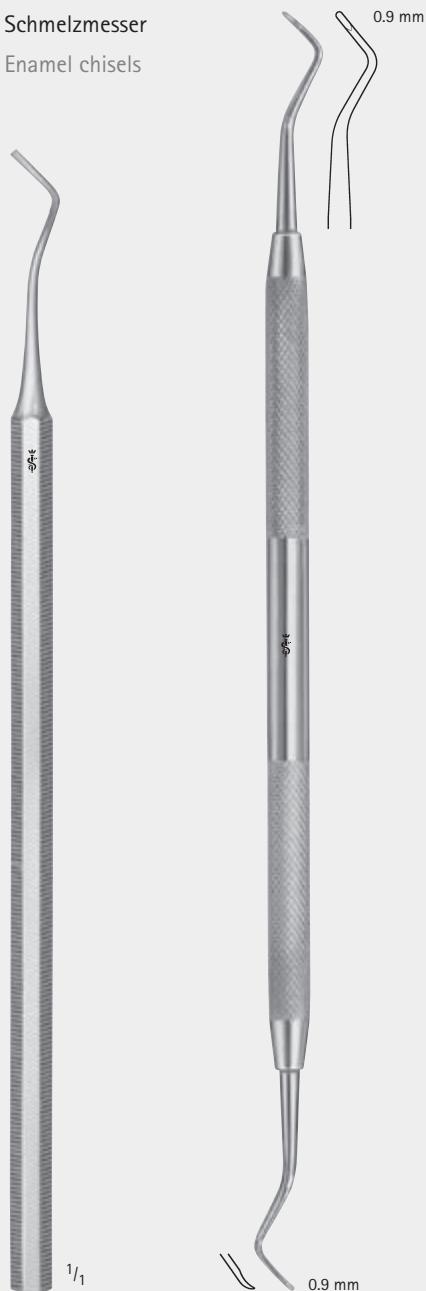
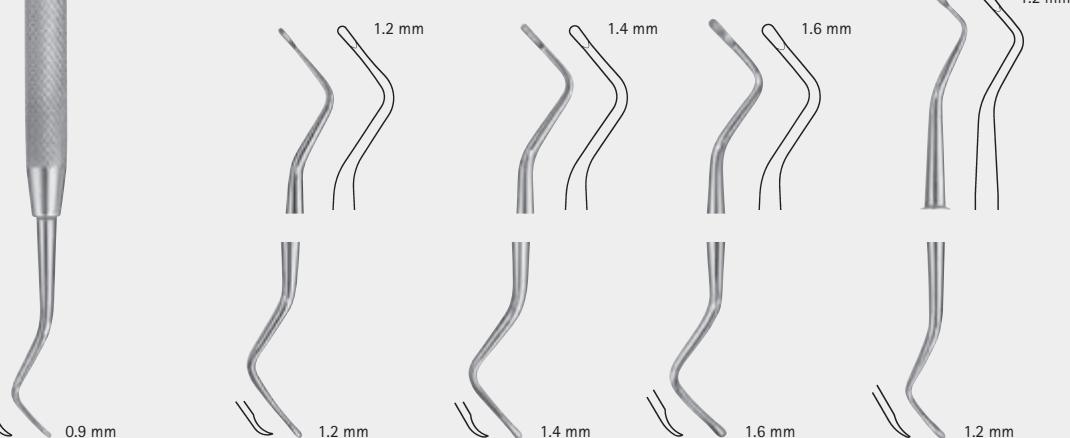
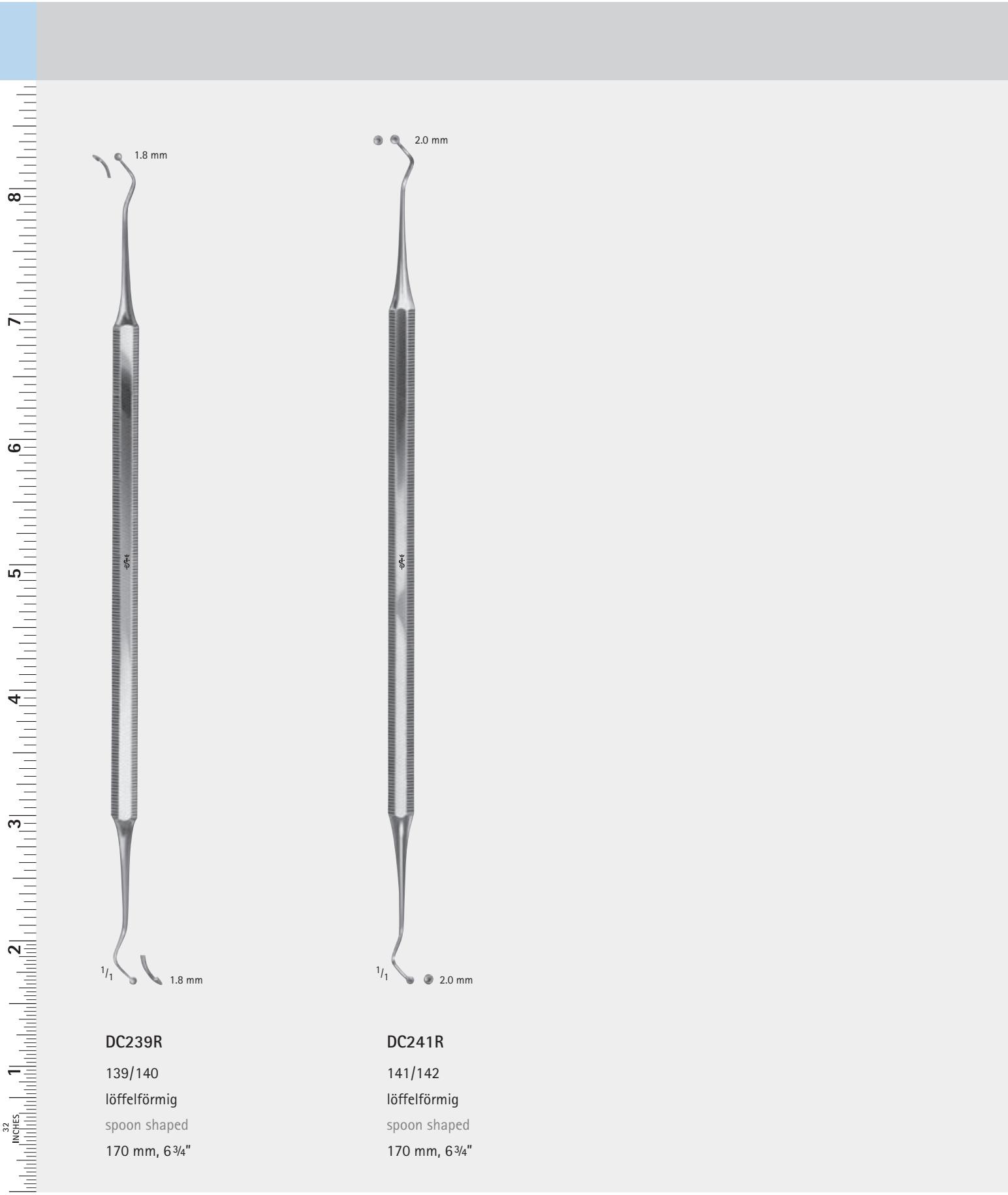
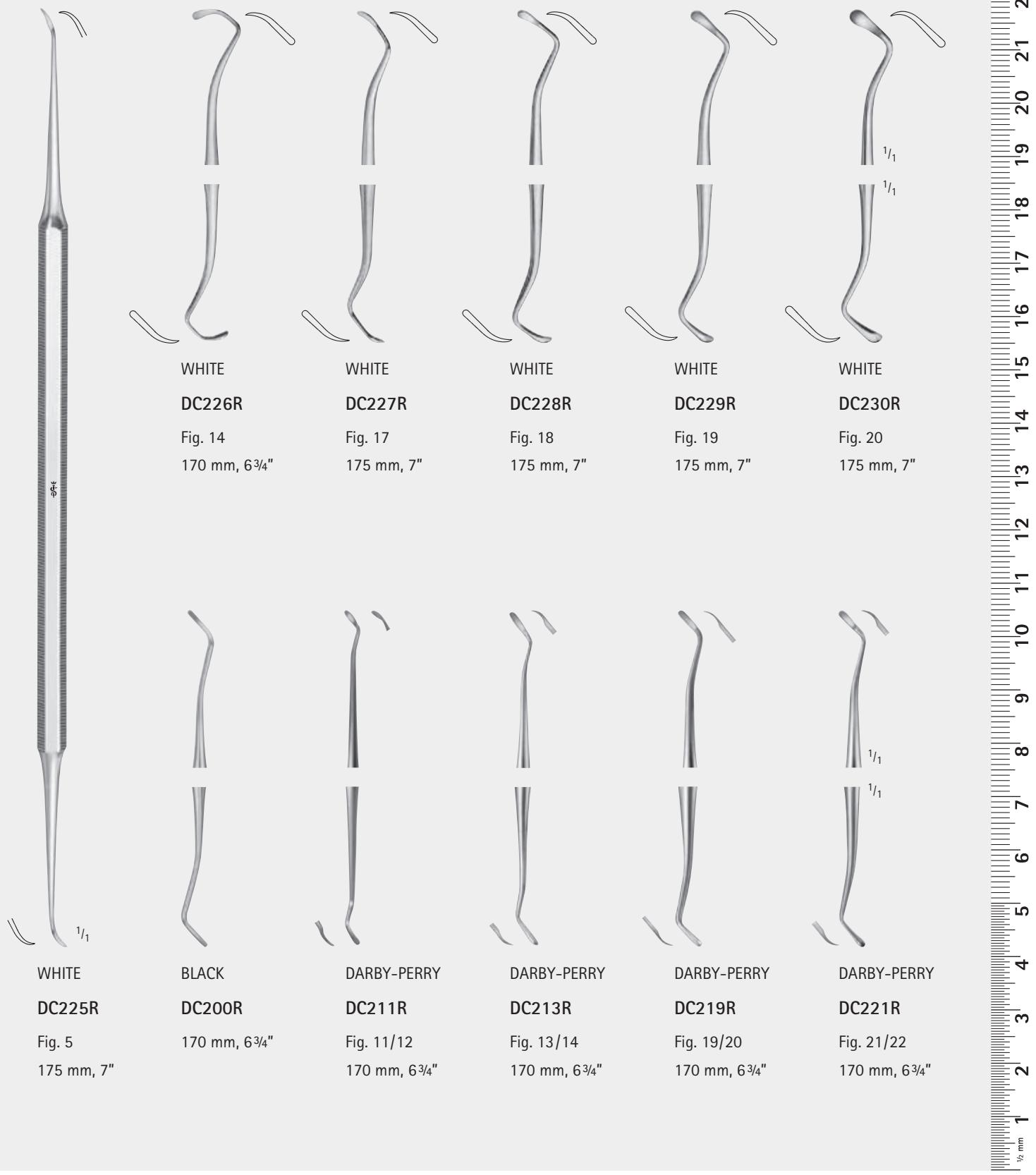
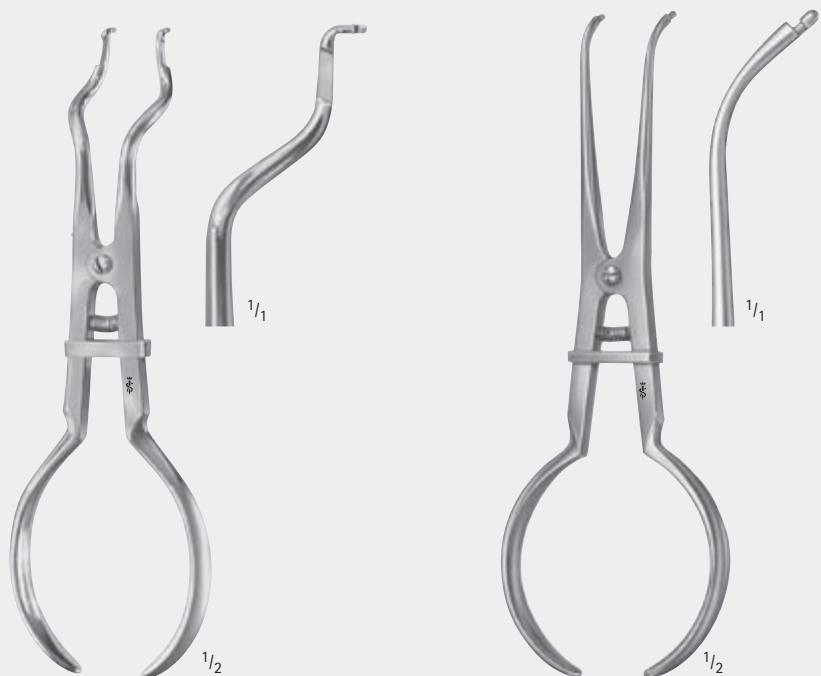
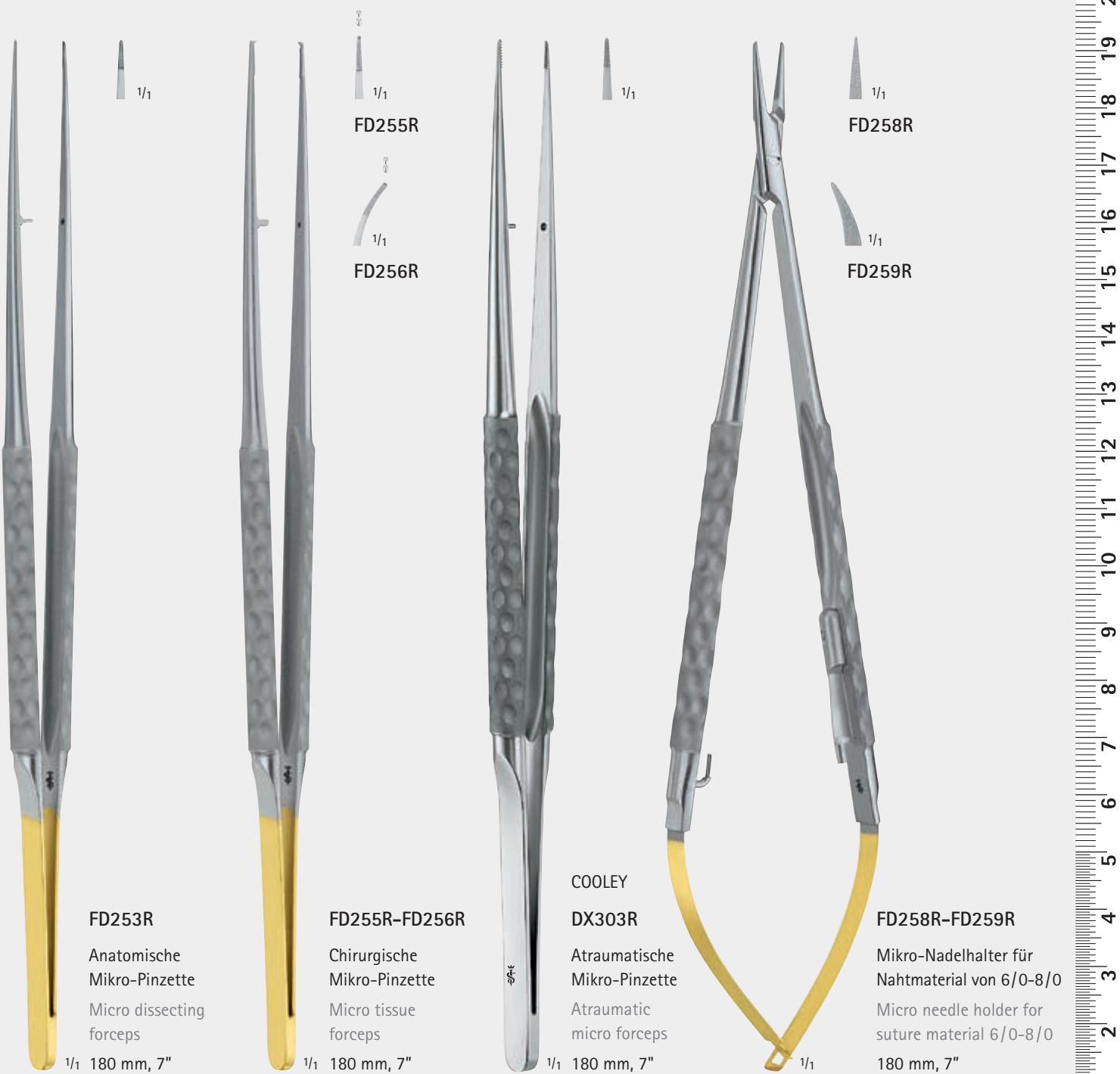
Titanium scalpel handle for
microsurgery blades, round
handle. Light, rotatable and
handy for precise cutting

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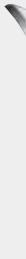
DENTAL SURGERY

Skalpellgriffe für sterile Skalpellklingen
BB510 - BB515, BB540, BB542

Scalpel handles for sterile scalpel blades
BB510 - BB515, BB540, BB542

vakuum-vergütet
heat-treated in
vacuum

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- Designed to comply with TRBA 250
- Prevention of infections caused by cuts and stab wounds suffered during hand-over, take-over and disposal of scalpels
- Blade can be locked in safe disposal position

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Safety scalpels
Sales unit: PAK = Package of 10 pieces

SICHERHEITSSKALPELLE
SAFETY SCALPELS

BA815-1SU

Fig. 15-1

BA815-CSU

Fig. 15-C

BA818SU

Fig. 18

BA819SU

Fig. 19

BA820SU

Fig. 20

BA836SU

Fig. 36

BA821SU

Fig. 21

BA822SU

Fig. 22

BA823SU

Fig. 23

BA824SU

Fig. 24

BA825SU

Fig. 25

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DENTAL SURGERY

BA210
Fig. 10

BA211
Fig. 11

BA212
Fig. 12

BA215
Fig. 15

BA215-C
Fig. 15-C

BA218
Fig. 18

BA224
Fig. 24

BA219
Fig. 19

BA220
Fig. 20

BA221
Fig. 21

BA222
Fig. 22

BA223
Fig. 23

BA210 – BA224

Sterile scalpels with plastic handle and blade protector
Sales unit: PAK = Package of 10 pieces

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Silicone mouth wedge with metal core
 Size: M

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Silicone mouth wedge with metal core
 Size: L

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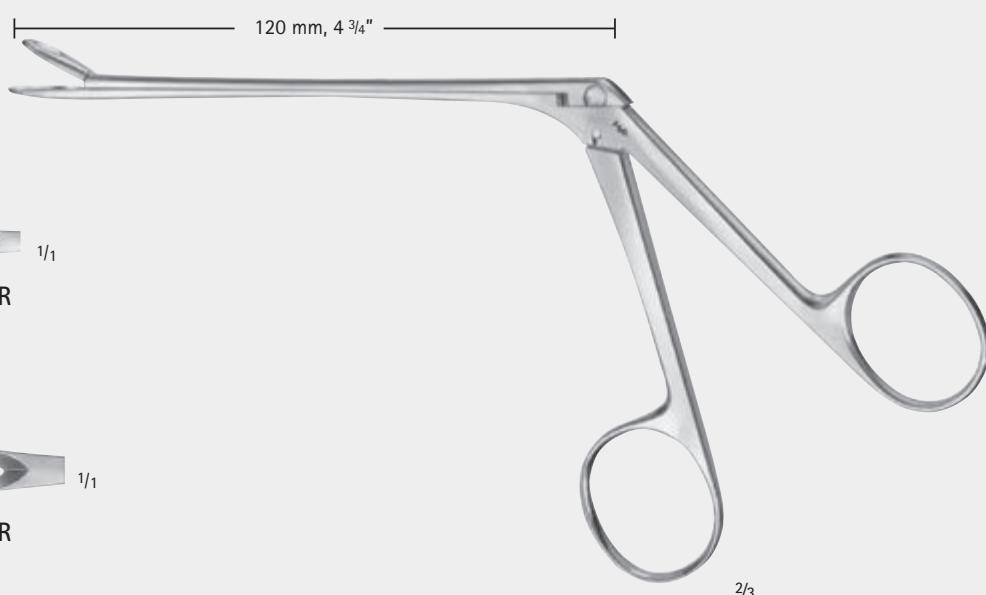
FF201R – FF202R

Hirnspatel, biegsam,
 doppelt endig

Brain spatula, malleable
 double-ended

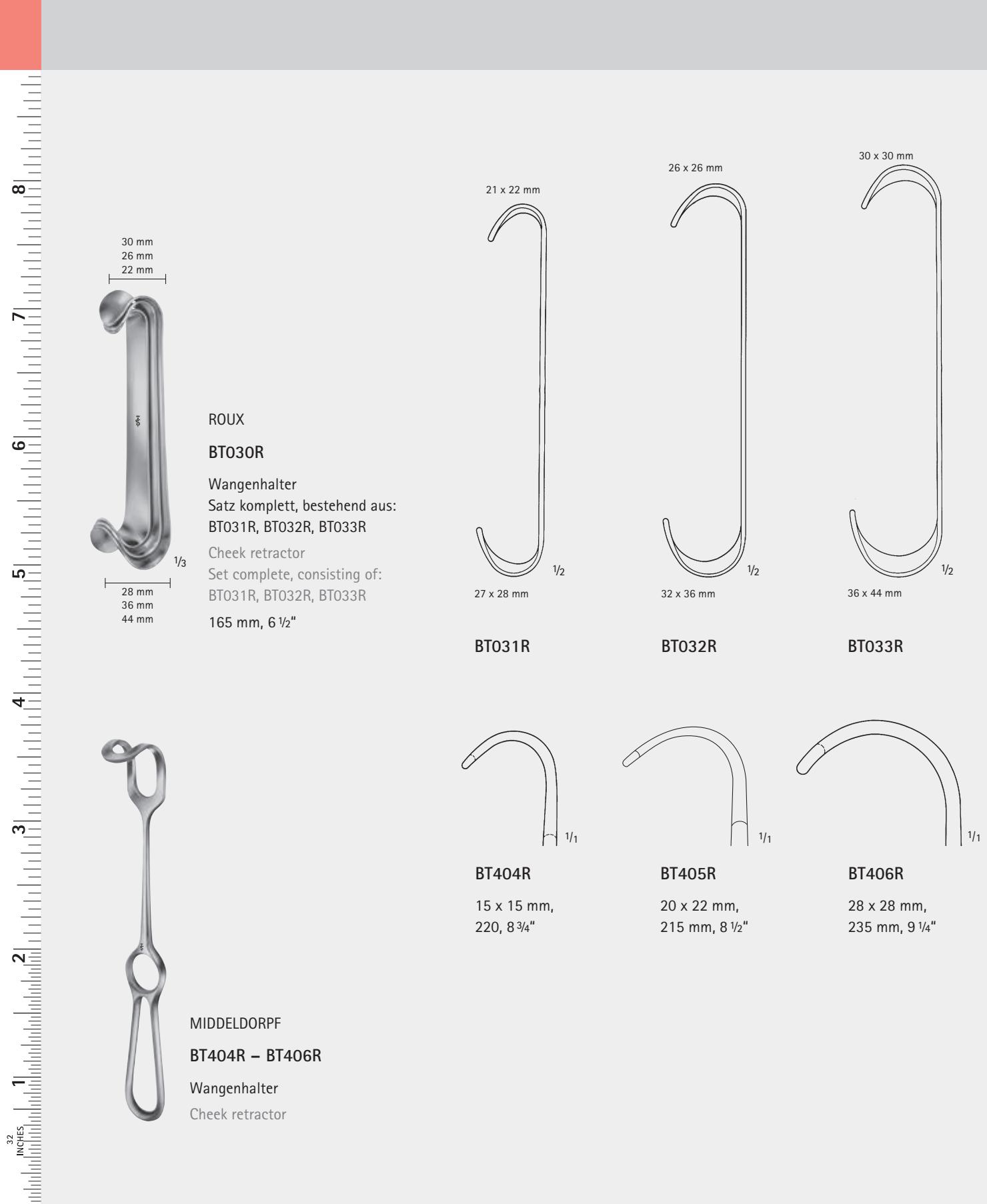
	Abmessungen Dimensions
FF201R	9 x 200 mm
FF202R	12 x 200 mm

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Retractors for retrograde root resection

Before the use of the retractor, 2 slight deepenings are burred in the jaw. The 2 tips at the working end of D0373R are inserted and will therefore also provide a reliable grip.

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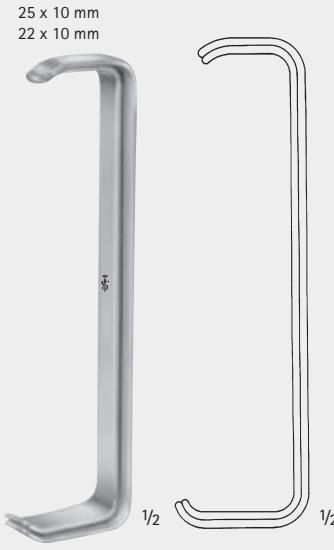
DENTAL SURGERY

8
7
6
5
4
3
2
1
32 INCHES

25 x 10 mm
22 x 10 mm

28 x 12 mm
32 x 12 mm

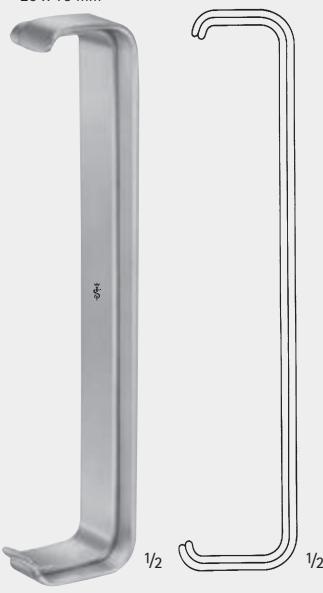
FARABEUF

BT020R

Wundhaken
Retractor
125 mm, 5"

23 x 16 mm
20 x 16 mm

24 x 16 mm
28 x 16 mm

FARABEUF

BT021R

Wundhaken
Retractor
150 mm, 6"

BT311R
30 x 10 mm

BT315R
40 x 10 mm

BT316R
43 x 13 mm

LANGENBECK

BT311R - BT316R

Wundhaken
Retractor
230 mm, 9"

BT354R
25 x 6 mm

BT356R
25 x 8 mm

BT357R
35 x 11 mm

BT358R
35 x 15 mm

BT359R
41 x 11 mm

BT361R
55 x 11 mm

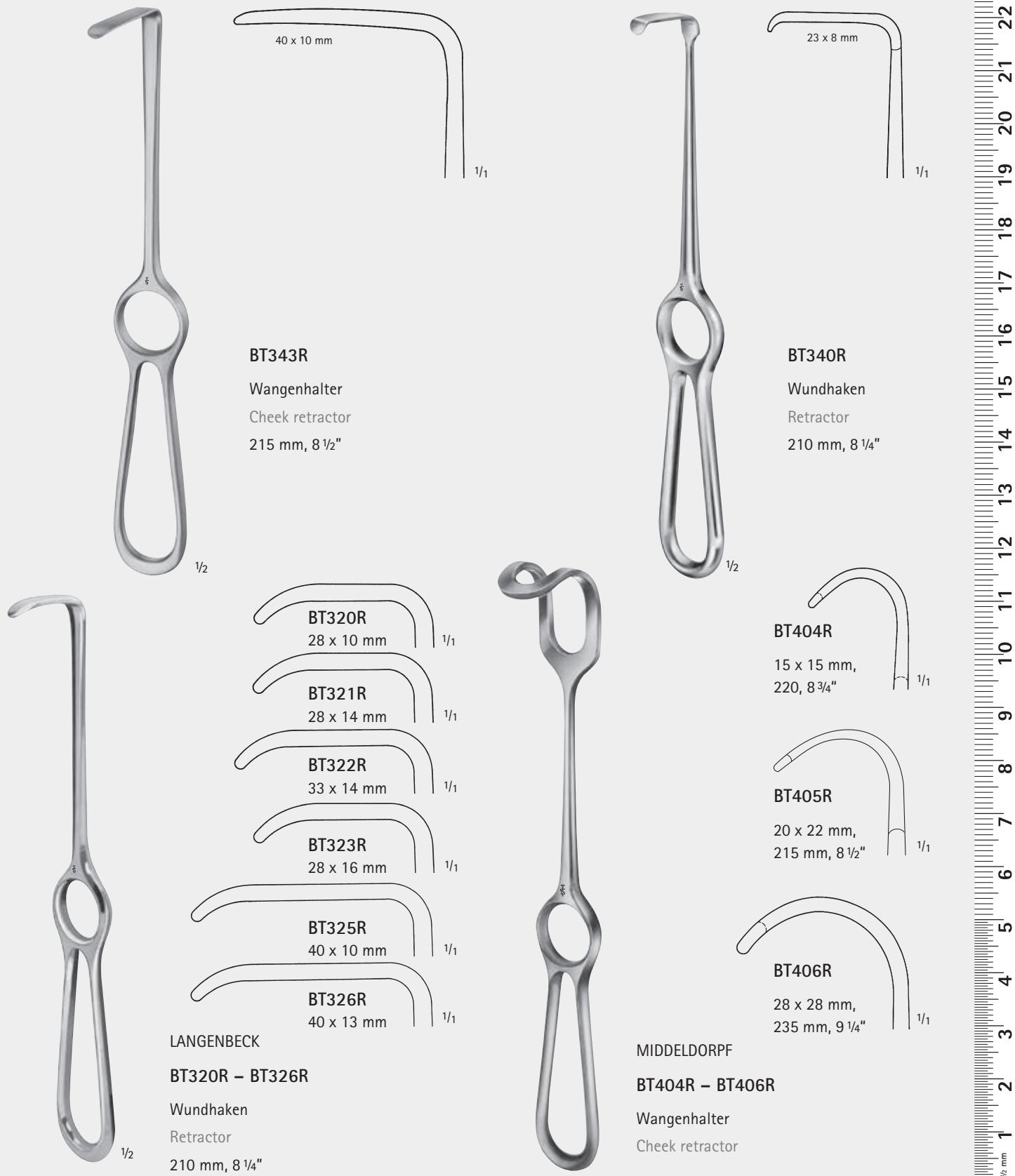
KOCHER-LANGENBECK

BT354R - BT361R

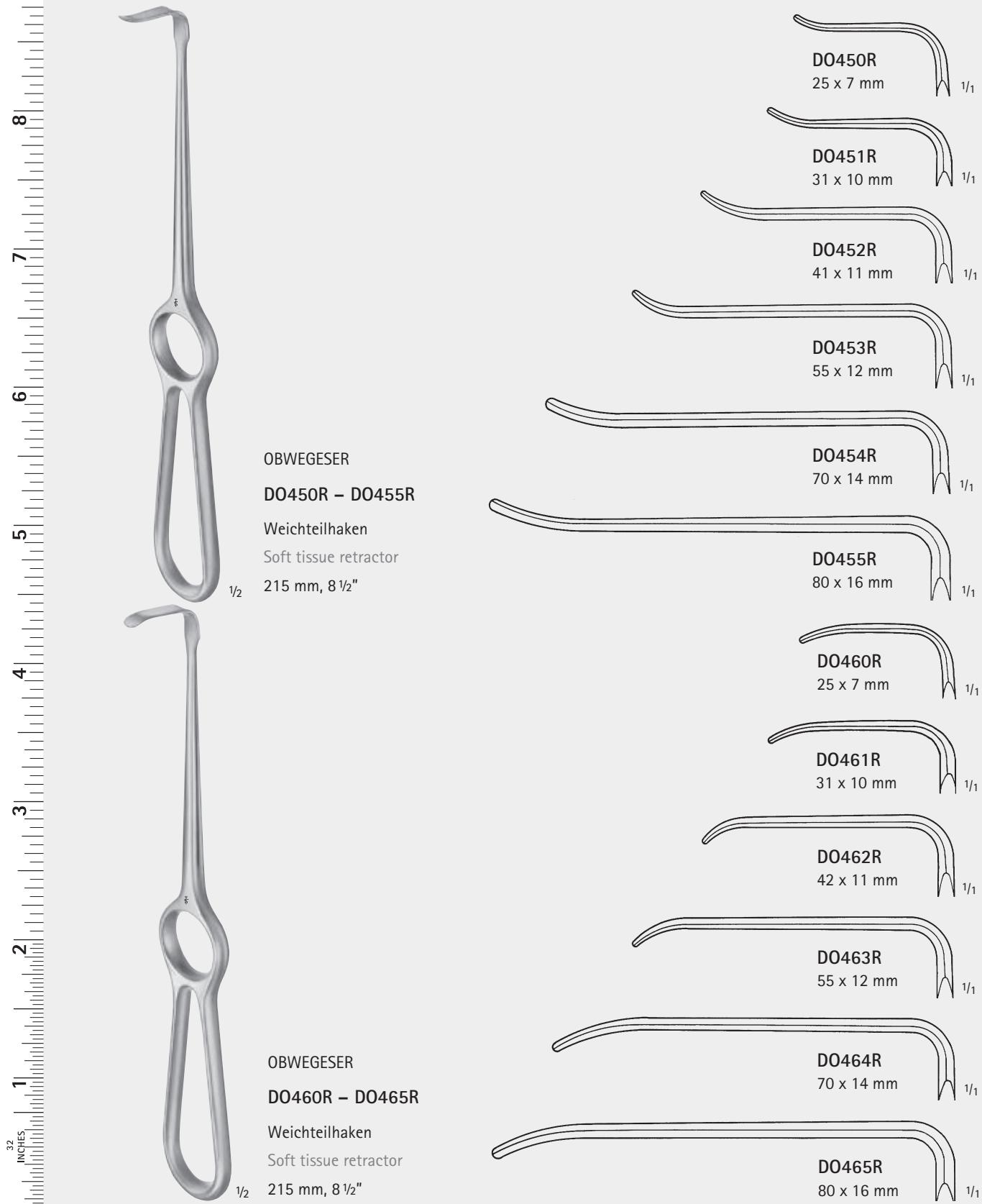
Wundhaken
Retractor
215 mm, 8 1/2"

WUNDHAKEN, WANGENHALTER
RETRACTORS, CHEEK RETRACTORS

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Especially designed for upper impacted wisdom teeth. The design of the retractor enables the retraction of the cheek and mucoperiostal flap as well as one hand simultaneous support of retractor and an additional instrument.

LASTER
DO472R
Wangenhaken
Cheek retractor
100 mm, 4"

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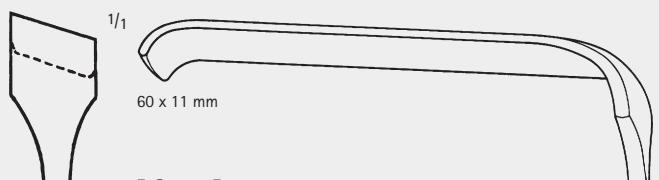
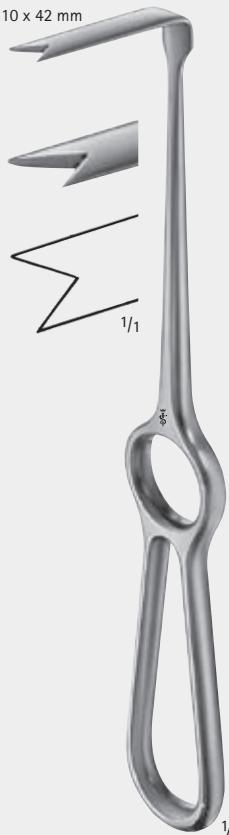
SOFT TISSUE RETRACTORS, ERGOPLANT SOFT TISSUE RETRACTORS, DIAGONAL CHEEK RETRACTORS, RETRACTORS

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Ramus retractor
for separation of
ramus and tissue

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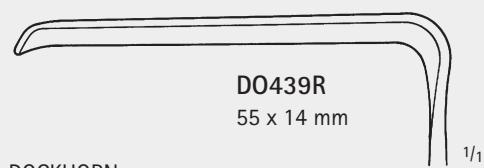
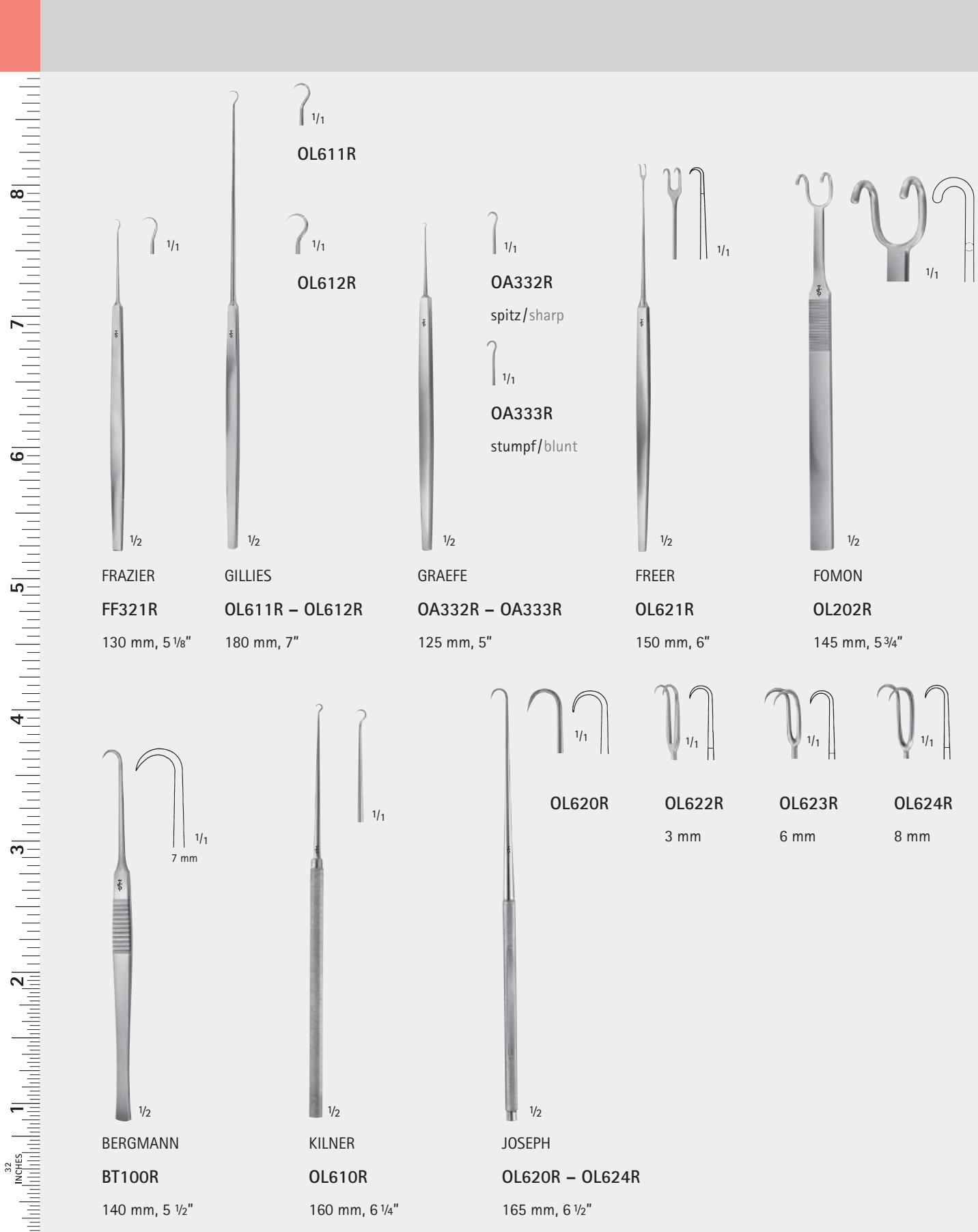
200 mm, 8"

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DENTAL SURGERY

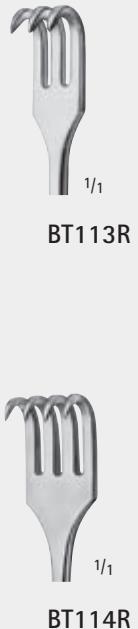
WUNDHAKEN, FEINE WUNDHÄKCHEN
RETRACTORS, FINE WOUND RETRACTORS

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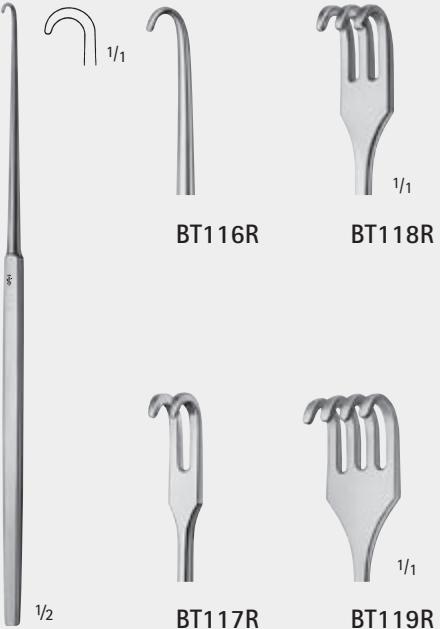
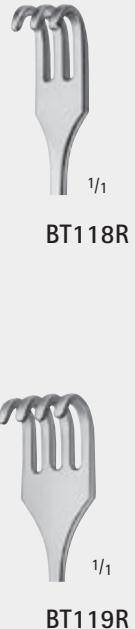
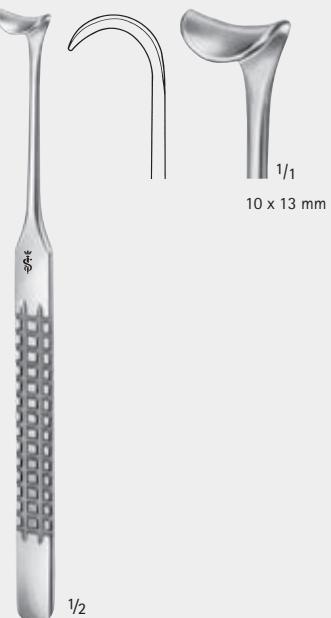
Retraction thread packer,
flexible

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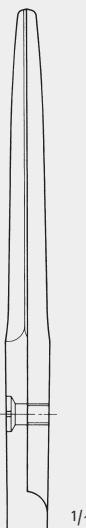
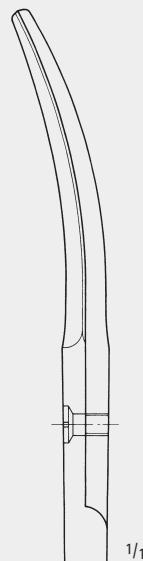
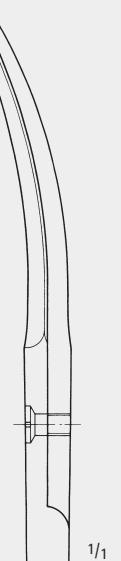
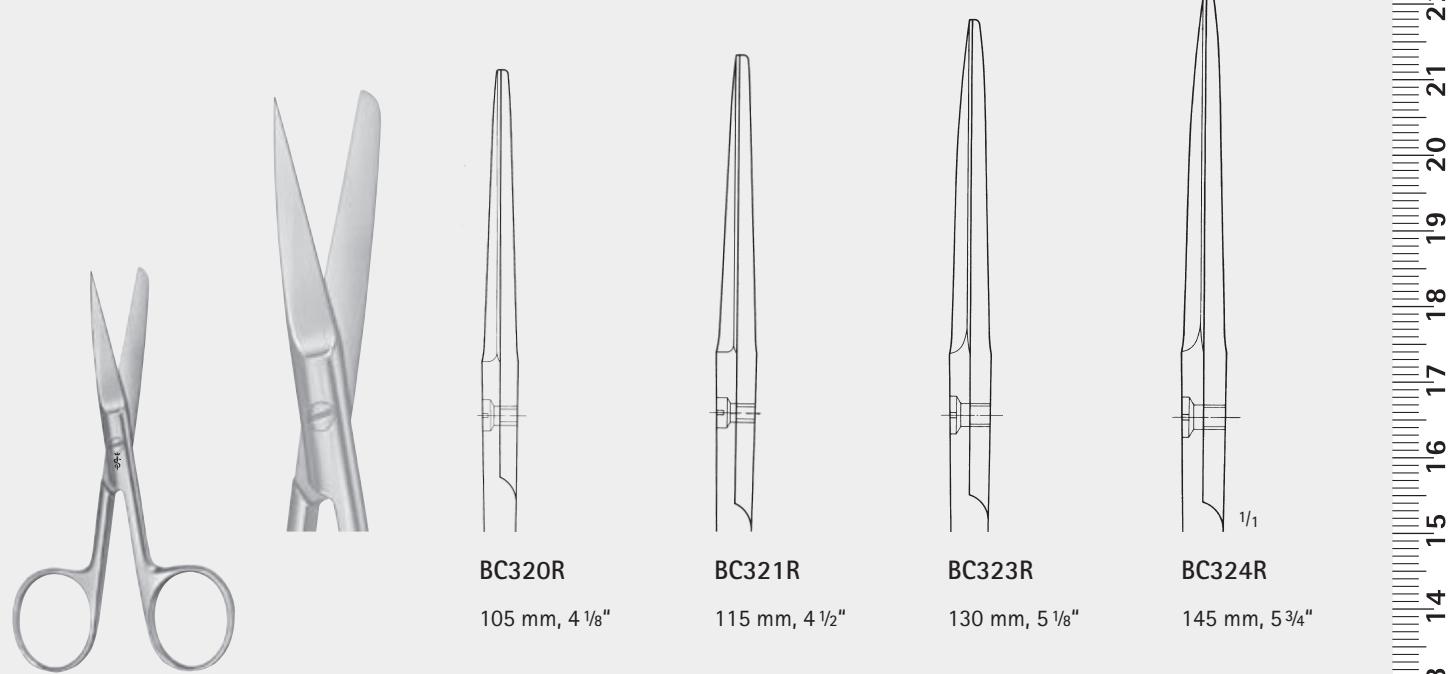
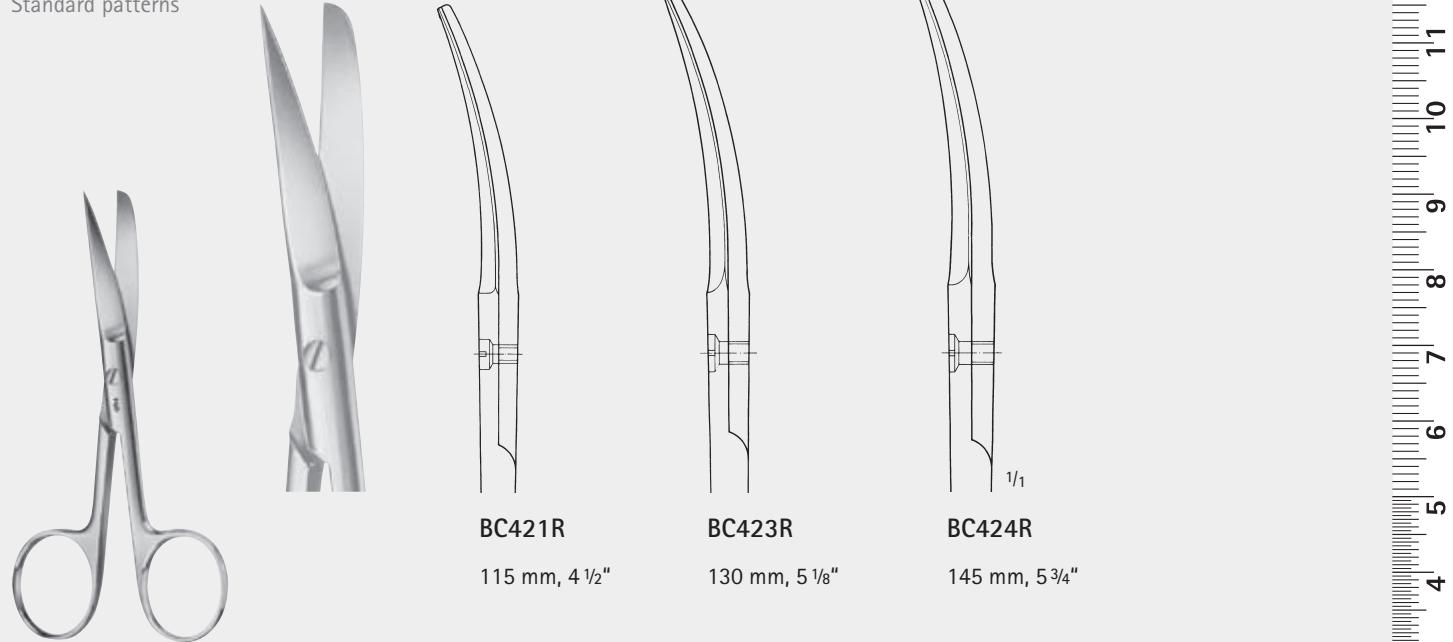
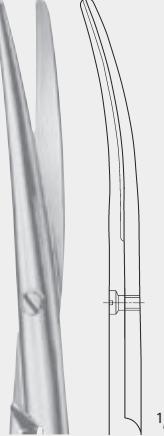
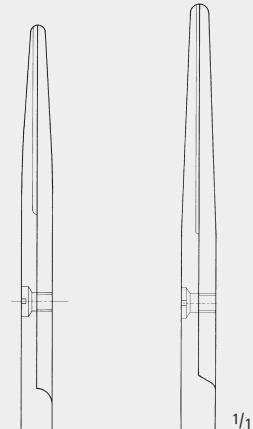
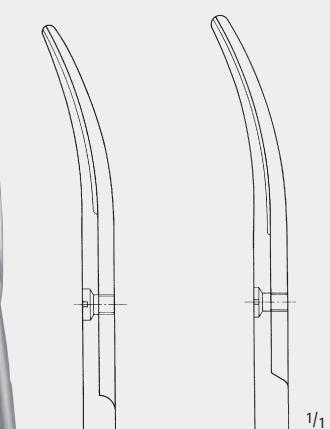
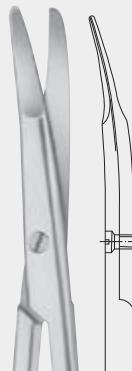
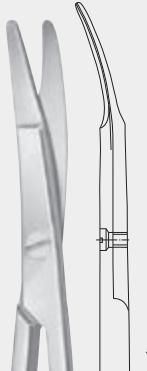
Special flat points for dissecting and undermining the skin

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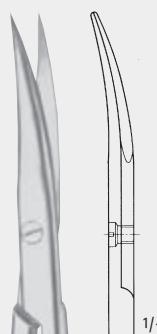
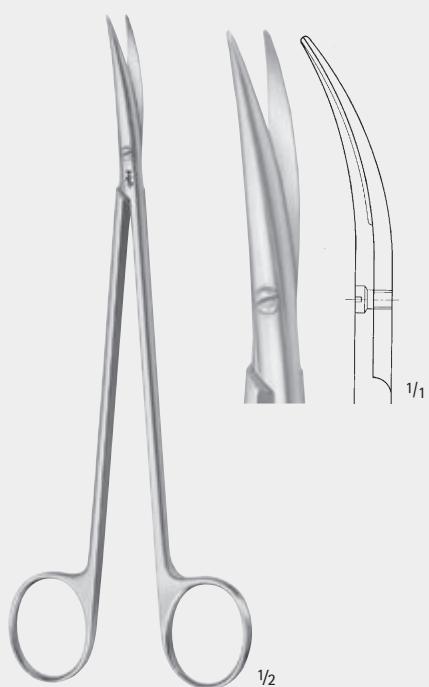
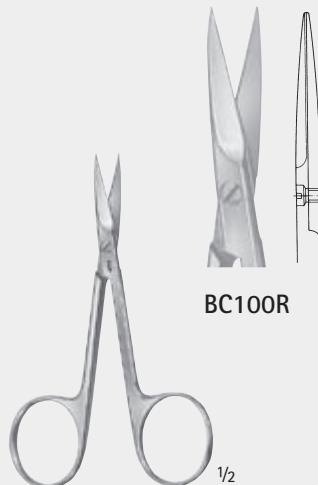
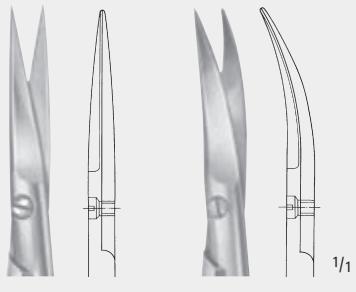
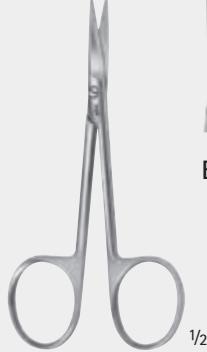
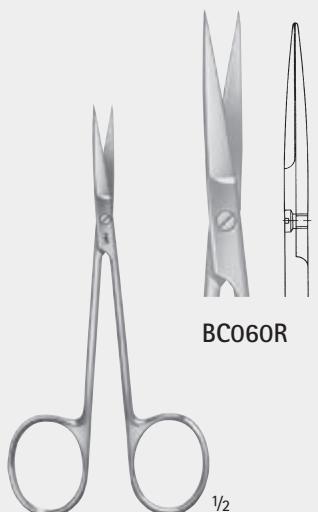
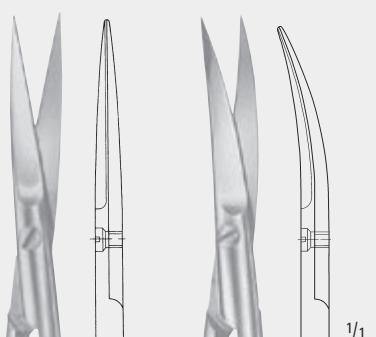
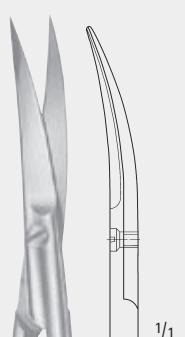
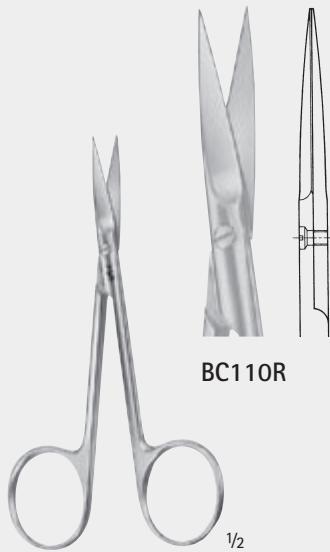
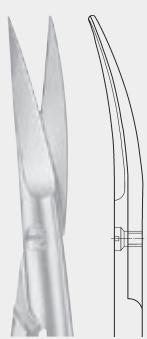
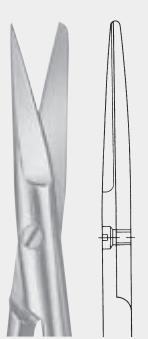
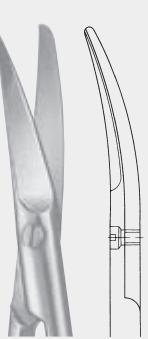
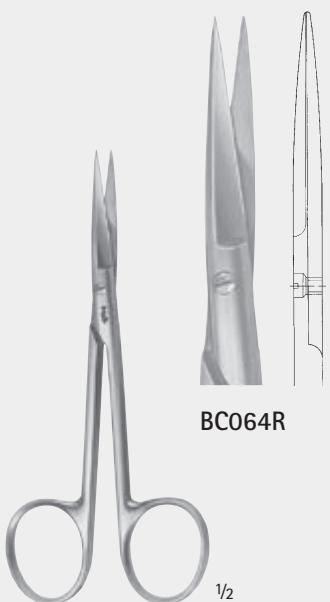
Curved to side,
especially suited to
remove ligatures

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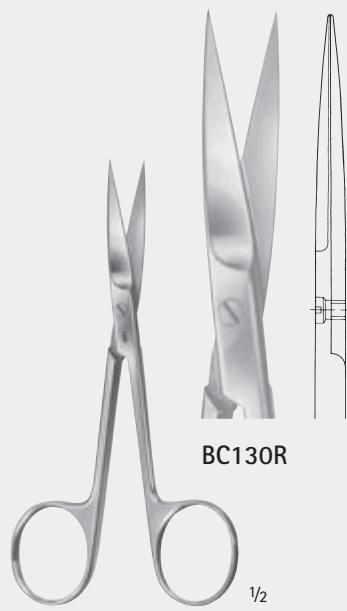
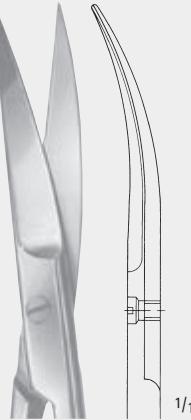
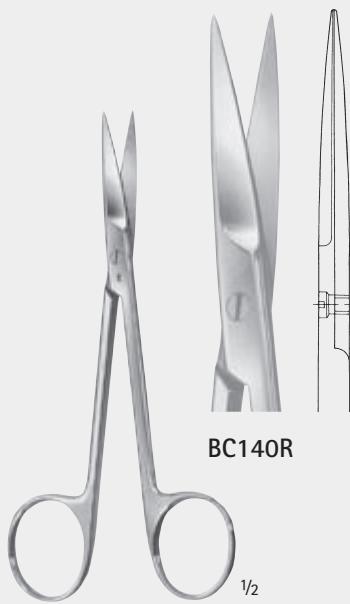
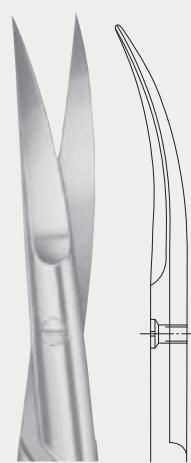
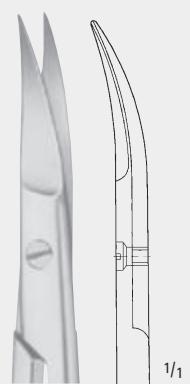
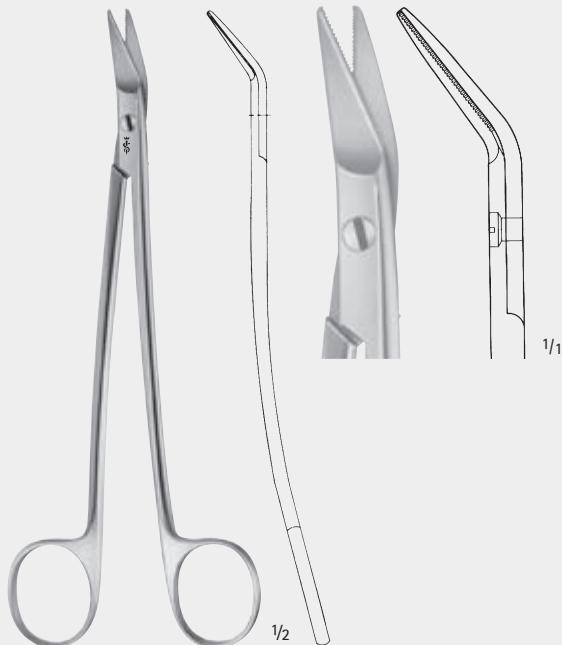
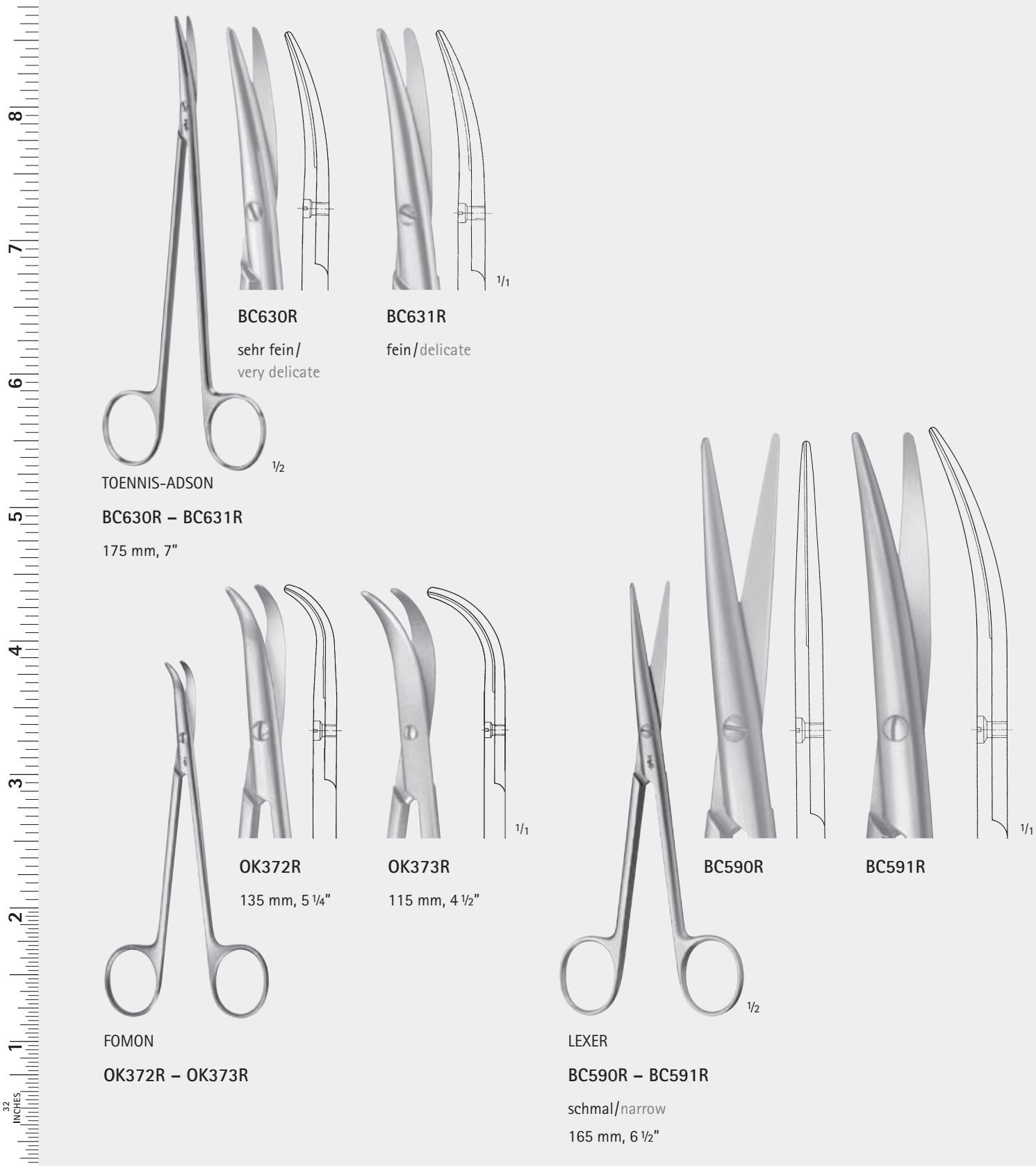
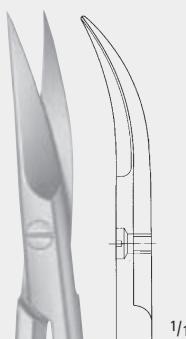
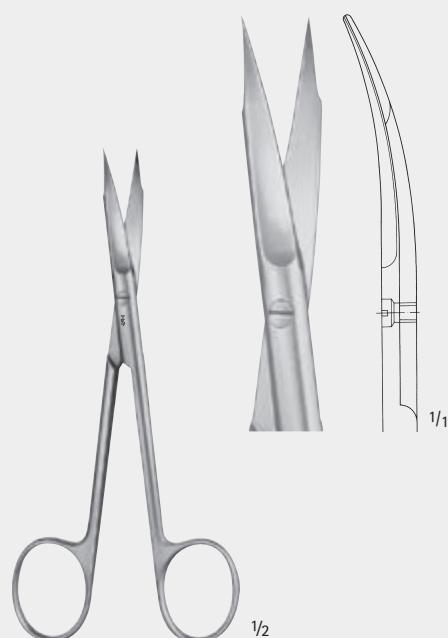
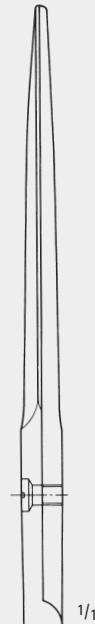
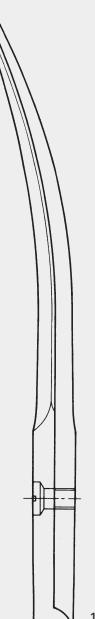
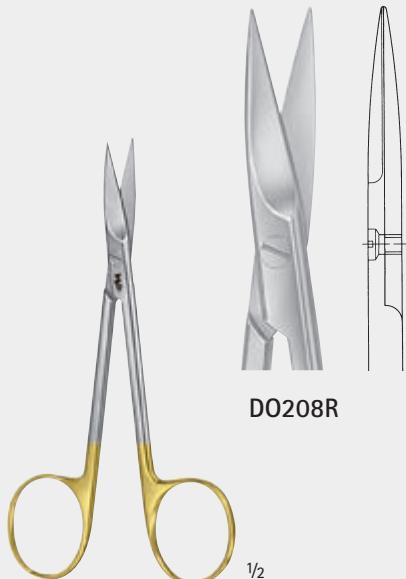
- 3 Jahre Garantie
- Vergoldete Ringe

DUROTIP®-Scissors
with inserted carbide cutting edges

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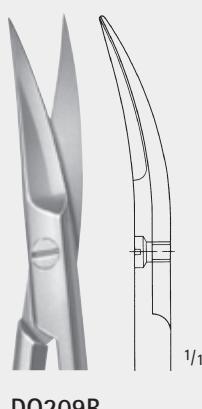
DUROTIP®-Scissors
with inserted carbide cutting edges
■ 3 Years guarantee
■ Gold-plated rings

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~W-Cut directly after the catalogue number indicates that the resp. item has one serrated cutting edge.
(When ordering please indicate ~W-cut together with the catalogue number e. g.: BC211W)

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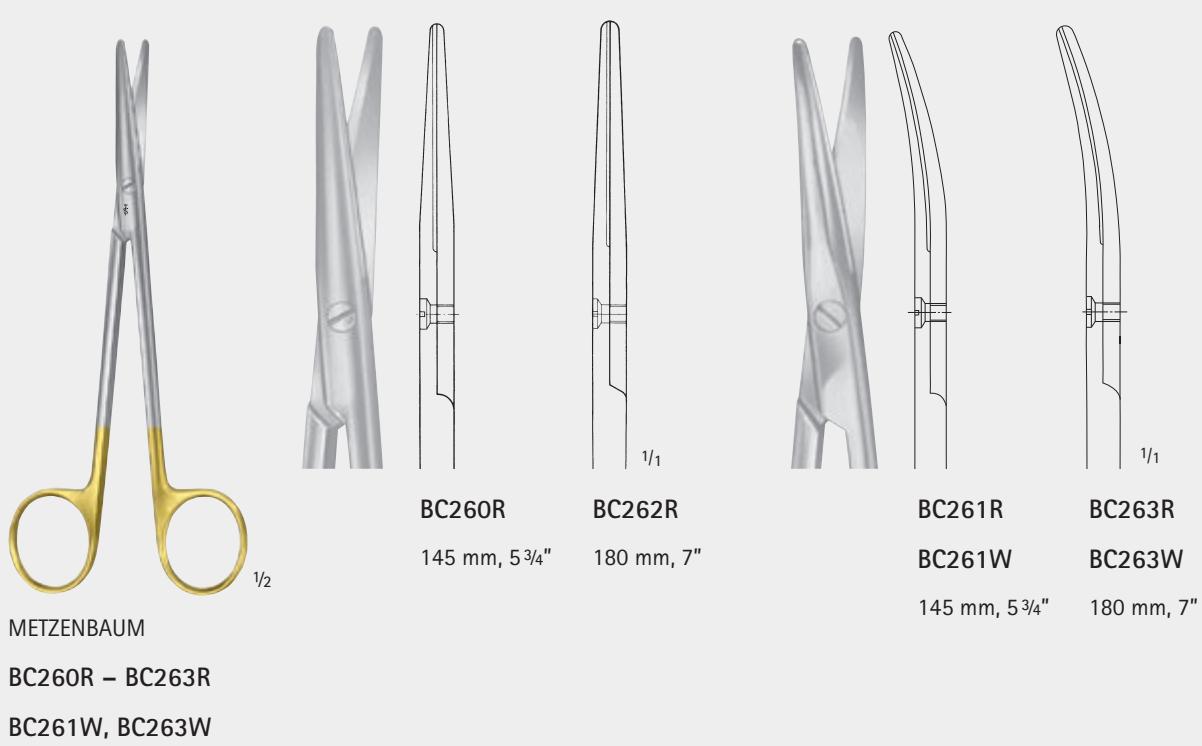
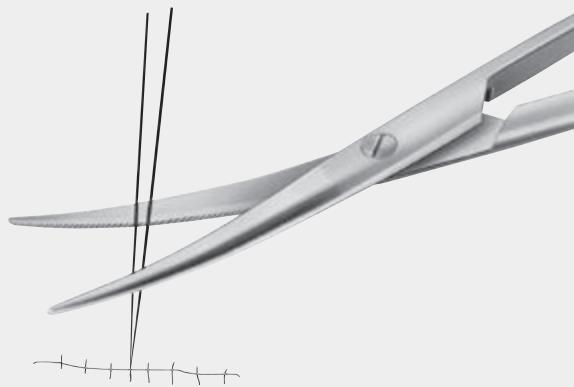
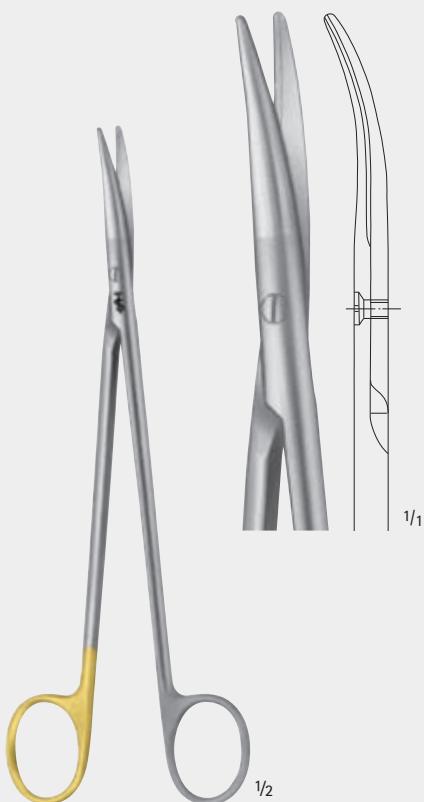
The special qualities of the Noir® dissecting scissors allow dissection that is finer than ever before. Noir®. Gentle dissection with minimum use of force.

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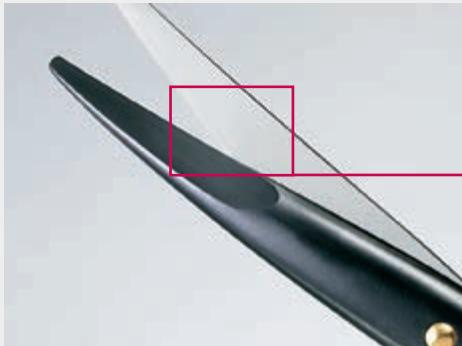
Microtome edge - the extremely sharp cutting edge of the scissors blade facilitates gentle cutting of tissue with minimal use of force.

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The combination of the renowned AESCULAP® tungsten carbide technology with the AESCULAP® Noir® coating results in an extremely high surface hardness and therefore high wear resistance in daily hospital use.

The extremely hard (2000HV) and low reflection Noir® coating leads to high wear resistance of the instruments and at the same time provides high resistance against corrosion.

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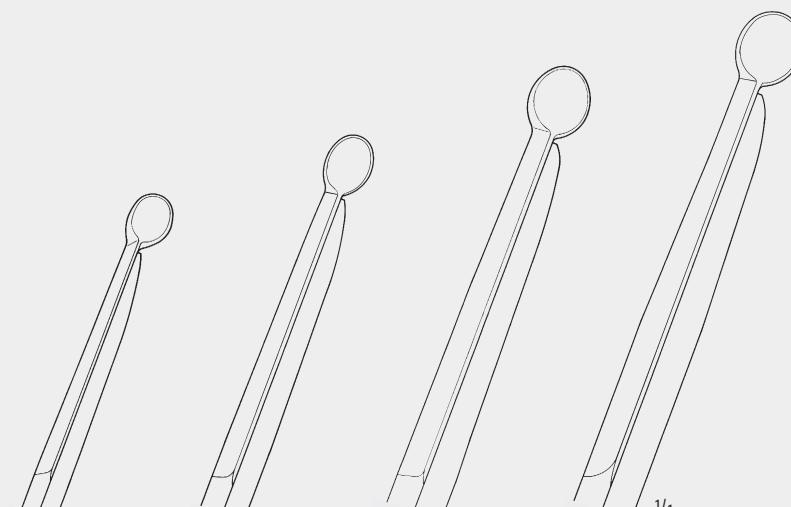
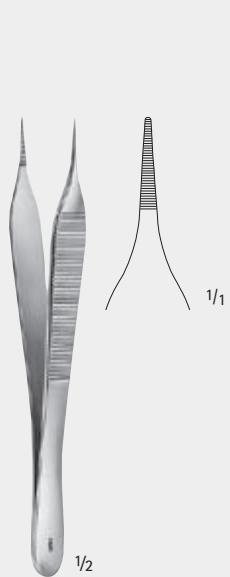
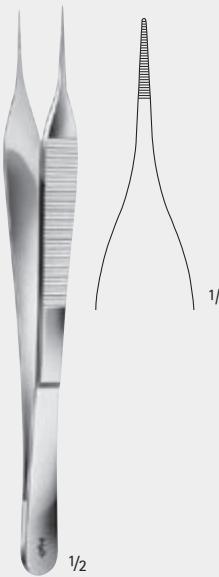
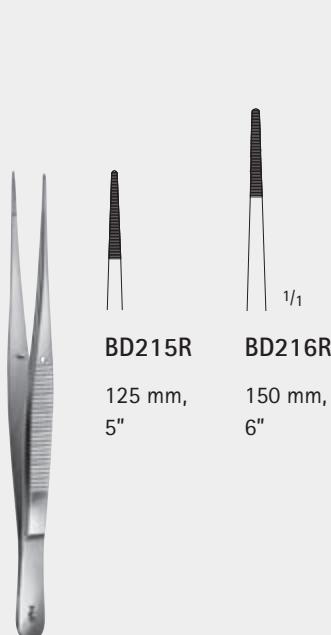
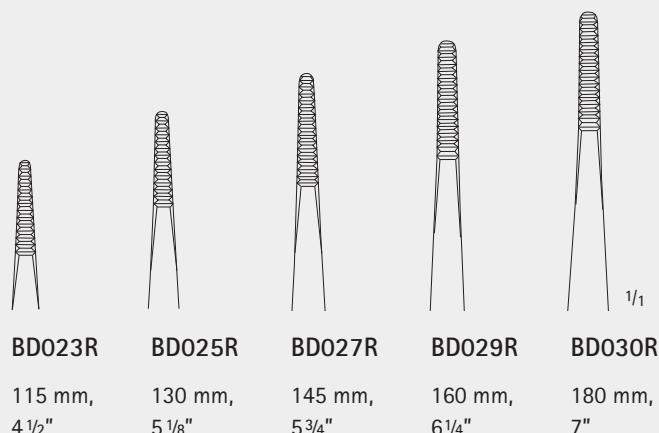
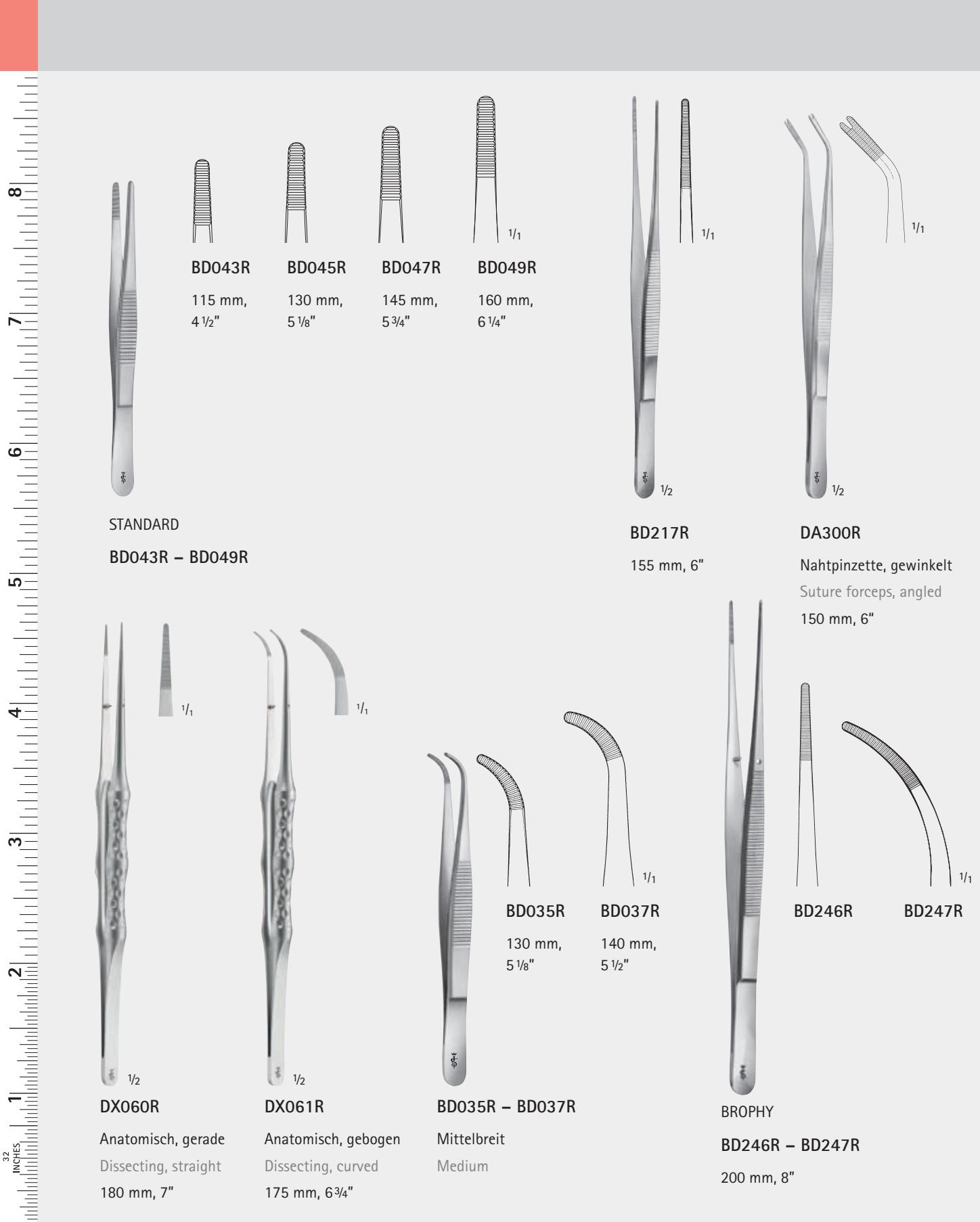
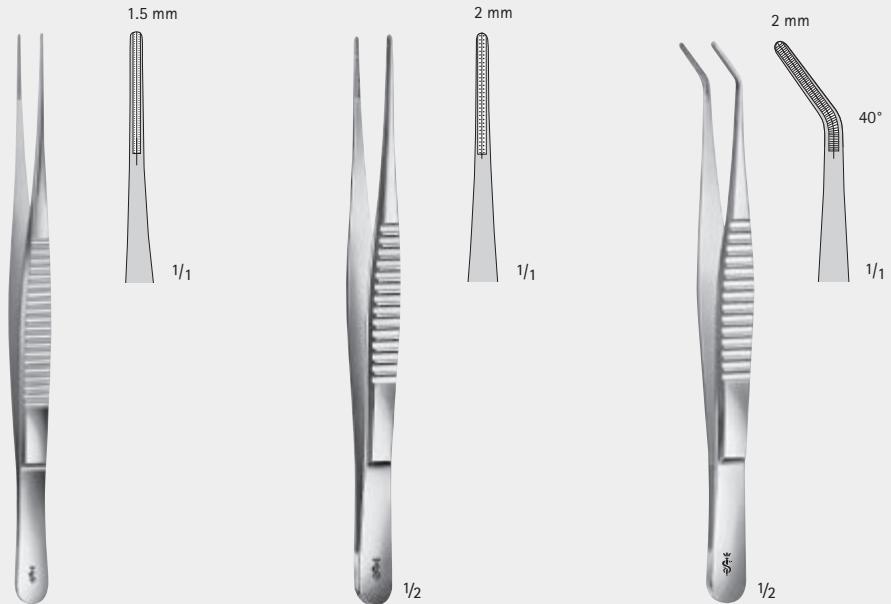
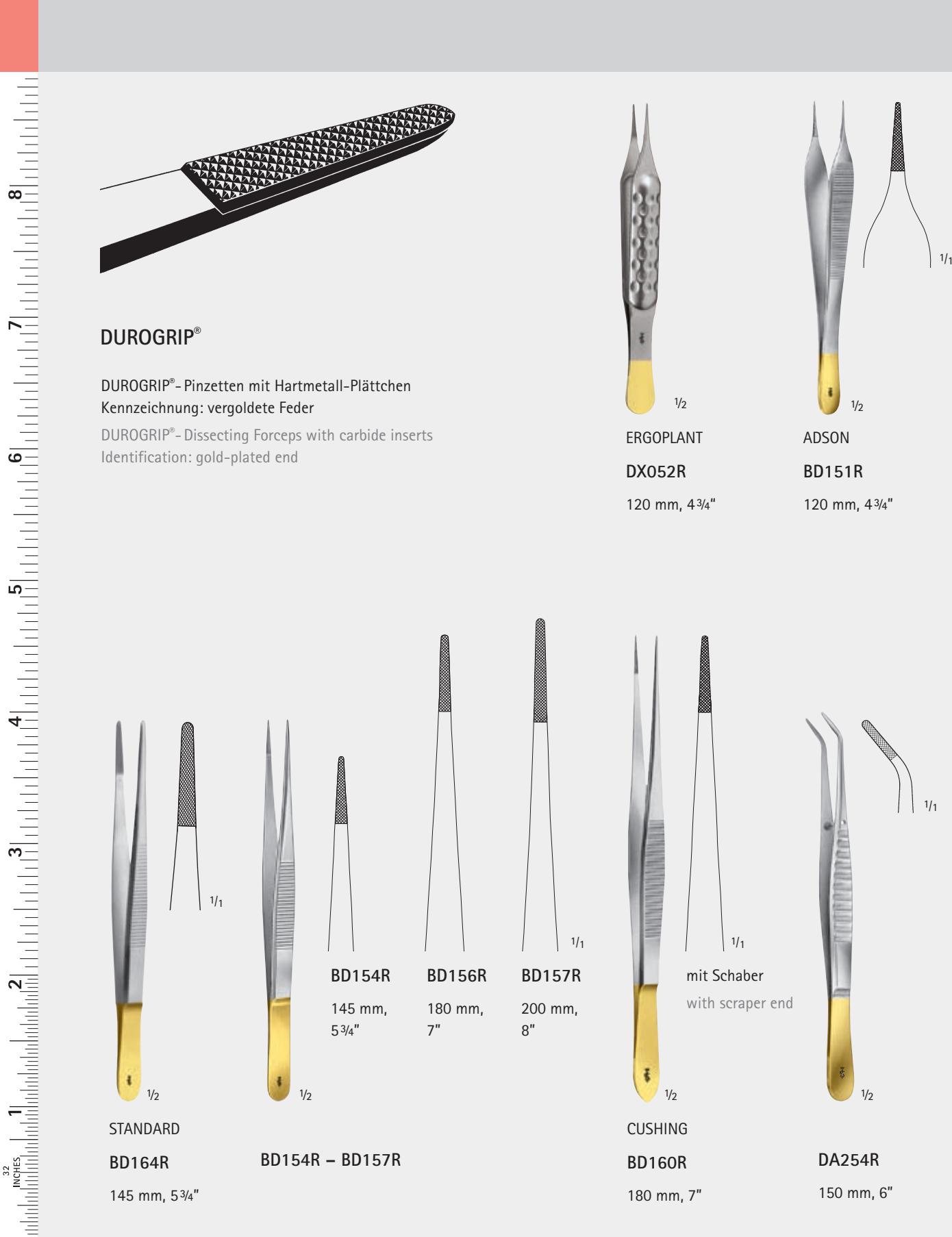
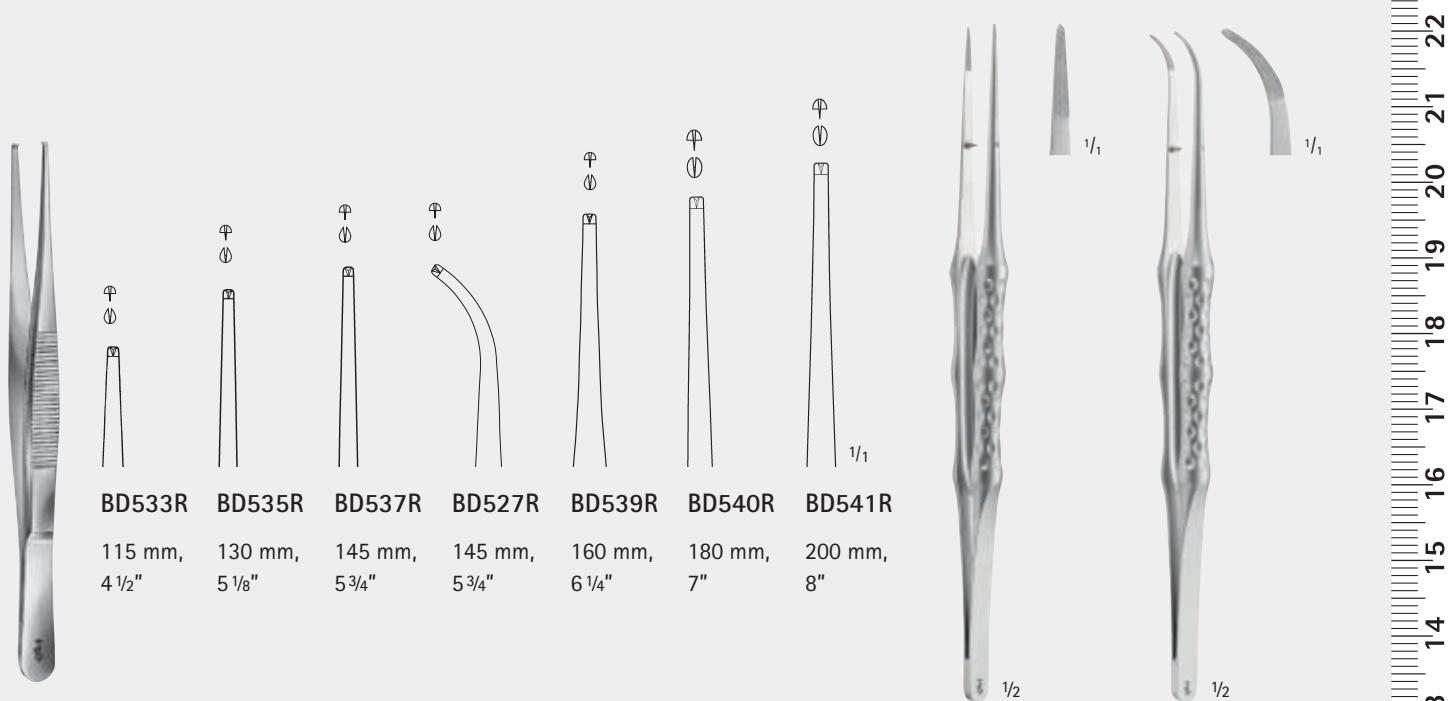
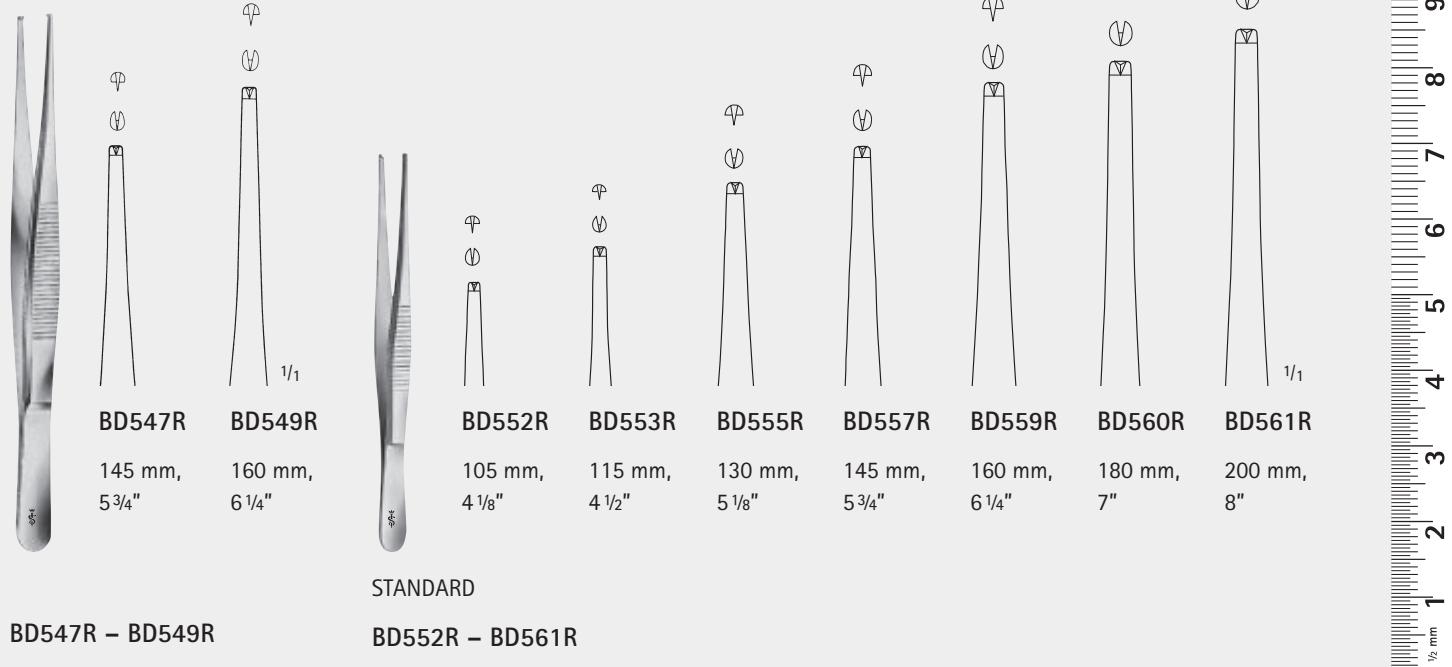
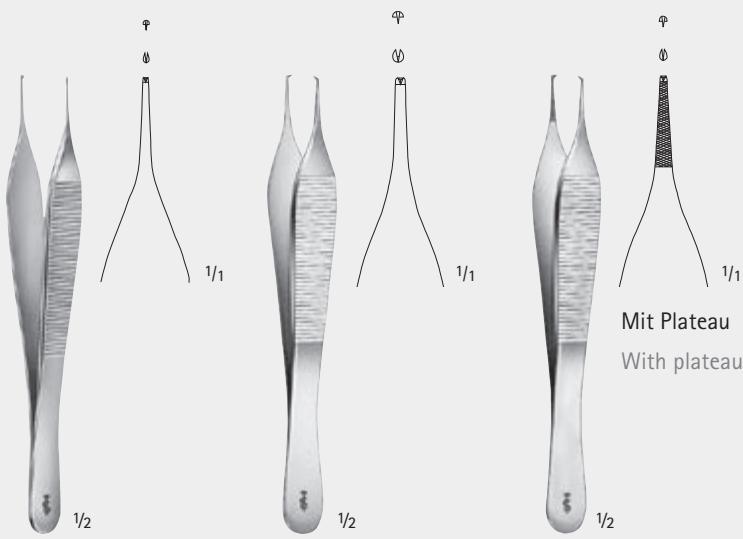
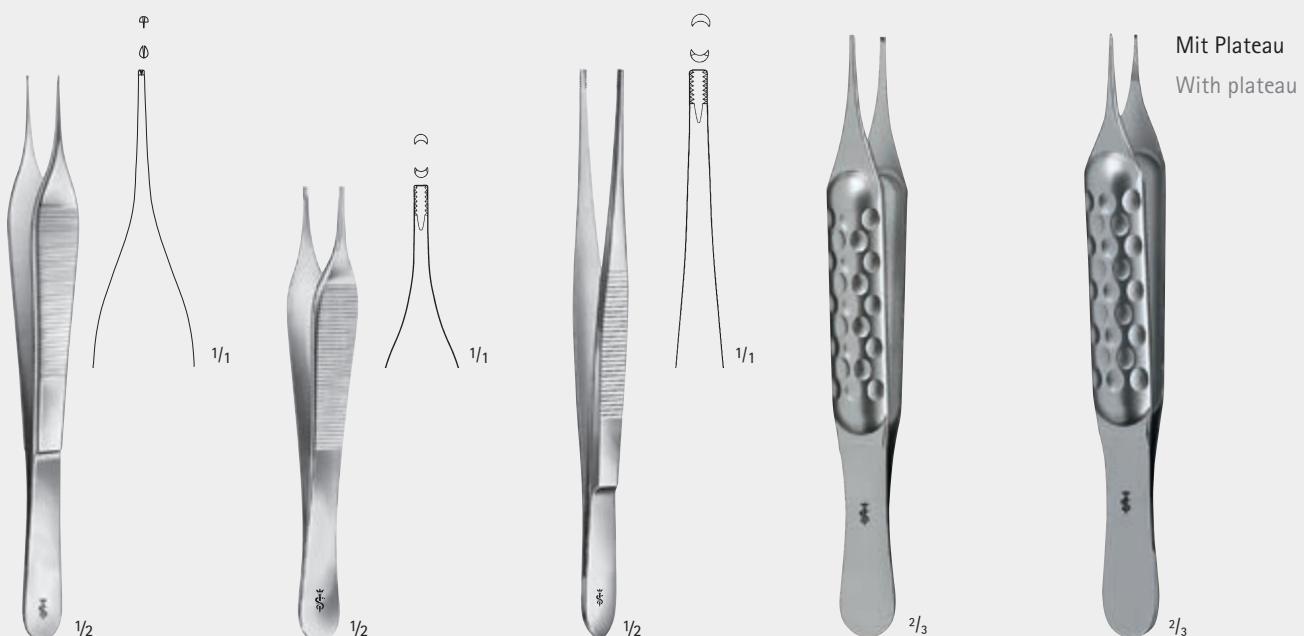
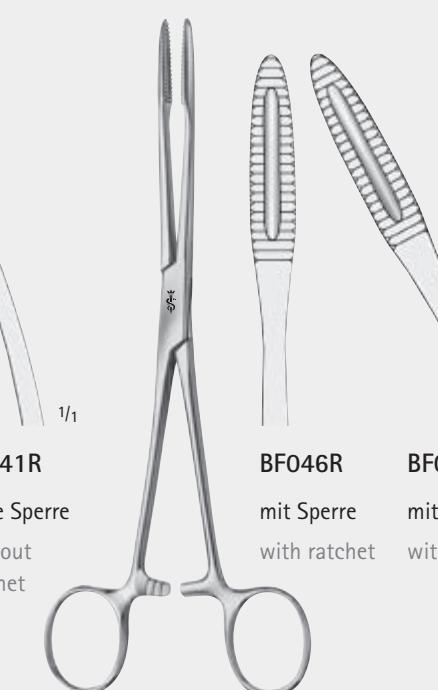
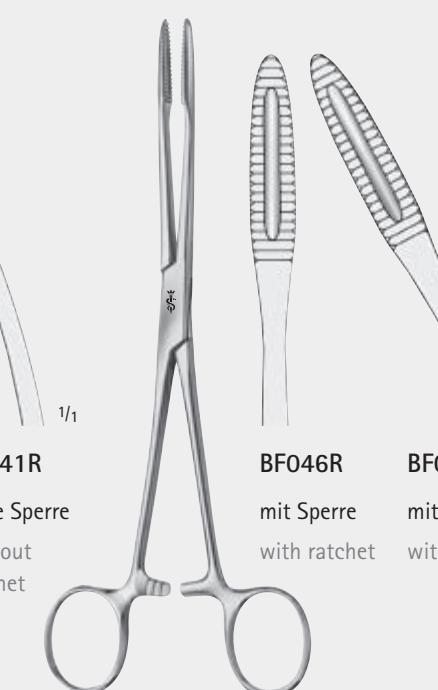
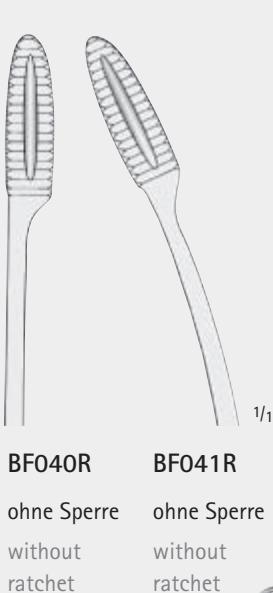
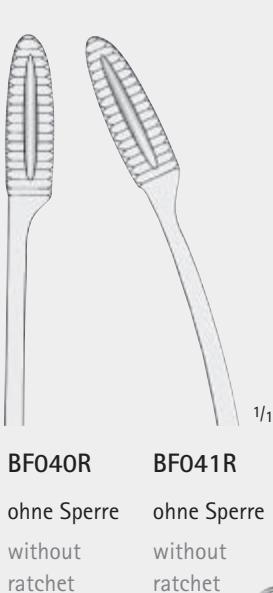
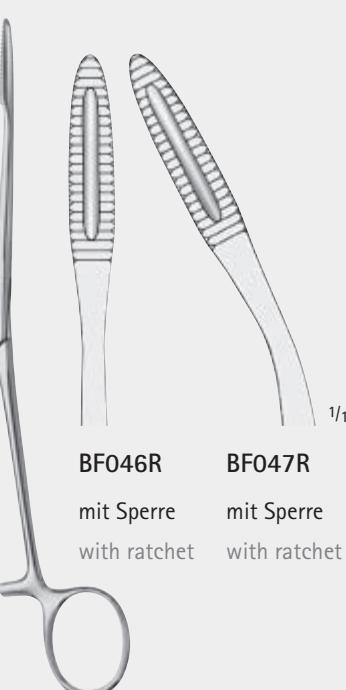
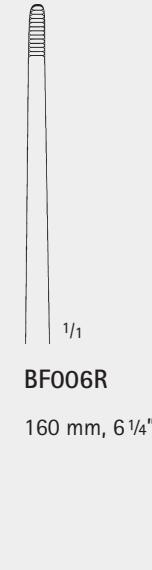
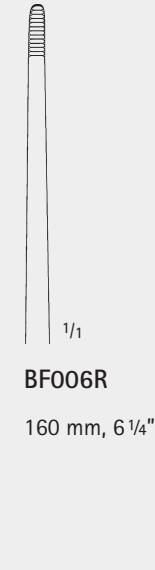
DENTAL SURGERY

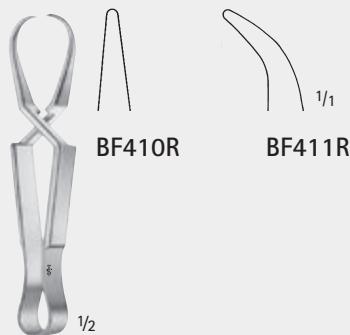
KORN- UND TUPFERZANGEN, TUCHKLEMMEN
SPONGE AND DRESSING FORCEPS, TOWEL CLAMPS

JONES
BF407R
Tuchklemme
Towel Clamp
85 mm, 3 3/8"

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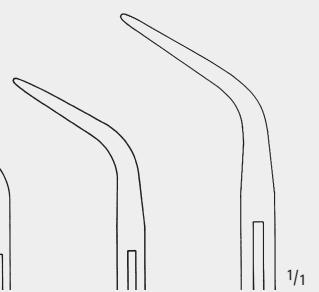
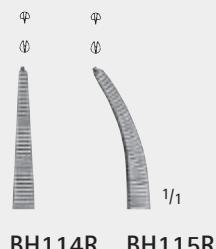
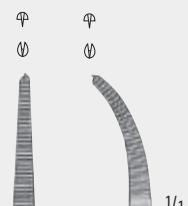
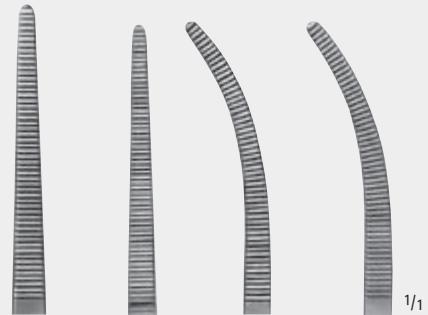
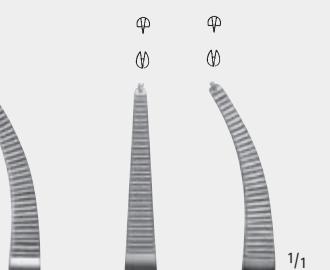
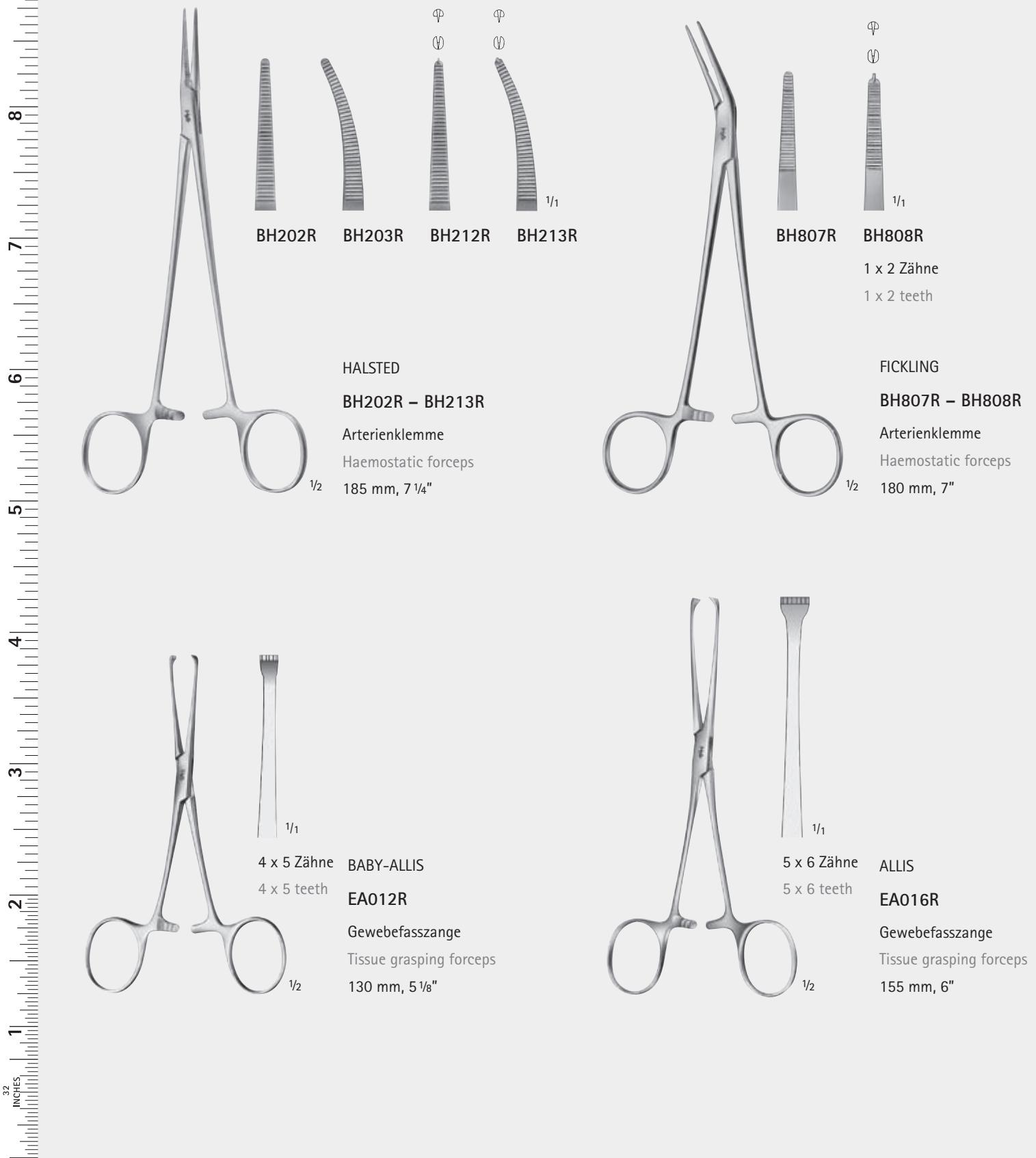
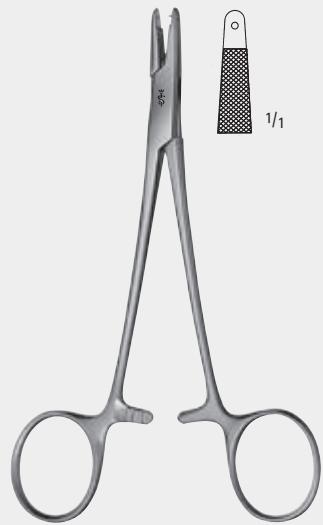
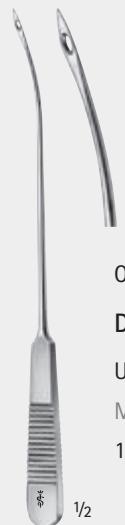
32 INCHES

PRICHARD
DX200R
Raspatorium
Raspatory
200 mm, 8"

PRICHARD
DB020R
Raspatorium
grazil
Raspatory
delicate
195 mm, 7 3/4"

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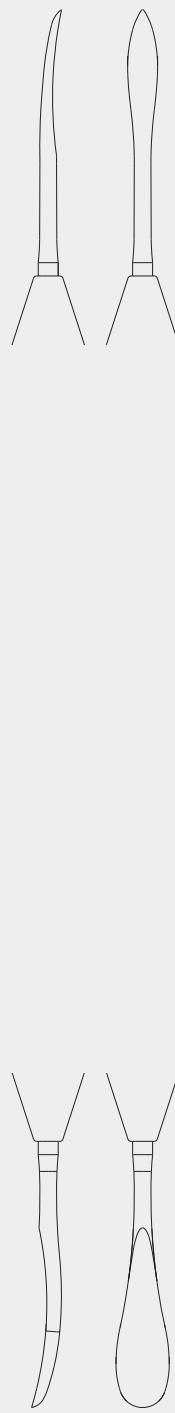
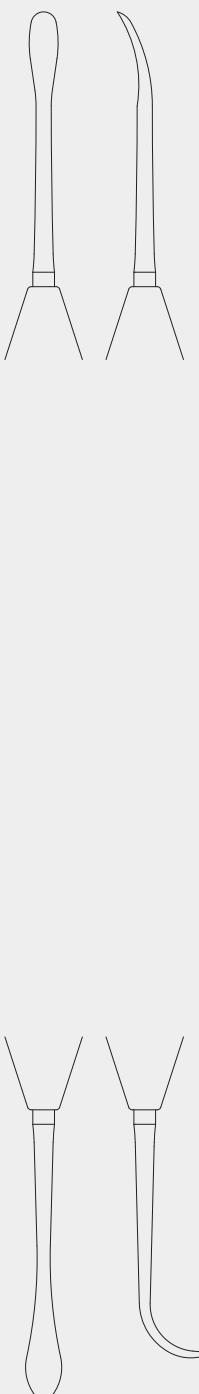
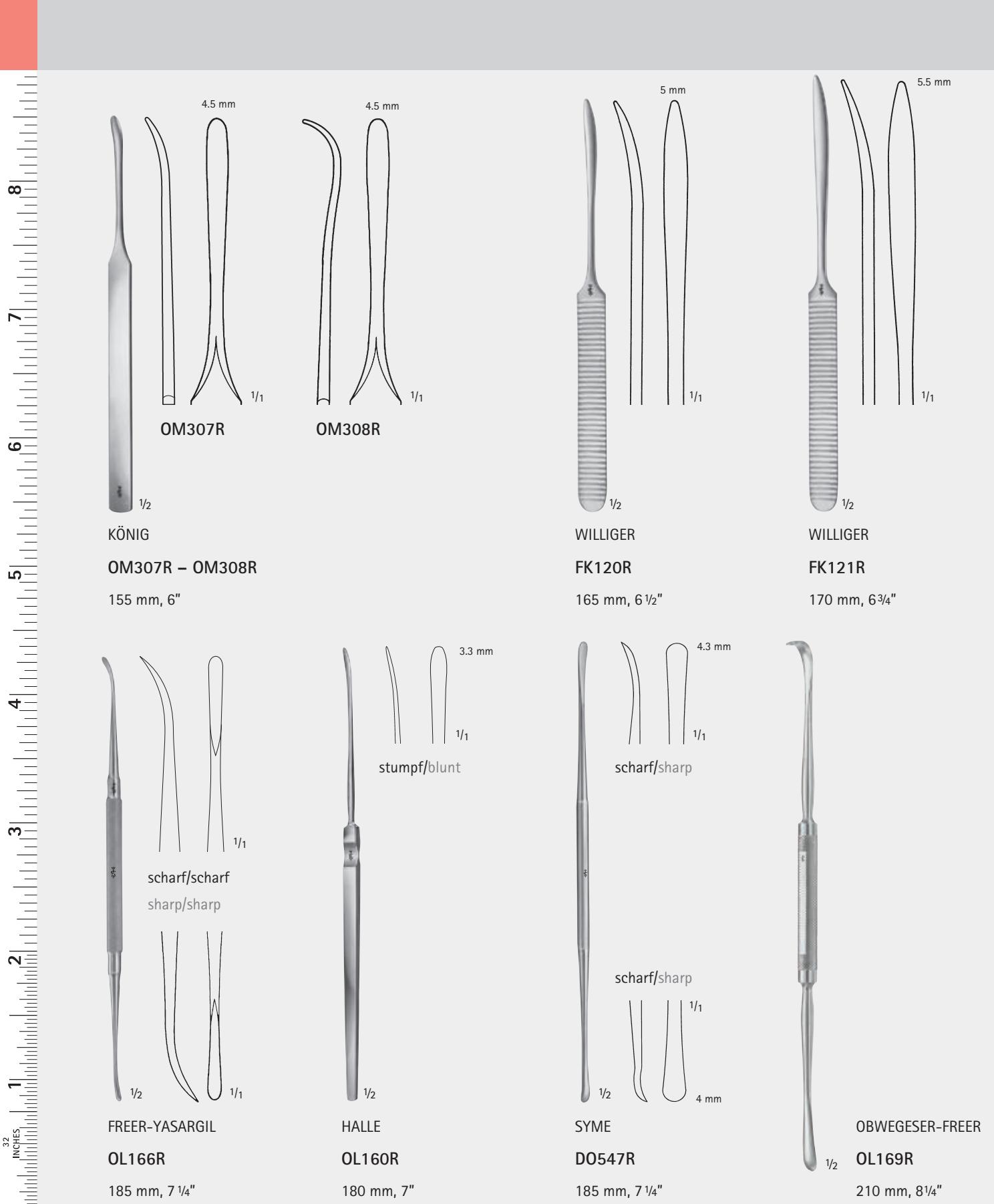
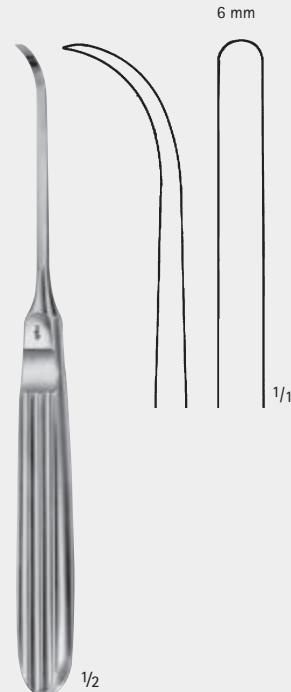
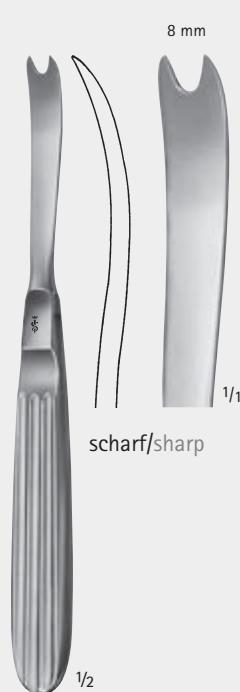
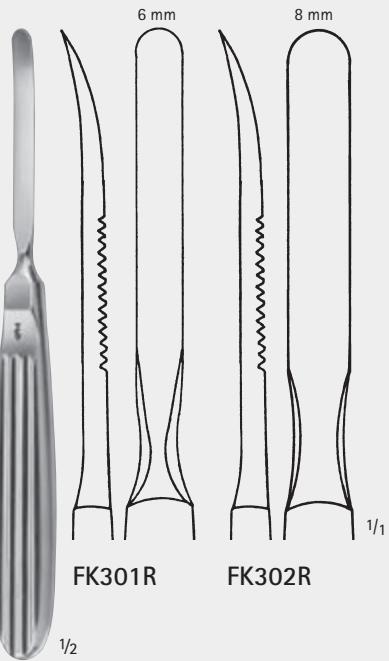
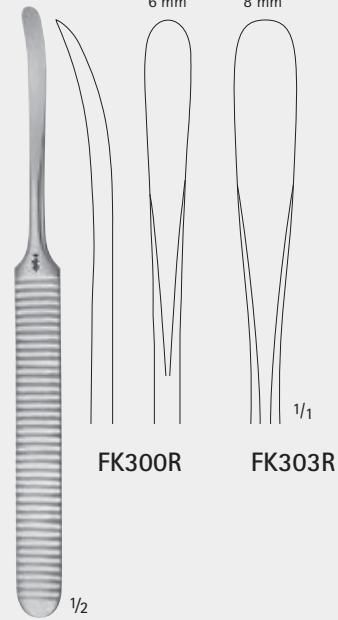
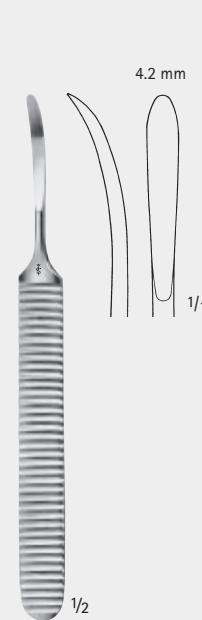
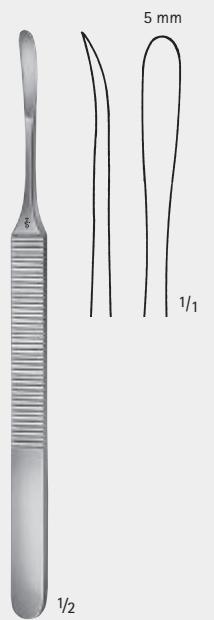
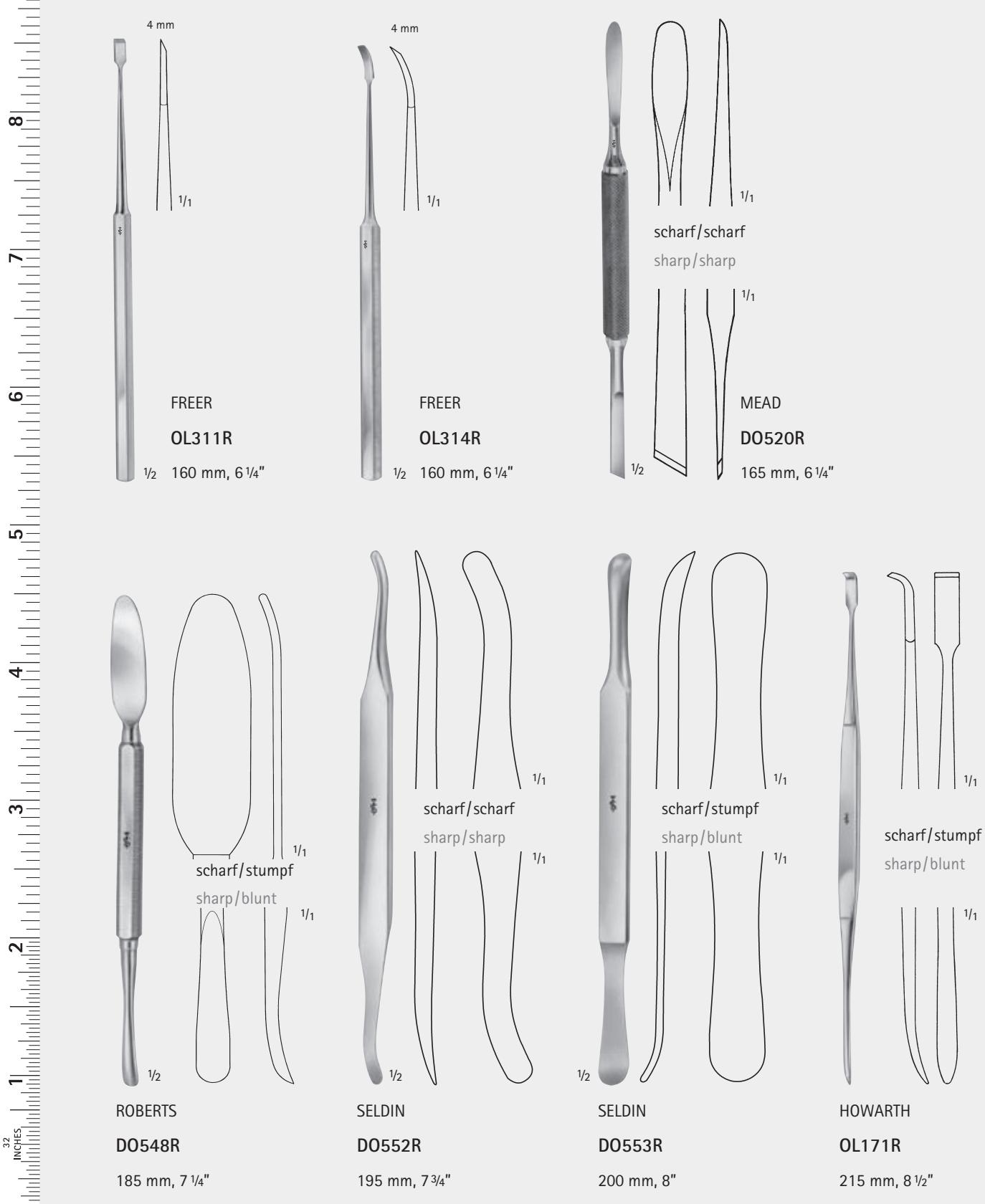
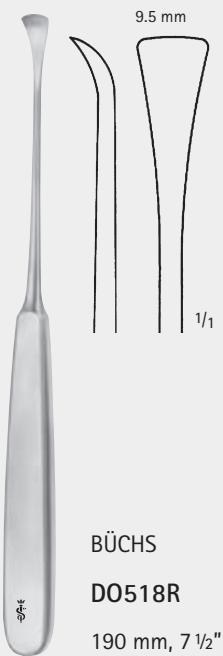
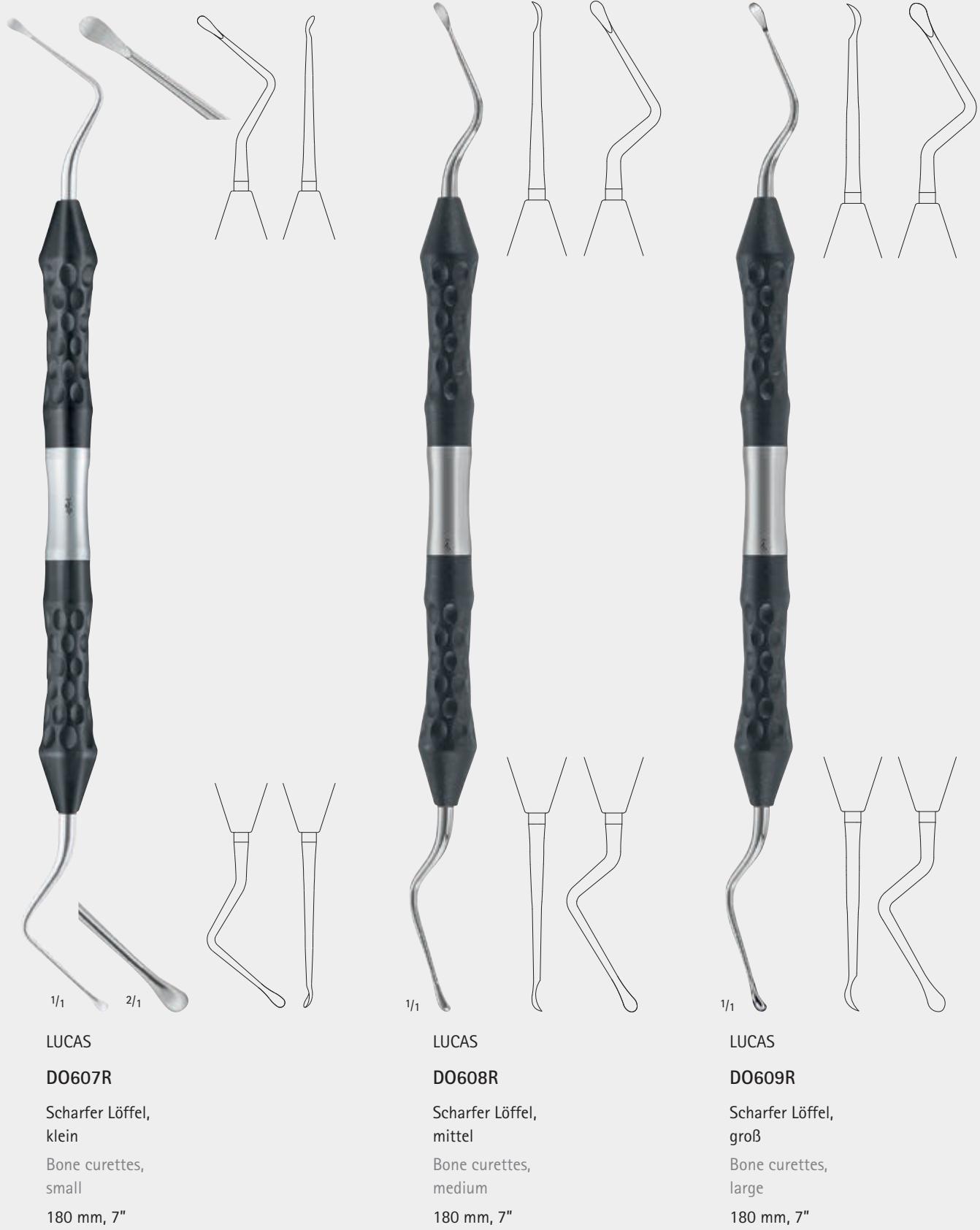
Range of application:

- For loosening parts of the mucosa
- For cyst resection
- For parodontal surgery

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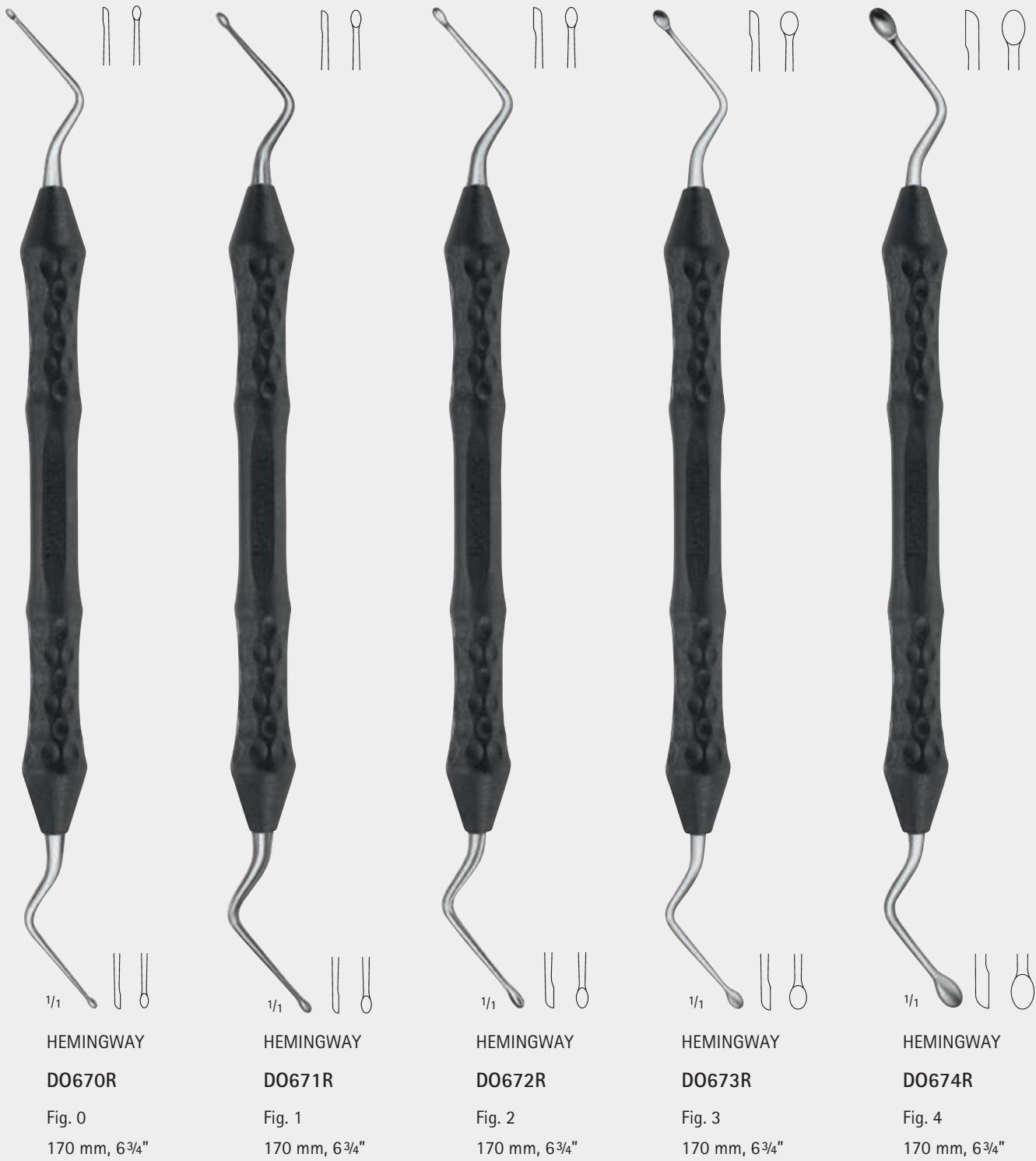
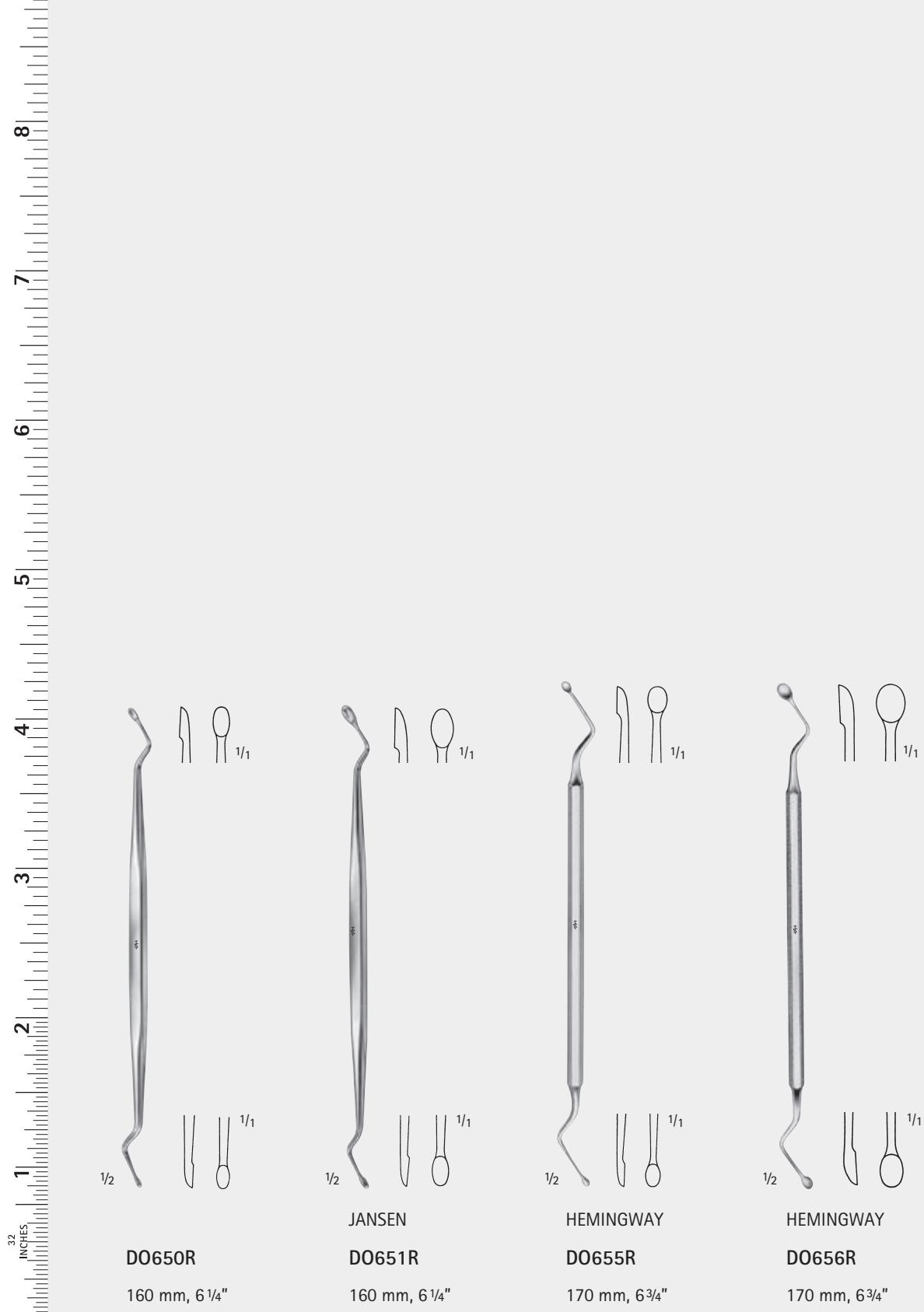
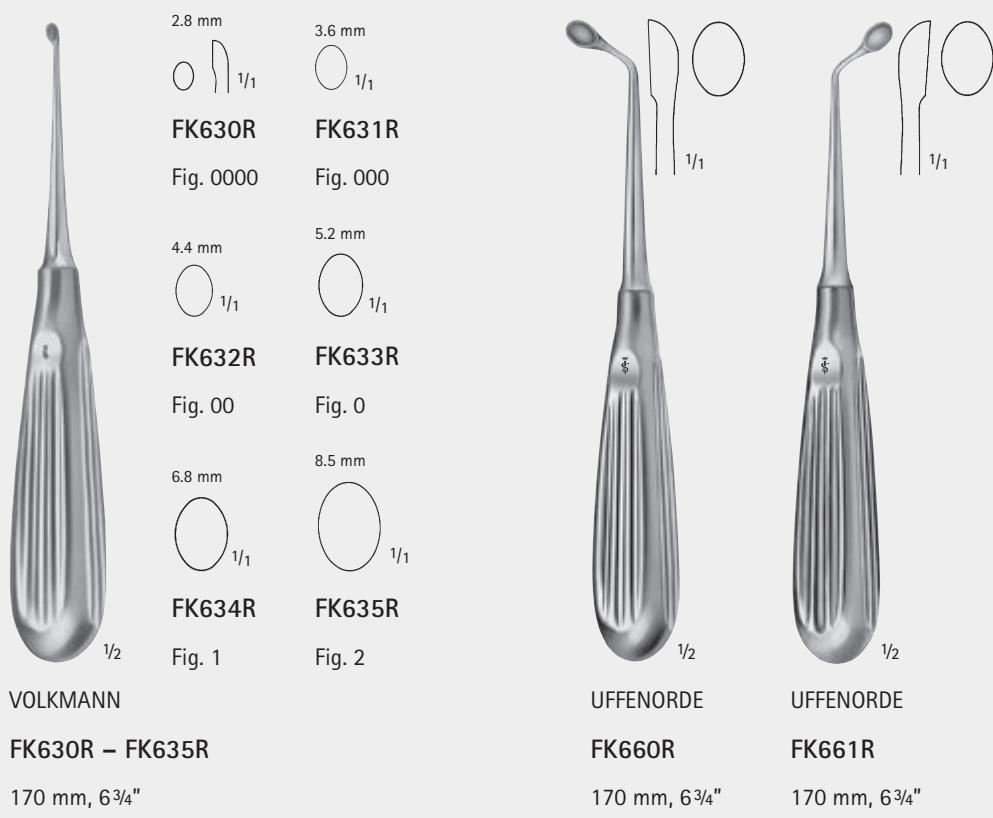
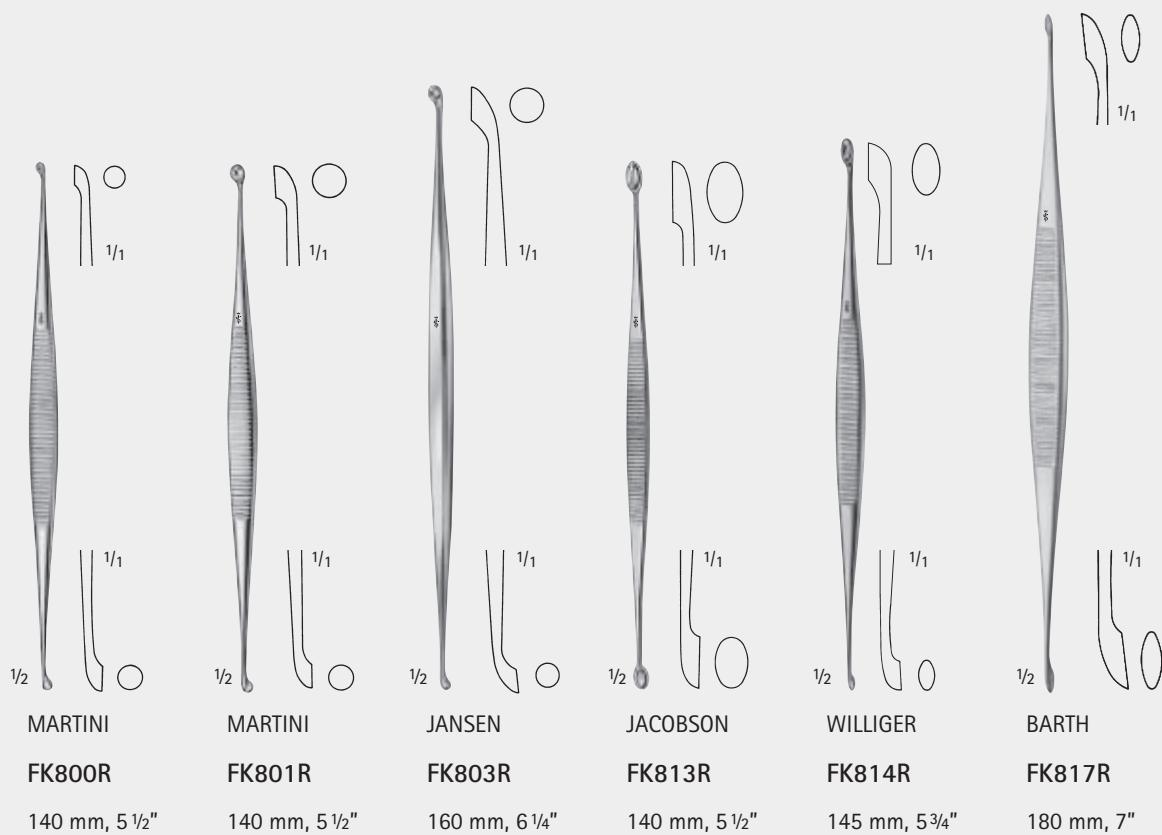
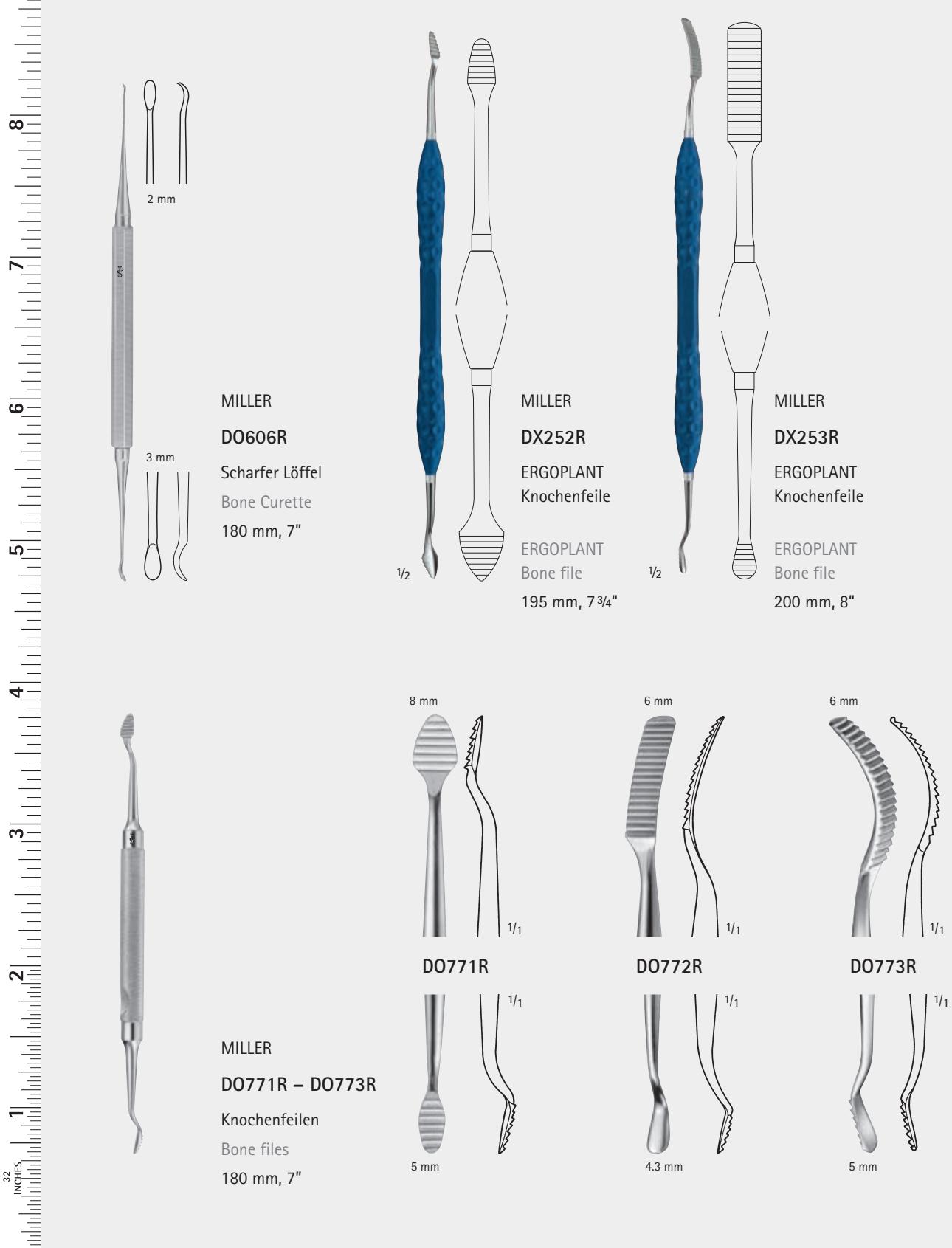
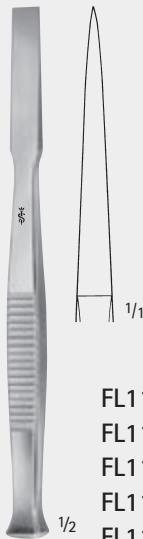
LAMBOTTE

FL101R 4 mm
FL102R 6 mm
FL104R 8 mm
FL106R 10 mm
FL108R 12 mm
FL109R 15 mm

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
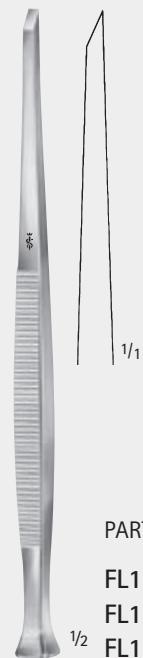
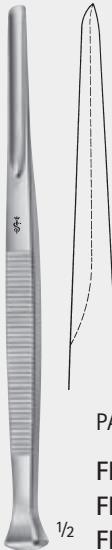
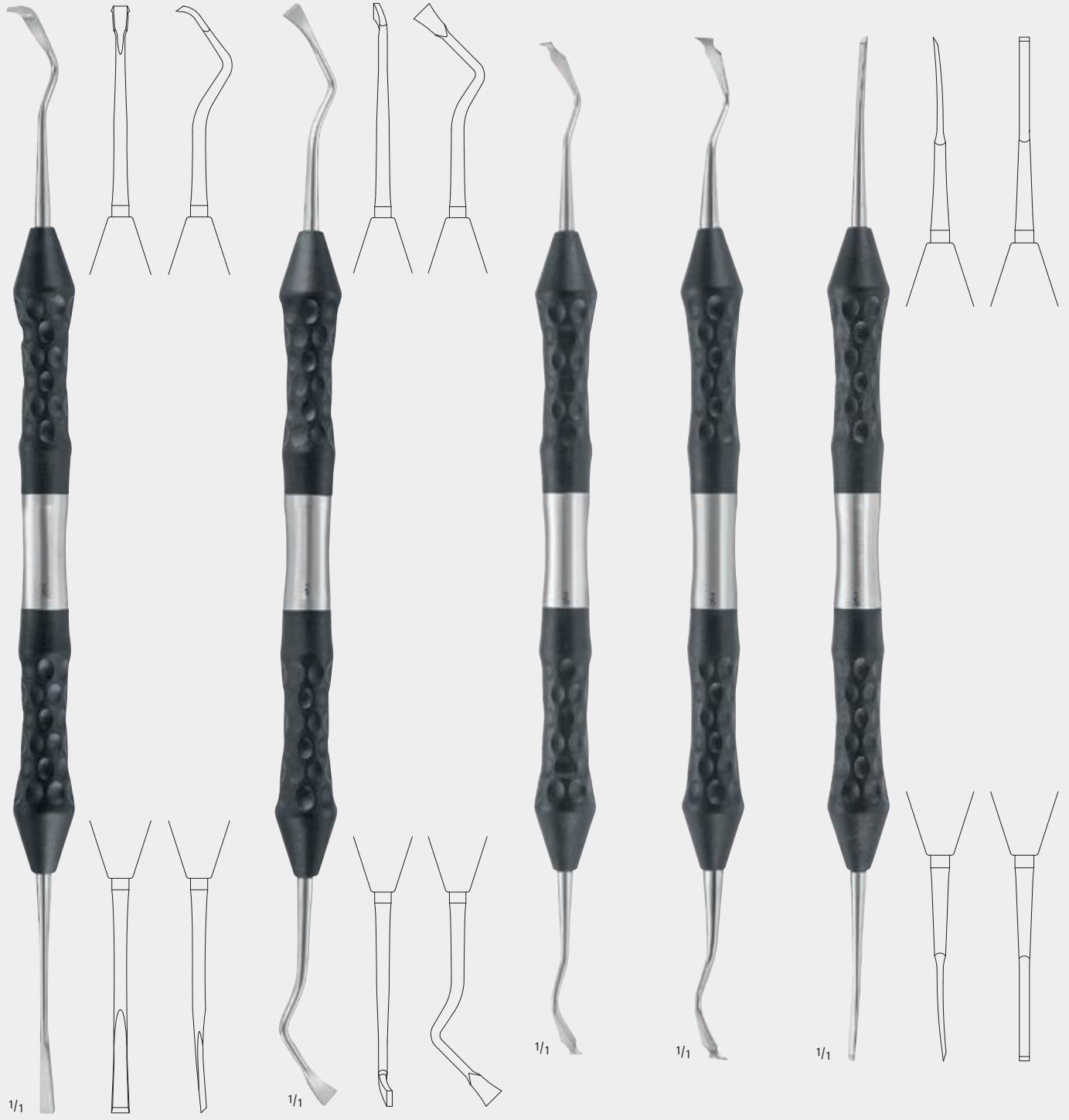
LUCAS
DO715R 3 mm
Hohlmeißel
Gouges
160 mm, 6 1/4"

1/1

MEISSEL, OSTEOTOME
CHISELS, OSTEOTOMES

1/1
KIRKLAND

DB030R

Fig. S13K/TG

Mini Meiβel

Mini chisel

190 mm, 7½"

1/1
OCHSENBEIN

DB034R

Fig. 4

Mini Meiβel

Mini chisel

190 mm, 7½"

1/1
RHODES

DB835R

Fig. M CR 36/37

Mini Meiβel

Mini chisel

175 mm, 7"

1/1
RHODES

DB836R

Fig. CR 36/37

Mini Meiβel

Mini chisel

175 mm, 7"

1/1
KRAMER-NEVINS

DB838R

Fig. M CKN 1/2

Mini Meiβel

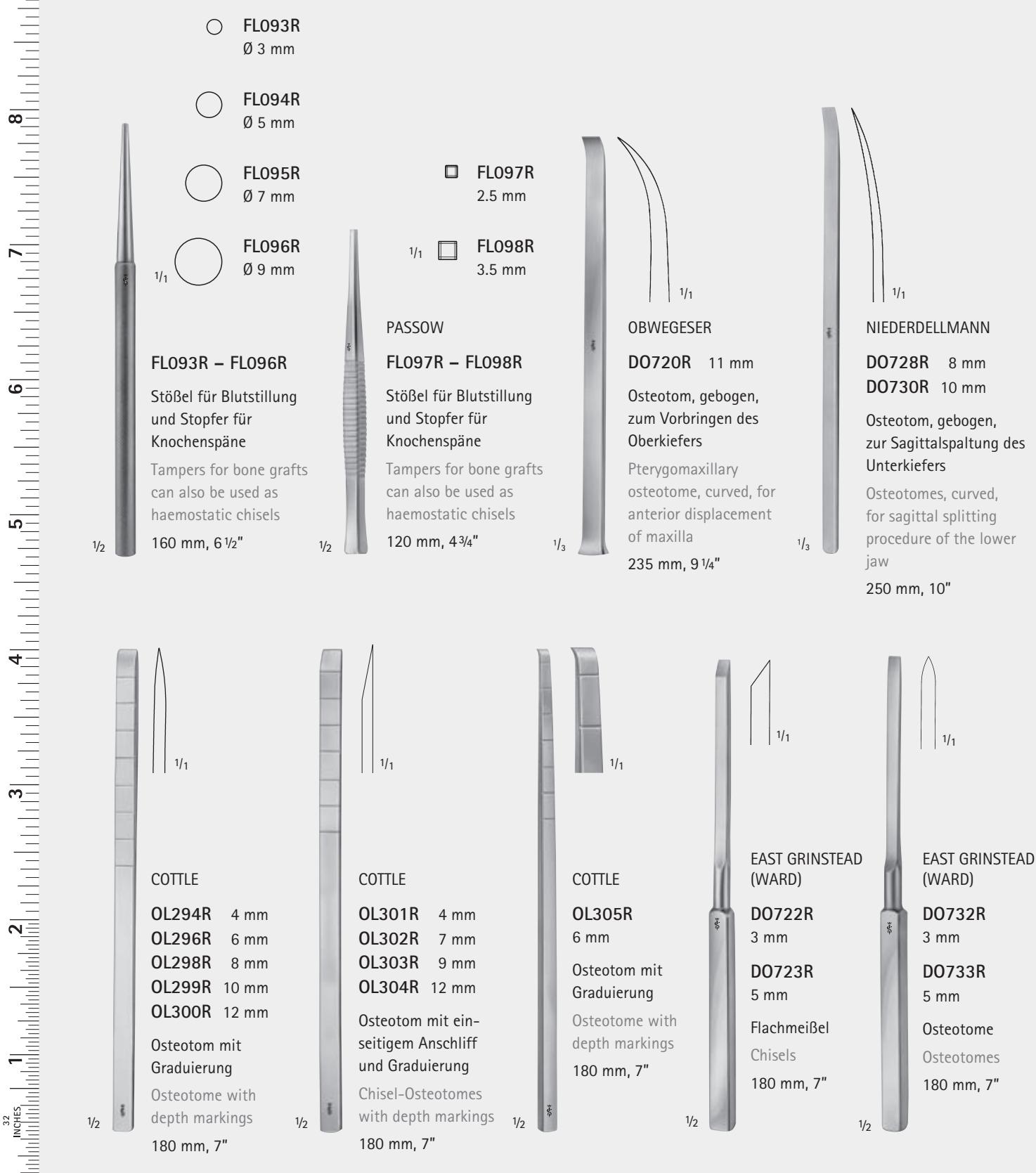
Mini chisel

175 mm, 7"

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DENTAL SURGERY

STÖSSEL, MEISSEL, OSTEOTOME
TAMPERS, CHISELS, OSTEOTOMES

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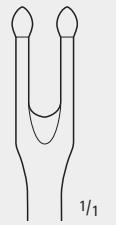
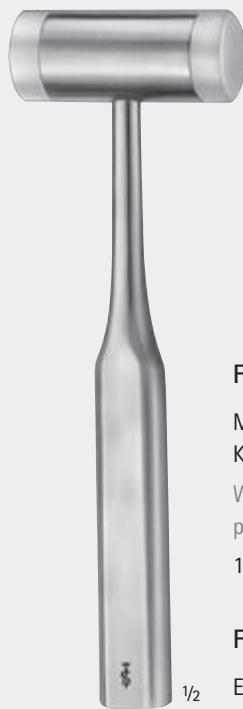
Weights refer to the heads of the mallets

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ERGOPLANT Mallet,
with interchangeable plastic disks

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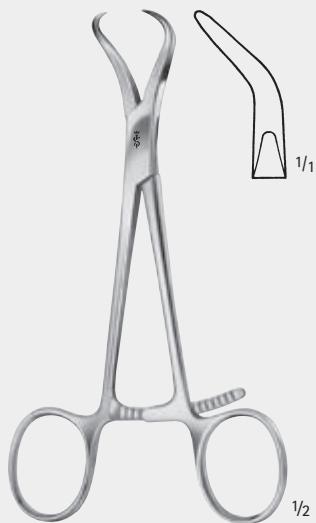
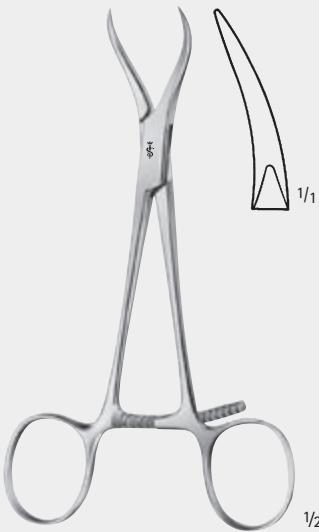
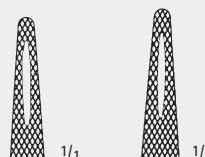
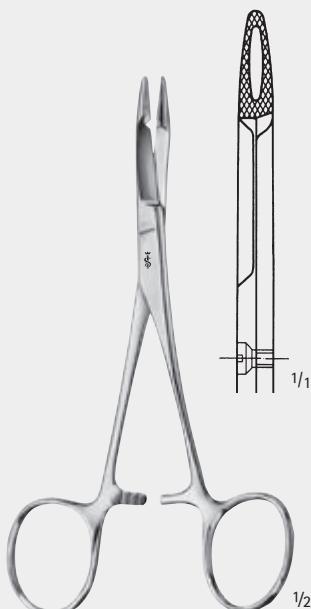
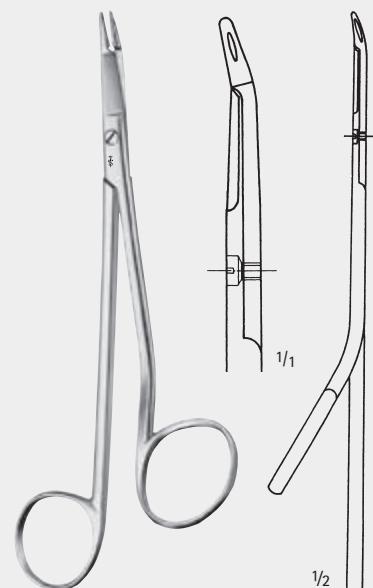
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
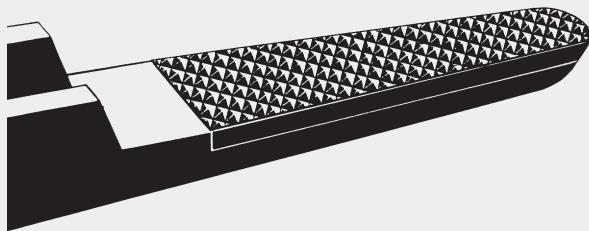
AESCLAP® needle holders have jaws with carbide inserts and according to the use required the resp. structure can be chosen. A smooth jaw surface as well as 3 different serrations enable a „grip“ of the respective needle.

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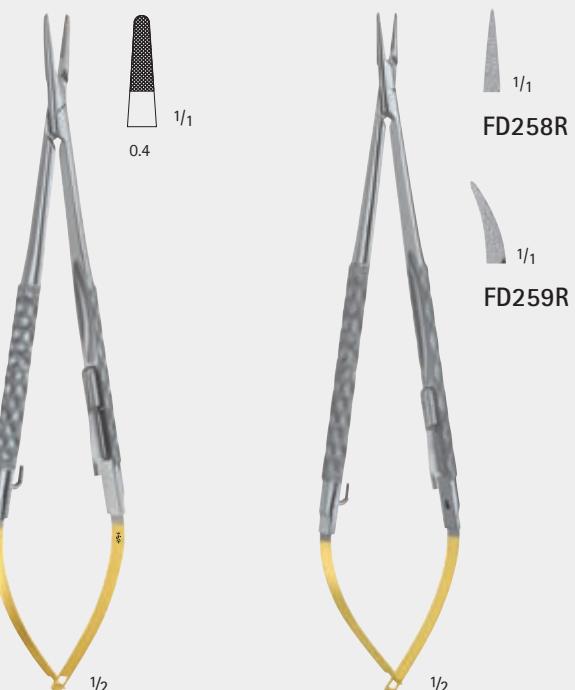
We recommend carbide inserts with smooth jaw surface for suture material with 0.2 mm pitch of serration for suture material
0.4 mm
0.5 mm

DUROGRIP® Nadelhalter mit Hartmetall-Plättchen
Kennzeichnung: Vergoldete Ringe, Branchenenden und Sperren
DUROGRIP® Needle holders with carbide inserts
Identification: gold-plated rings, handle ends and ratchets

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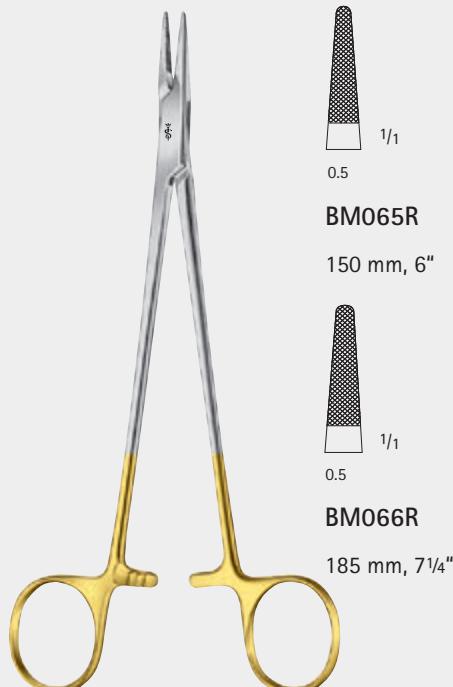
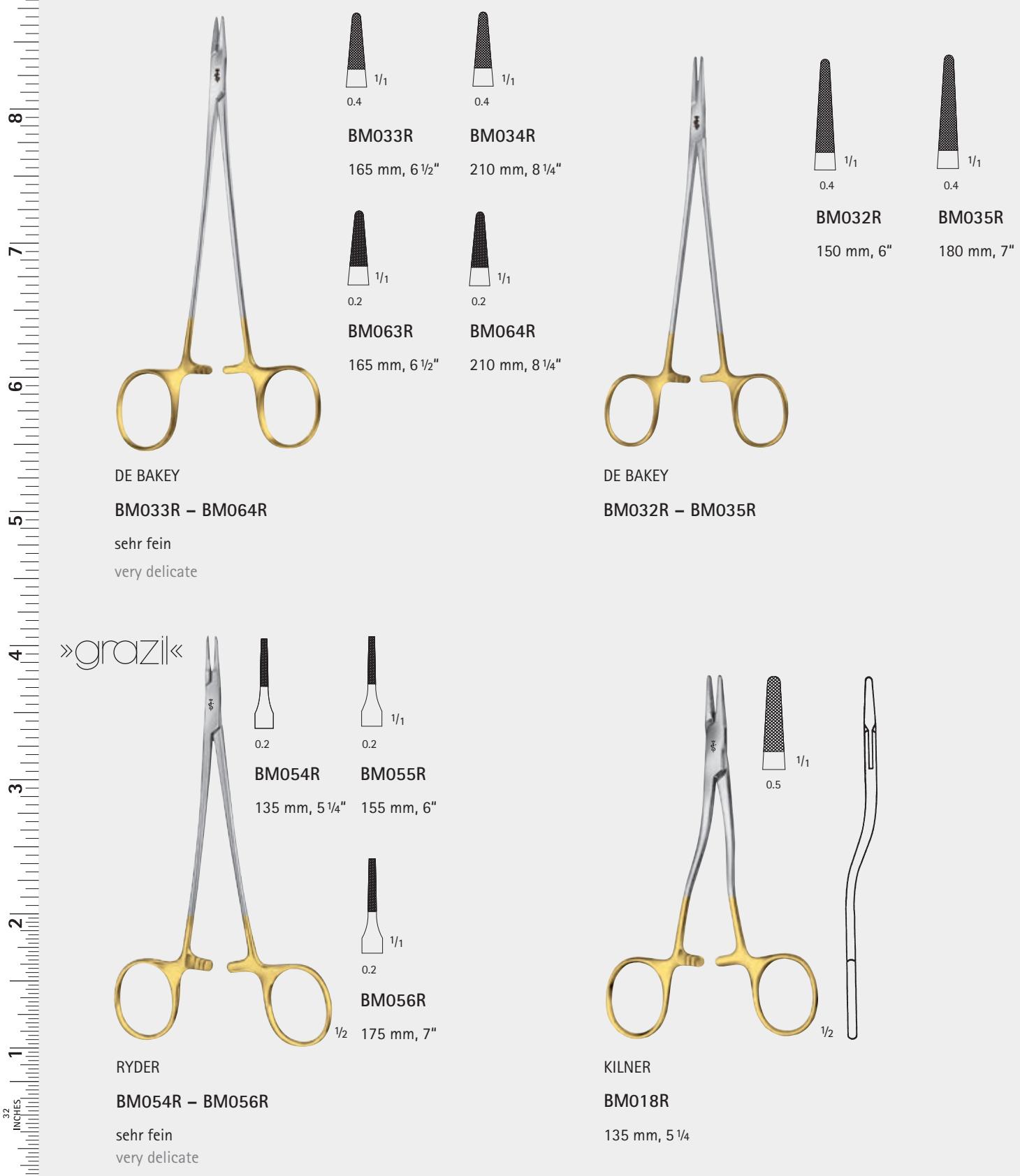
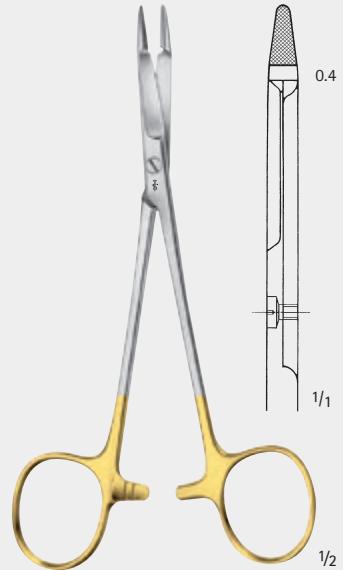
Needle holder with
scissors, one blade fine
serrated

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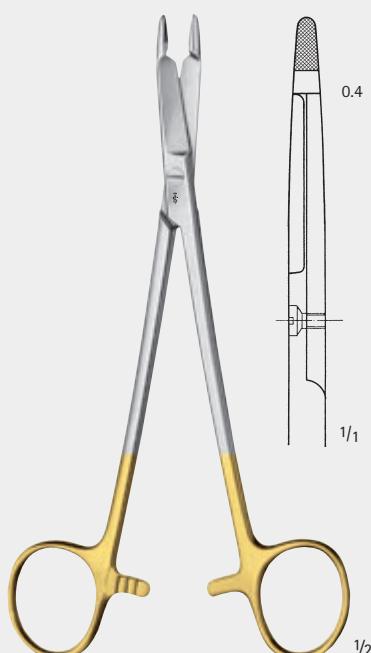
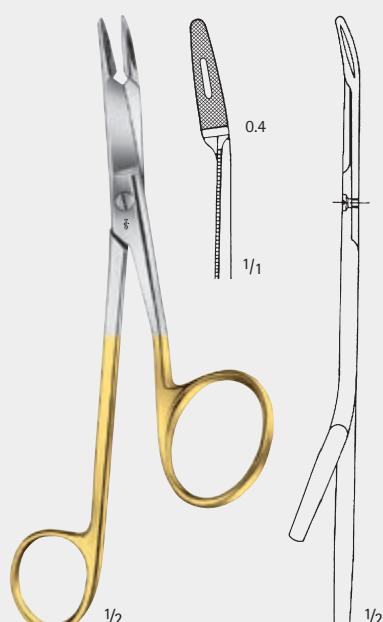
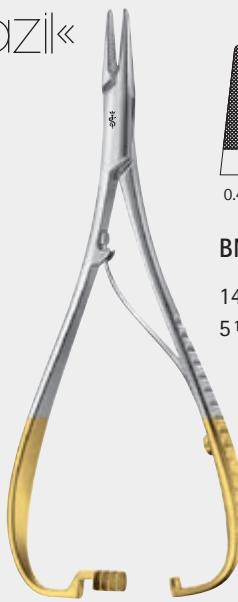
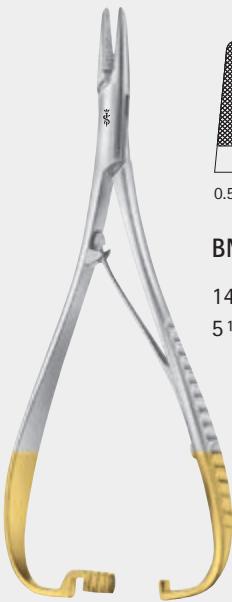
DUROGRIP® Needle
holder with long neck

205 mm, 8"

1/1

0.4

1/2

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DENTAL SURGERY

8
7
6
5
4
3
2
1
32 INCHES

BL907R 70 x 53 x 9 mm

Nadeldose

Needle case

BL941R 150 x 90 x 10 mm

Nadeldose

Needle case

BL910R 50 x 25 x 5 mm

BL911R 50 x 30 x 6 mm

BL915R 75 x 25 x 5 mm

Nadeldose

Needle case

BL930R 65 x 13 mm

BL932R 95 x 16 mm

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Needle case complete,
with interior box

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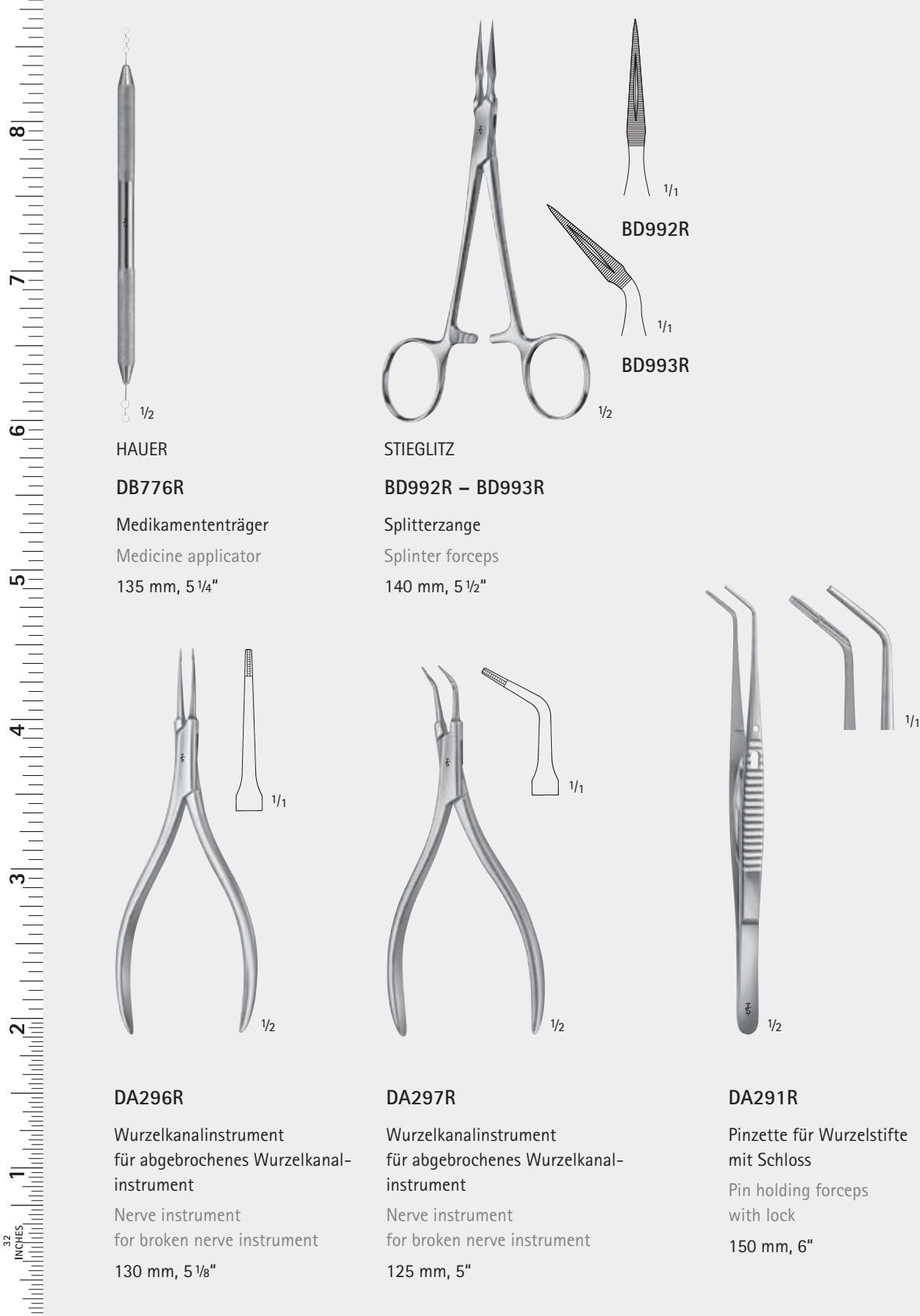
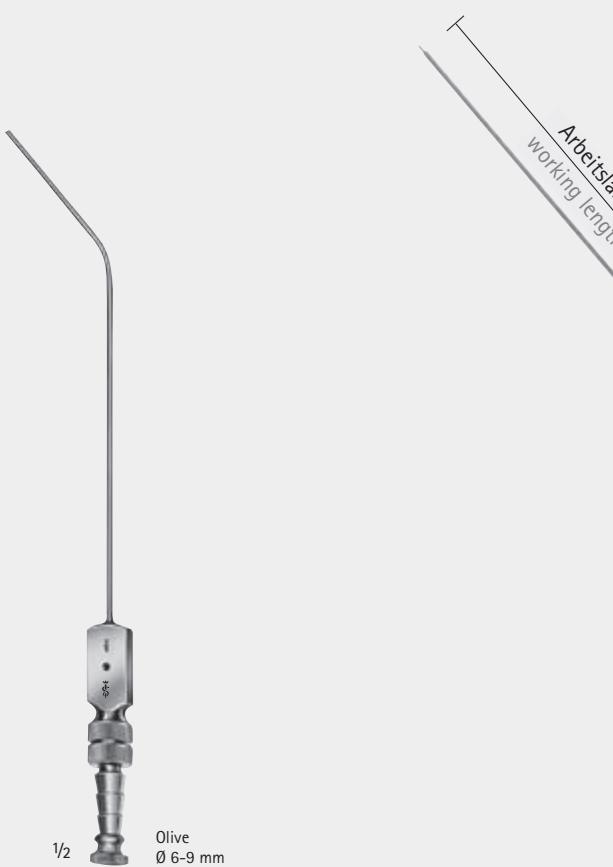
With finger cut-off and stylet,
thumb piece parallel to curve, with LUER hub
195 mm, 7 3/4"

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With finger cut-off and stylet, thumb piece 90° to curve, with
LUER hub

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AESCULAP® Cartridge Syringe „Aspira-plus II“ made of stainless instrument-steel (no chrome), with aspiration device

After having inserted the ampulla and the cannula, the syringe is immediately ready for injection.

By a slight twist of the piston rod to the right the double corkscrew tip will grasp the foam rubber of the ampulla.

This enables a controlled aspiration.

Advantages:

- Processing by using corrosion- and acid resistant instrument steel instead of chrome material
- Cartridge insert and ejection by folding back the bush

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Syringe with long needle attachment
(SL002R)

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Syringe with crutch handle, with long needle attachment
(SL002R)

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Needle attachment, long, suitable for all disposable needles with a thread diameter of 6 mm and long cartridge pick (e.g. Sterican® Z, B. Braun etc.)

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Syringe with short needle attachment (SL033R)

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Short needle attachment only, for American disposable needles with small thread (\varnothing 5.5 mm) and short cartridge pick

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Short needle attachment only, for disposable needles, with large thread (\varnothing 6 mm) and short cartridge pick.

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DENTAL SURGERY

AESCLAP® Single Use Surgical Instruments

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For further details see
brochure No. **C82002**

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For further details see
brochure No. **4008000**

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For further details see
brochure No. **022702**

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For upper incisors, canines, premolars
and roots

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For canines and premolars, universal tooth
forceps and tooth forceps for children

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For upper incisors, canines, premolars and
roots

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For upper roots and premolars with narrow beaks (can also be used for molars)

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For narrow roots and root splinters, for upper and lower jaws

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For lower molars, either side, gripping in depth

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For lower incisors, canines, premolars and roots

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The tooth forceps is positioned between the 7th and 8th tooth. By pressing the working ends together, the wisdom tooth can be removed.

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Tooth forceps for children for upper incisors,
canines and premolars

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Tooth forceps for children for lower incisors,
canines, premolars and molars

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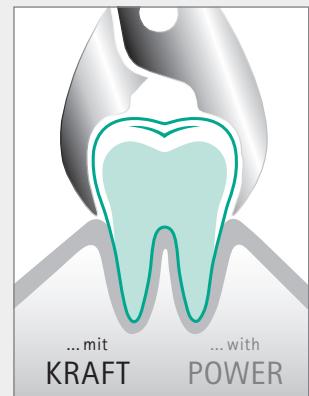
THERE ARE DIFFERENT WAYS TO EXTRACT A TOOTH ...

The importance of tissue preserving extraction is growing strongly. Especially in the field of implantology tissue preserving can avoid complex bone augmentation (1, 2). ExtraLux instruments enable easy and comfortable tissue preserving extraction of all teeth by

- Sharp working tips: Convenient cutting of the Sharpey bundles
- Thin working tips: Bone preserving cutting of the Sharpey's fibers
Easy access to the Periodontology gap
- Ergonomics: Enables a better control of force
- Haptics: Instruments are also handy intra-operatively

(1) Muska E, Walter C, Knight A, Taneja P, Bulsara Y, Hahn M, Desai M, Dietrich T. Atraumatic vertical tooth extraction: a proof of principle clinical study of a novel system. *Oral Surg Oral Med Oral Pathol Oral Radiol.* 2013 Nov;116(5):303-10.

(2) Sharma SD, Vidy B, Alexander M, Deshmukh S. Periotome as an Aid to Atraumatic Extraction: A Comparative Double Blind Randomized Controlled Trial. *J Maxillofac Oral Surg.* 2015 Sep;14(3):611-5.

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The straight model is particularly suitable for removing root residues.

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The curved elevators permit access to the maxillary molar roots.

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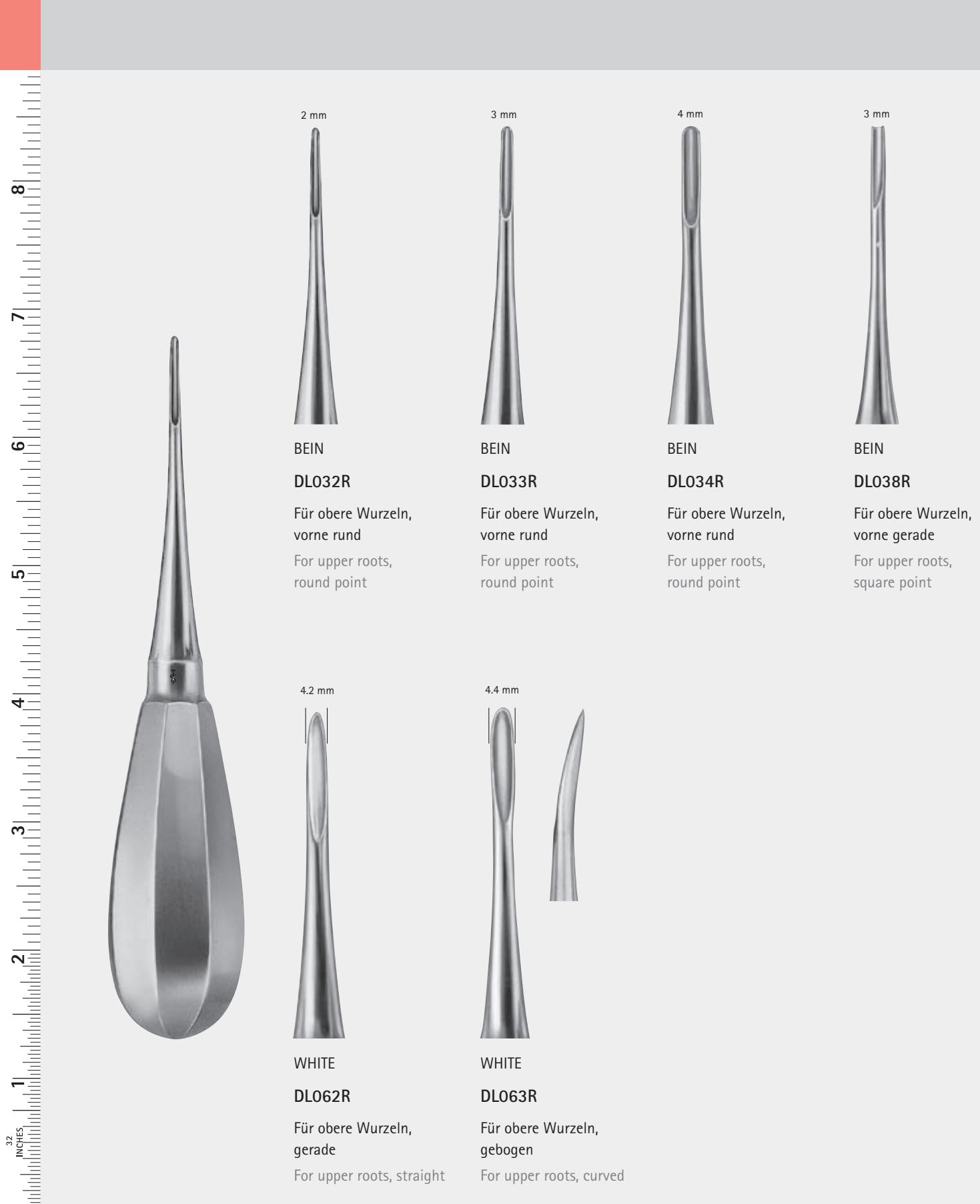
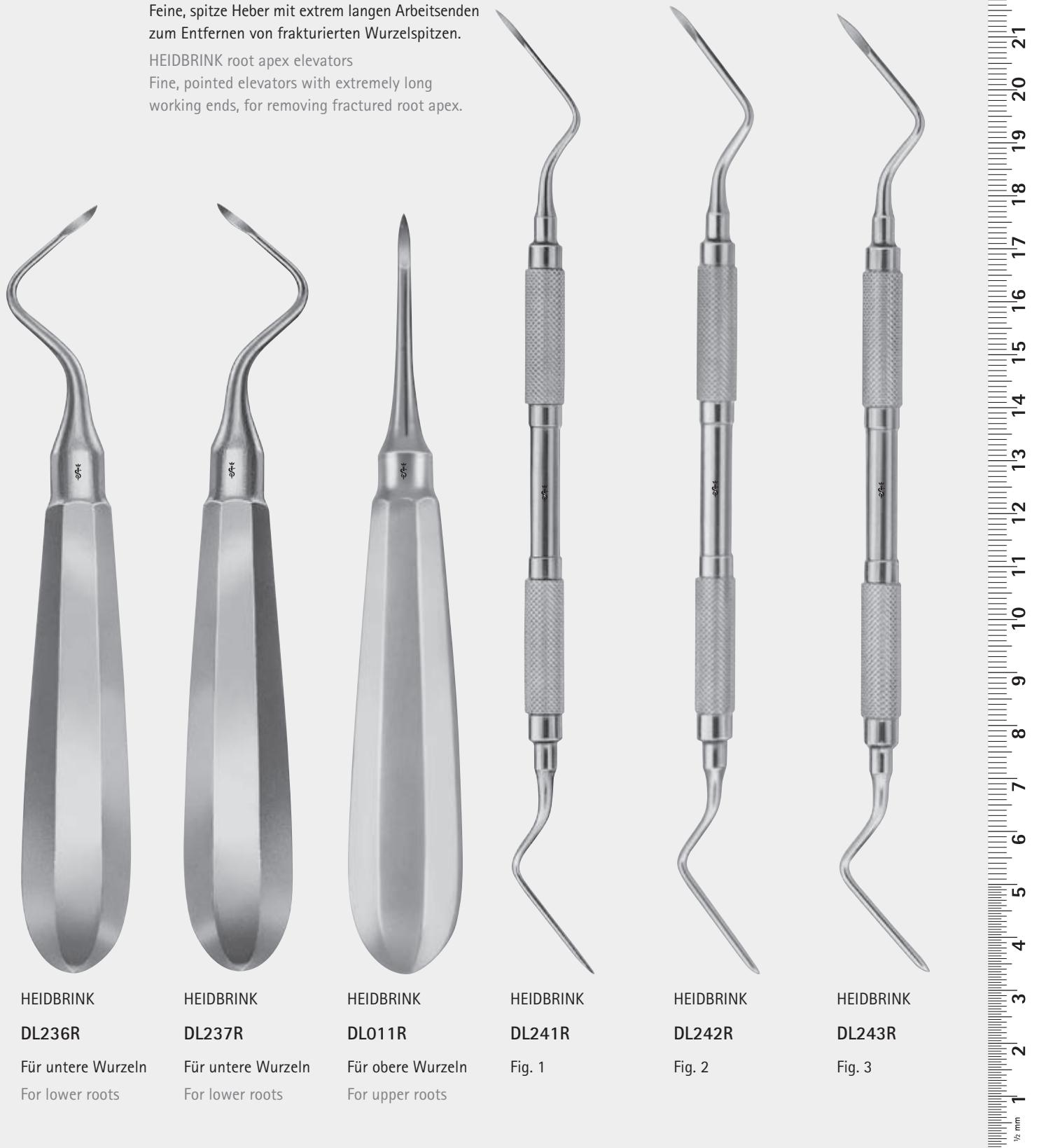
With a slight rotary movement, the LINDO-LEVIAN root elevator is pushed between alveolus and root until the groove becomes lodged in the roots. By pulling the root elevator straight down, the barb effect arises in which the root is extracted without leverage.

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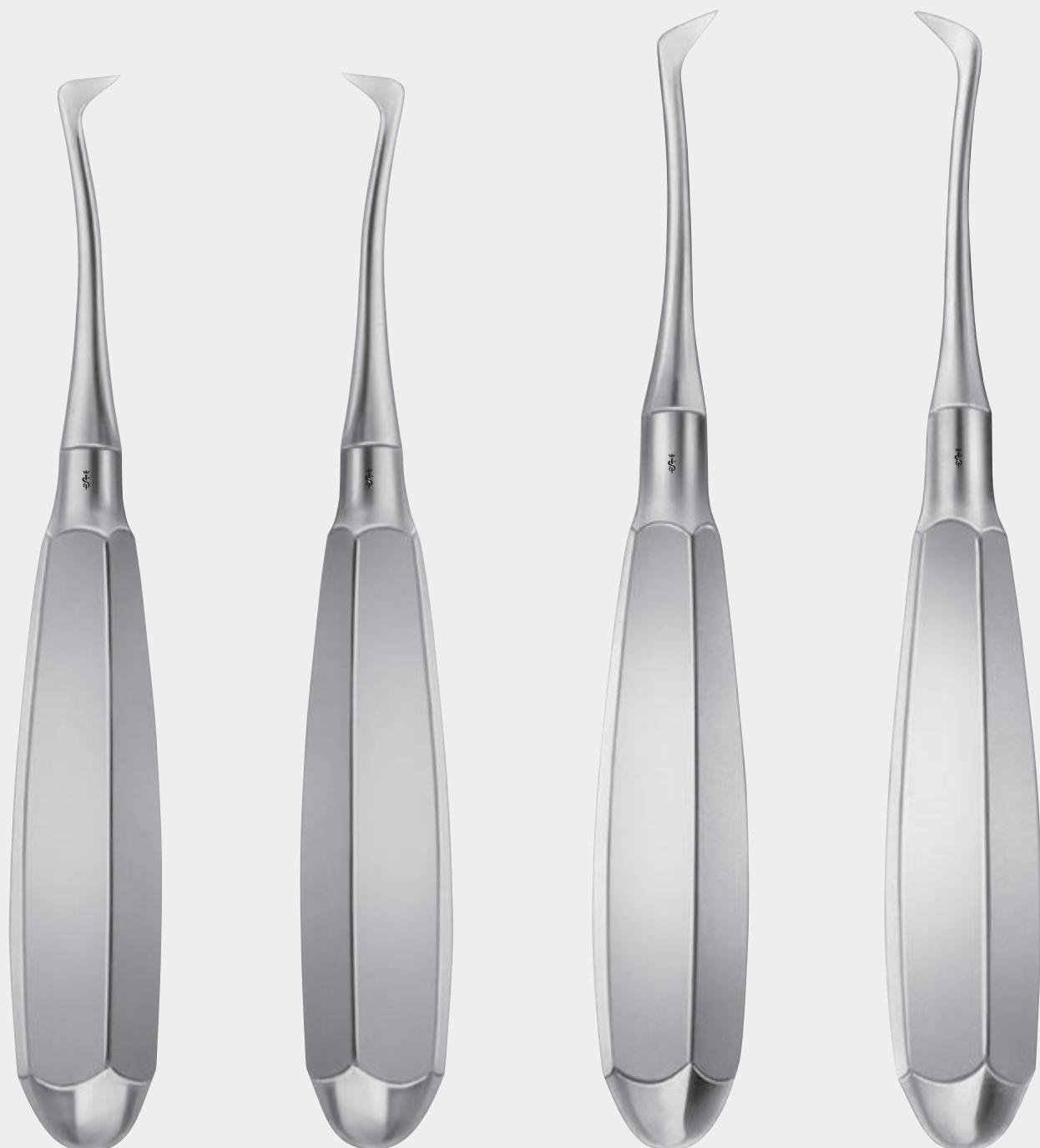
Important note:
 The luxation of the wisdom tooth is not achieved by using the second molar as the leverage point for the instrument. The instrument should be used with a rotational force in distal direction to luxate the tooth in a buccal movement.

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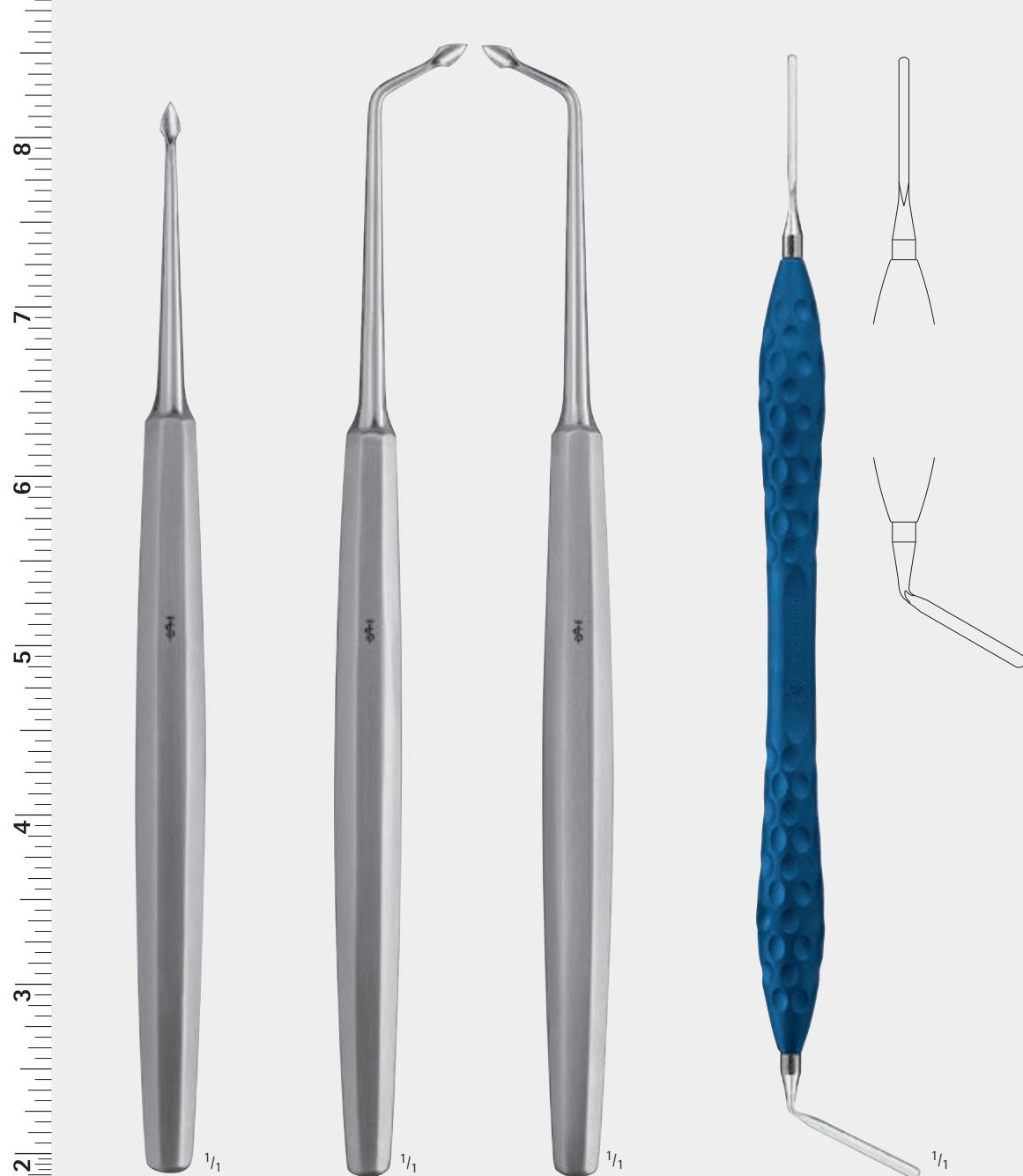
Continuous corrosion-resistant
instrument steel

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Instrument tight-covered
by a PEEK handle

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In close collaboration with Dr. Dr.
Andres Stricker, Freiburg Univer-
sity, an instrument range especially
designed for dental implantology
has been developed for ergonomics
concerning the working tips and
handle for any specific indication.

ERGOPLANT thus enables working
efficiently with minor physical
strain under practice conditions.

All ERGOPLANT instruments are
suitable for steam sterilization
134°C (Autoclav).

8
7
6
5
4
3
2
1
32 INCHES

BASIS INSTRUMENTE
BASIC INSTRUMENTS

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Sales unit: PAK = Package of 1 piece

Mikrochirurgische Klinge, stainless steel,
steril verpackt

Microsurgical blade, stainless steel,
sterile packed

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Titanium-microblade holder with round handle, light.
Easily rotated and easy to handle for precision incisions.

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Scalpel blade holder with round handle

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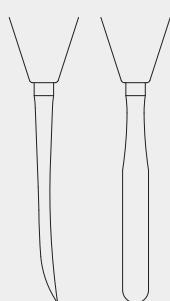
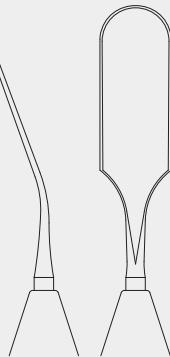
Mouth mirror, Rhodium
Specially recommended
for dental implantology
Sales unit: PAK =
Package of 12 pieces

PRICHARD

DX200R
Raspatorium
Raspatory
200 mm, 8"

BASIS INSTRUMENTE
BASIC INSTRUMENTS

PRICHARD

DB020R

Raspatorium, grazil

Raspatory, delicate

195 mm, 7 3/4"

MOLT

DB023R

Raspatorium

Raspatory

190 mm, 7 1/2"

FREER

DB025R

Raspatorium, scharf / stumpf

Raspatory, sharp / blunt

190 mm, 7 1/2"

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DENTAL IMPLANTOLOGY

32 INCHES

BUSER

DX301R

Mikro-Raspatorium

Micro-Raspatory

180 mm, 7 1/2"

FRETWURST

DB024R

Raspatorium/Desmotom

Raspitory/Desmotomes

190 mm, 7 1/2"

BASIS INSTRUMENTE
BASIC INSTRUMENTS

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Atraumatic dissecting of the flap by means of the papilla elevator without any damage to the papillae.

PHW

DX405R

Atraumatischer Papillenheber
Atraumatic papilla elevator
190 mm, 7 1/2"

BUSER

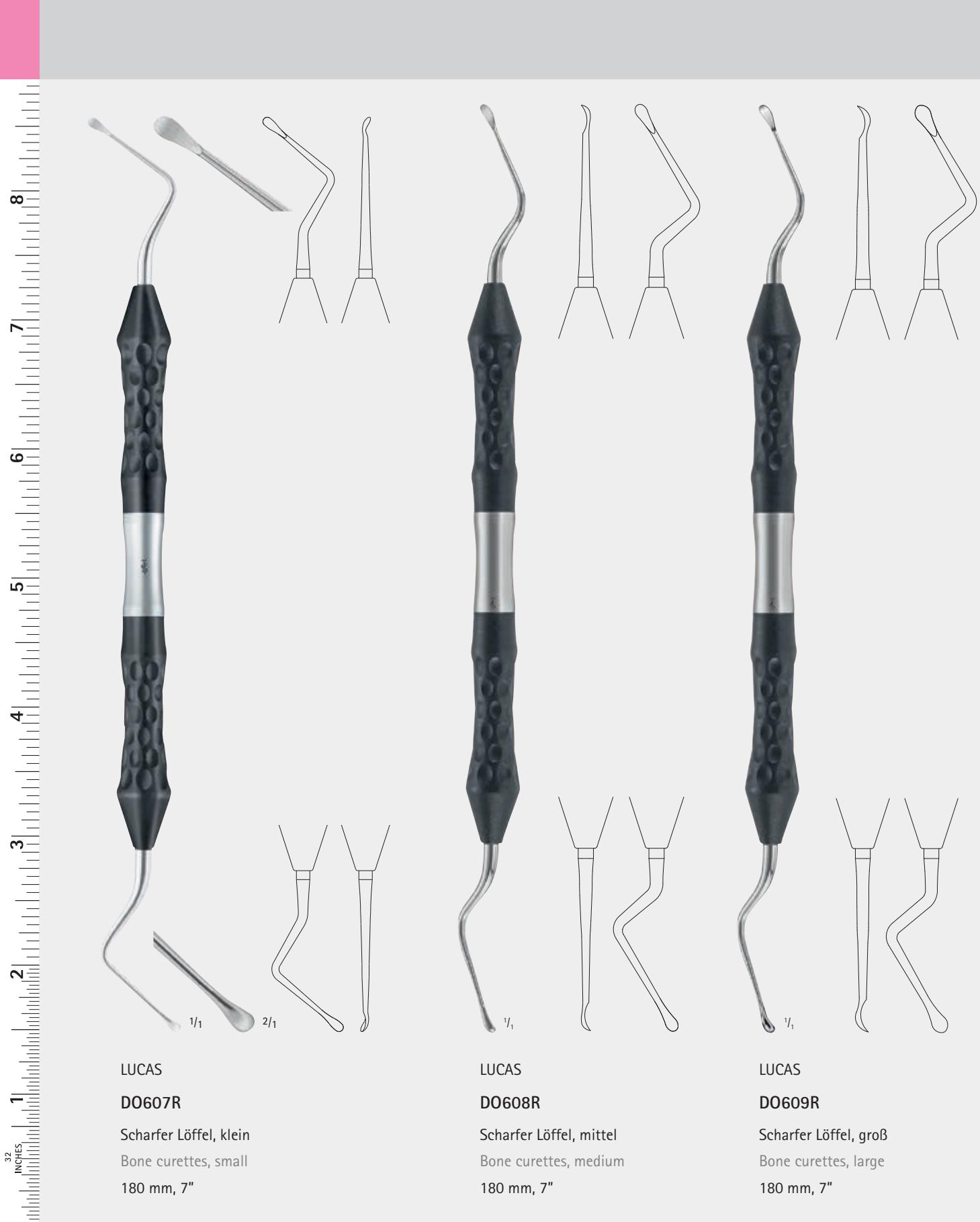
DX201R

Raspatorium
Raspatory
185 mm, 7 1/4"

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DENTAL IMPLANTOLOGY

BASIS INSTRUMENTE
BASIC INSTRUMENTS

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The straight tunnelling instrument is mainly used in the upper jaw, in the frontal and lateral region. This allows large areas of the gingiva to be undermined using both tunnel and sulcular techniques.

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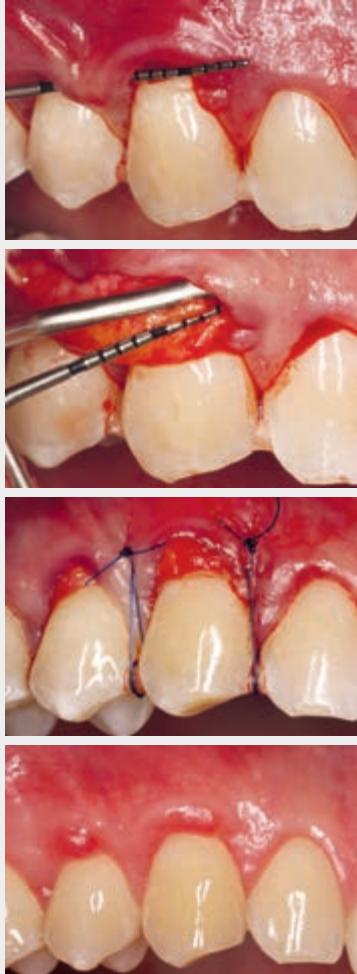
The angled tunnelling instrument is primarily used for preparing lateral, buccal regions. It can also be used distally, without tearing the mucosal tissue and also for the lower jaw.

195 mm, $7\frac{3}{4}$ "

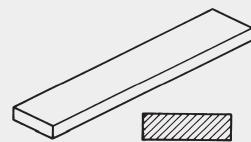
JG598

Sterilit® i AESCULAP®-Schleifstein-Öl

Inhalt: 50 ccm

Sterilit® i AESCULAP® oil for grinding stones

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Original Arkansas grinding stone,

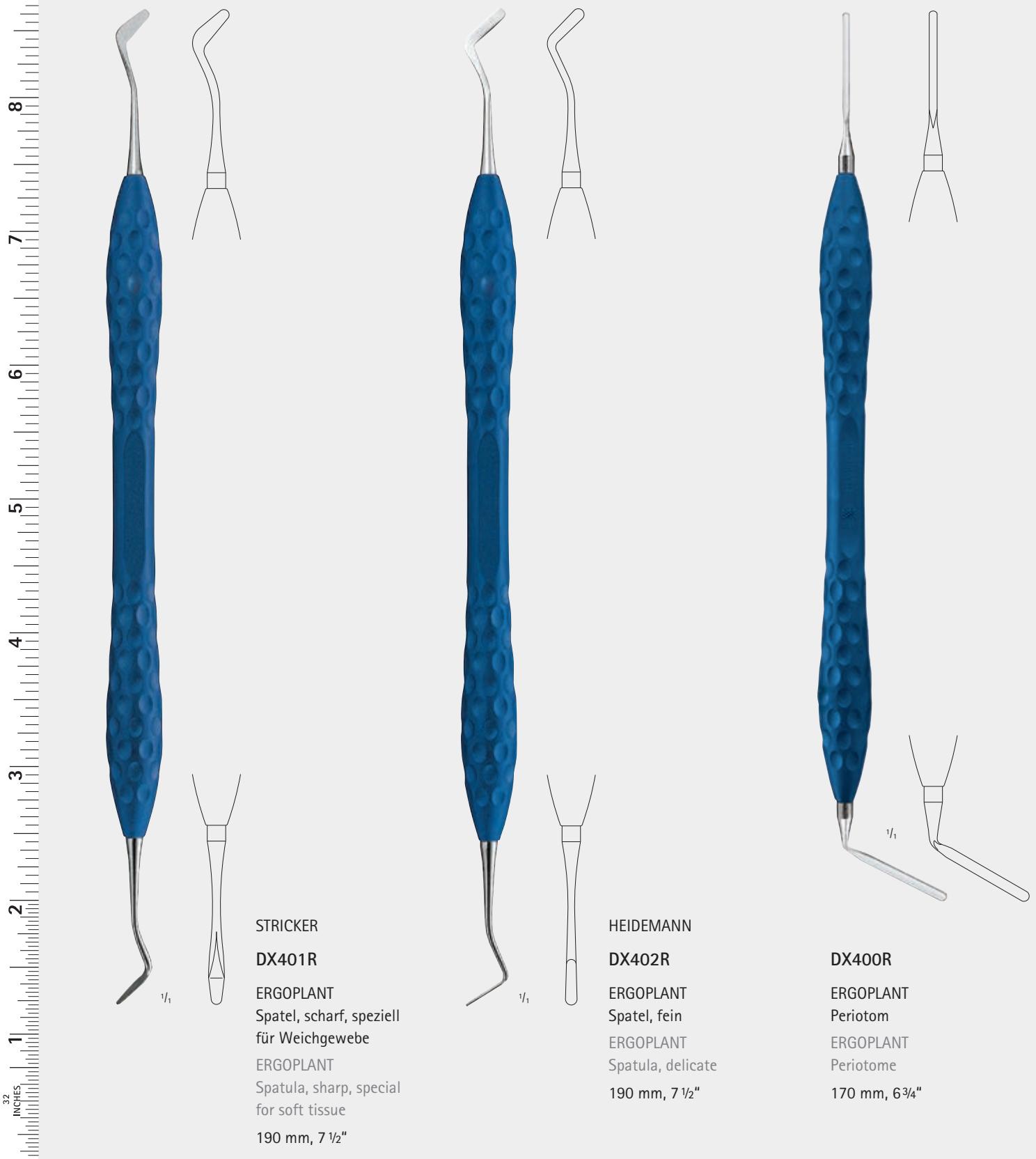
for grinding on tunnelling instruments

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- 3 Jahre Garantie
- Vergoldete Ringe

DUROTIP®-Scissors with inserted carbide cutting edges

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~W-Cut directly after the catalogue number indicates that the resp. item has one serrated cutting edge. (When ordering please indicate **~W**-cut together with the catalogue number e.g.: BC211W)

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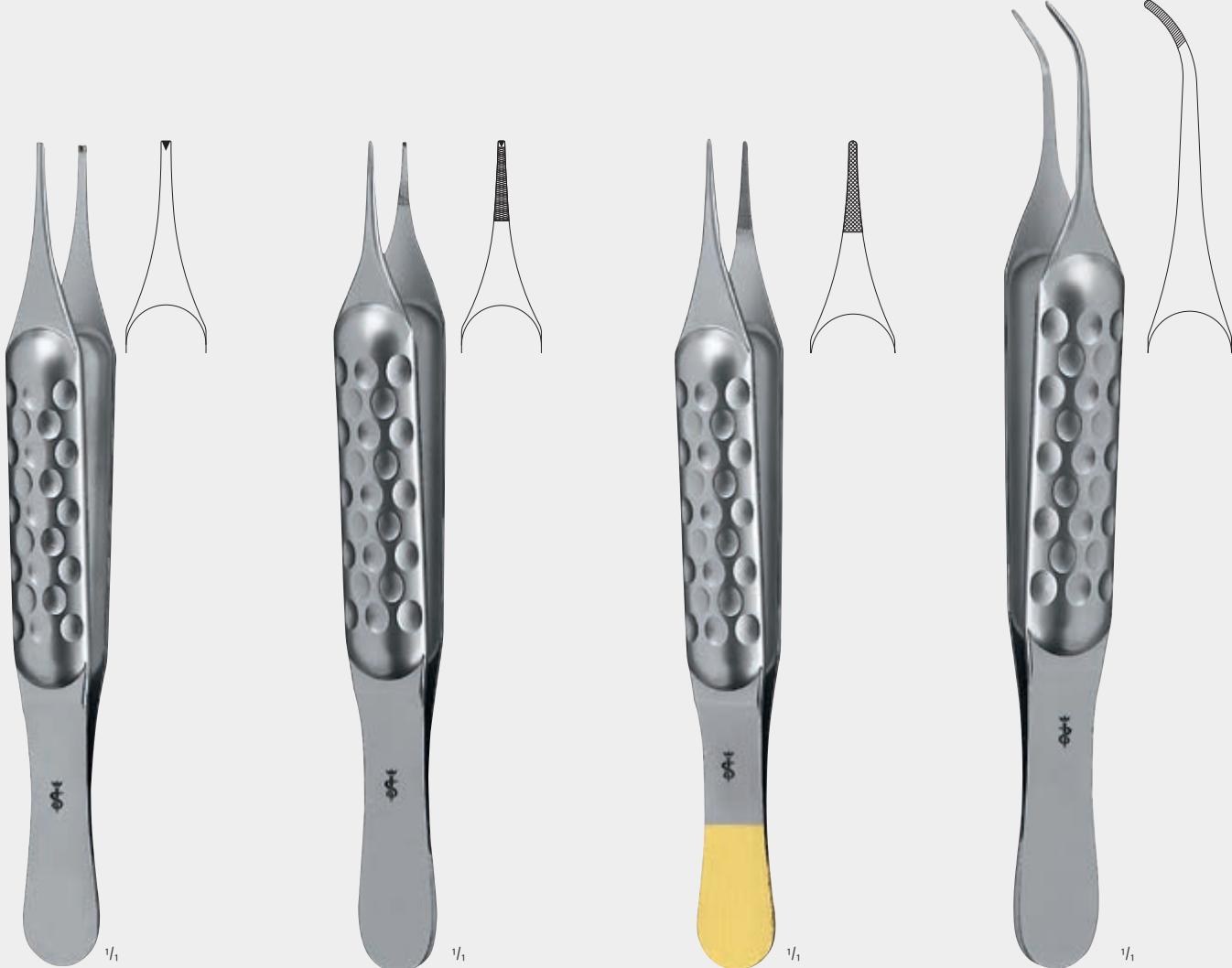
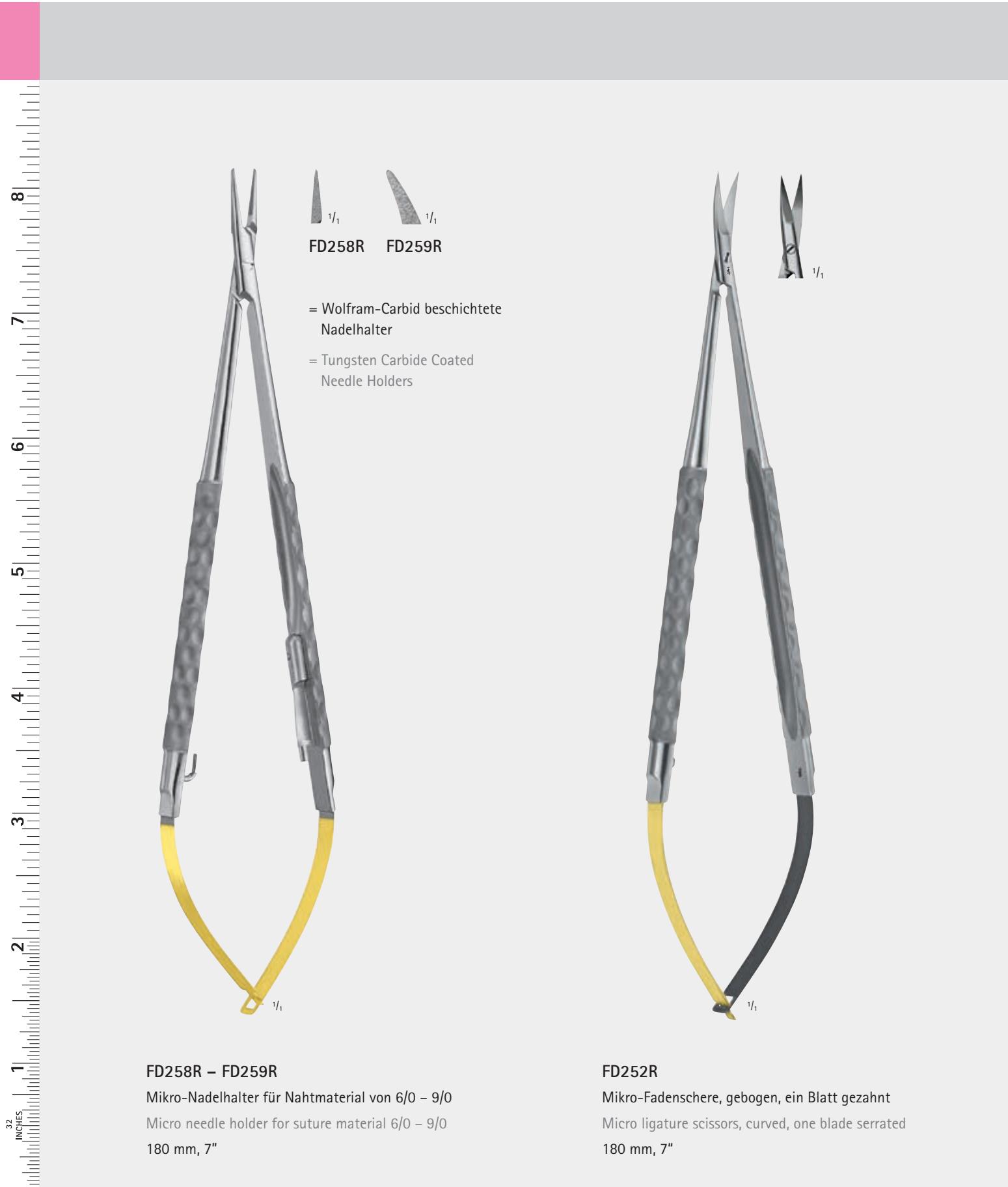
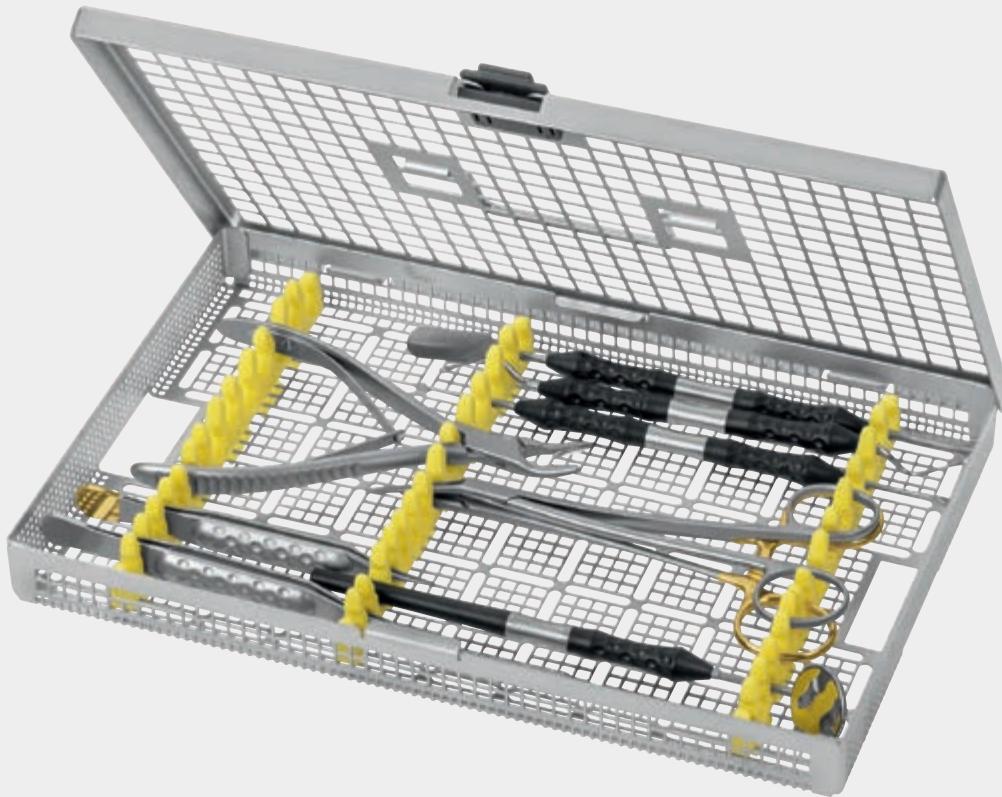
DX705

AESCLAP® Basic Kit for dental implantology,
consisting of:

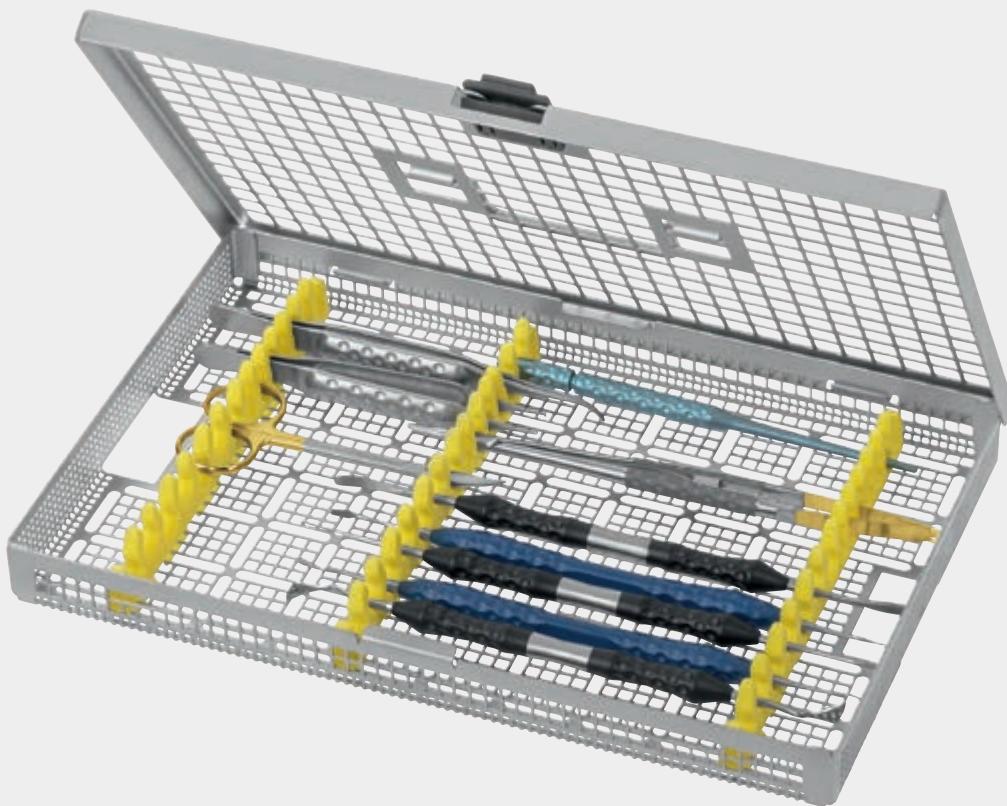
- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- BB063R 1 Scalpel handle, No. 3
- DX094 1 Ergoperio Mouth mirror holder
- DA036R 1 PAK = Package of 12 Rhodium-Mouth mirror, Ø 22 mm
- DX051R 1 ERGOPLANT Tissue forceps
- DX052R 1 ERGOPLANT Dissecting forceps
- DX201R 1 ERGOPLANT Raspatory, BUSER
- DX200R 1 ERGOPLANT Raspatory, PRICHARD
- DX220R 1 Ergoperio Bone curette, LUCAS
- BM013R 1 DUROGRIP® Nadelhalter, BABY-CRILE-WOOD
- DO219R 1 Gum scissors, LOCKLIN
- FO409R 1 Bone rongeur, LUER-FRIEDMANN
- JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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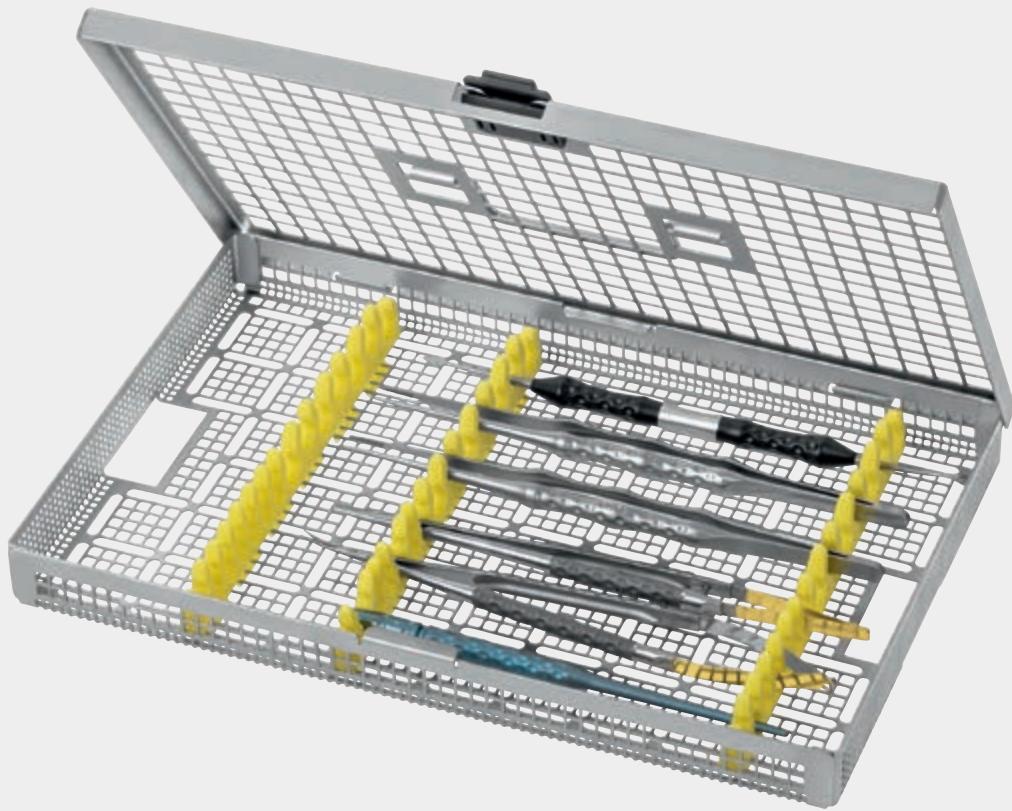
DX707

AESCULAP® Soft Tissue Kit for dental implantology, consisting of:

- BB045T 1 Titanium scalpel handle
- BB367R 1 PAK = Package of 10 micro scalpel blades, sterile, Fig. 367
- BB063R 1 Scalpel handle, No. 3
- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- DX050R 1 ERGOPLANT Tissue forceps
- BM003R 1 DUROGRIP® Needle holder
- DX450R 1 ERGOPLANT Gingivectomy knife, KIRKLAND
- DB025R 1 Ergoperio Raspatory FREER
- DX405R 1 ERGOPLANT Atraumatic papilla elevator, PHW
- DX401R 1 ERGOPLANT Spatula, sharp, STRICKER
- DX402R 1 ERGOPLANT Spatula, HEIDEMANN
- BC211W 1 Delicate scissors, curved
- DX055R 1 ERGOPLANT Soft tissue forceps
- JG381R 1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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DX720

Micro Soft Tissue Set, consisting of:

- BB045T Titanium-microblade holder, 145 mm
- DX301R Micro-Raspatoria, 180 mm
- DX070R Micro atraumata forceps, 180 mm
- FD252R Micro ligature scissors, curved, one blade serrated, 180 mm
- DX060R Micro dissecting forceps, 180 mm
- BM003R DUROGRIP® Needle holder, 180 mm
- JG381R Tray, 274 x 172 x 30 mm

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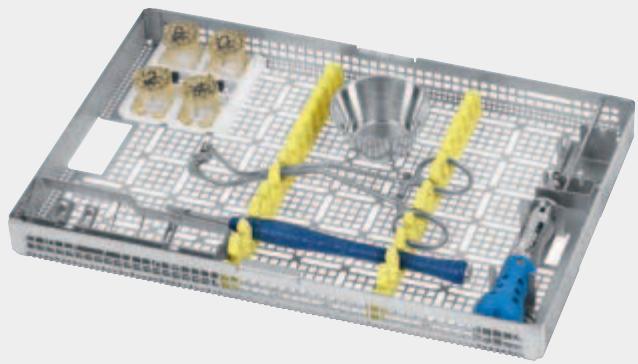
For advanced alveolar crest atrophies less than three millimeters wide it is preferable to pursue a two-stage approach, in which, an autologous bone graft from appropriate autologous donor sites should be transplanted to the deficient alveolar crest atrophy region prior to the actual implantation. This is exactly what the AESCULAP® Bone Fixation Set was developed for.

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Standard Titanium Screws 1.5 mm

High stability

- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy Ti-6Al-4V ensures high stability of the screws

Small & flat screw head

- The screw head is easy to countersink in the bone transplant

Micro thread

- Allows a good fixation even in very thin residual bone

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Screw magazine with 8 screws, self-taping, cross recess, Ø 1.5 mm

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Micro screws 1.2 mm

Micro screw head = Standard screw head

- Reduces the risk of screw fracturing

High stability

- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy allows high stability of the screws

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Screw magazine with 8 screws, self-taping, cross recess, Ø 1.2 mm

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Screw magazine

Mechanical cleanable

- The new open magazine storage system enables mechanical cleaning of the implants

Simple documentation

- The batch numbers are printed on the screw magazines

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Screwdriver

Ergonomic

- Reduces the force required for tightening and loosening the screw

Soft tissue sleeve

- The screw safety sleeve reduces the danger of soft tissue injuries

Screw fall protection

- The screw safety sleeve serves as a screw fall protection

Clean

- The open design enables efficient cleaning

Friction

- Enables a strong retention for the screw

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Bone fixation forceps:

■ Universal

Universal application in the upper and lower jaw.

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■ Adaptable

The working tips adapt to the respective anatomical situation.

■ Delicate

The delicate working tips enable drilling and screwing even in smaller bone blocks.

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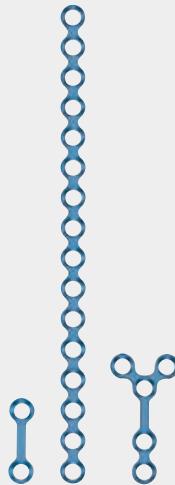
Ø 1.2 mm Ø 1.5 mm

Länge / length

DX646T	6 mm	-
DX648T	8 mm	DX608T
DX650T	10 mm	DX610T
-	12 mm	DX612T
-	14 mm	DX614T

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Screw magazine with 8 screws,
self-taping, cross recess

- FM960T**
2-Loch-Platte, gerade
2-hole plate, straight
- FM965T**
16-Loch-Platte, gerade
16-hole plate, straight
- FM964T**
Y-Platte, 5-Loch
Y-plate, 5-hole

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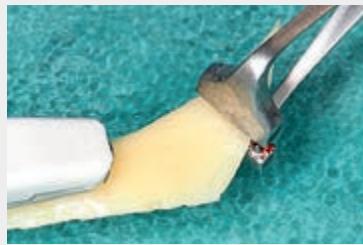
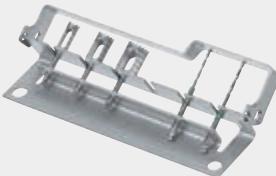
Rack for screw magazines
(Rack without contents)

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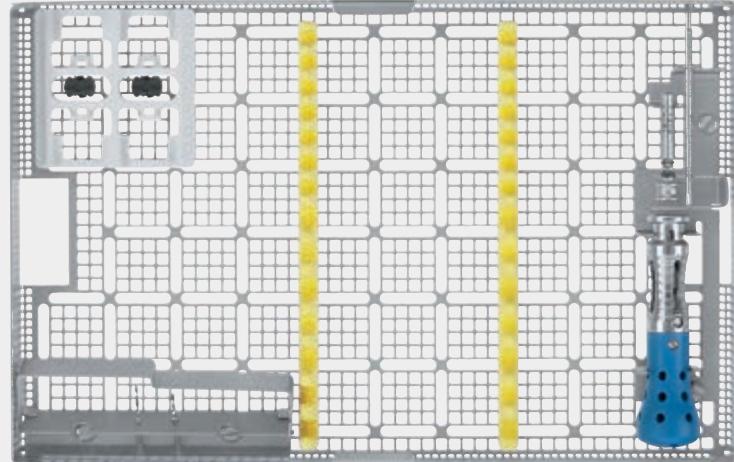
Rack for twist drills
and tools
(Rack without contents)

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DX715

Bone Fixation Set including tray with the
following contents:

JG403R	Dental tray
DX620	Rack for screw magazines
DX621R	Rack for twist drills
DX630R	Twist drill, Ø 1.25 mm
DX631R	Twist drill, Ø 1.5 mm
DX715200	Rack for screw driver
DX605R	Screw driver
DX606R	Screw driver blade
JG391	Silicone instrument rack (2x)

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Permanent filter for sterile container,
Order unit: PAK = Pack with 2 pieces

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Autogenous bone in augmentation has been established as the „golden standard“. The region where to gain the bone (Bone Harvesting) and the required consistence for augmentation can vary, depending on the case. Here the ERGOPLANT Micro Bone Mill shows its flexibility whereat a high yield of bone-grist is always ensured.

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ERGOPLANT Micro bone mill

- Small and handy
- Interchangeable cutting disks
- Cutting disks available for finer or coarser bone particles

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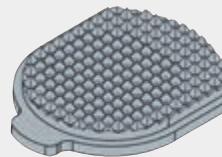
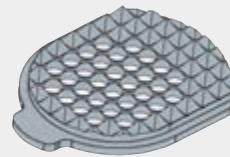
ERGOPLANT
Micro bone mill
(delivered with coarse
cutting disk DX803R)

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ERGOPLANT
Cutting disk for DX801R, fine

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ERGOPLANT
Cutting disk for DX801R, coarse

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Cleaning fork for punching out
valuable bone material of all cutting
disk holes at once in a row.

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ERGOPLANT
Bone mill cleaner fine
for DX802R

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DX800

AESCULAP® Micro Bone Mill Kit,
consisting of:

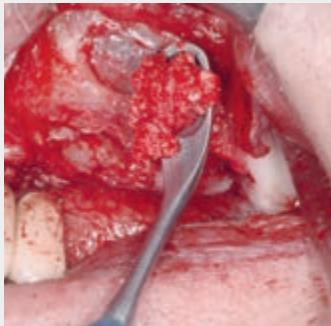
- DX801R 1 ERGOPLANT Micro bone mill
(incl. DX803R cutting disk, coarse)
- DX802R 1 ERGOPLANT Cutting disk, fine
- DX810R 1 ERGOPLANT Bone mill-cleaner, fine
- DX811R 1 ERGOPLANT Bone mill-cleaner, coarse
- JF284R 1 Tray (274 x 172 x 60 mm)
with storage aid for micro bone mill,
suitable for dental container JN095
- TE888 1 Packing stencil (only available for DX800)

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Enables carrying and placing of enough bone material. The other working tip serves for pushing and plugging bone material into the augmentation area.

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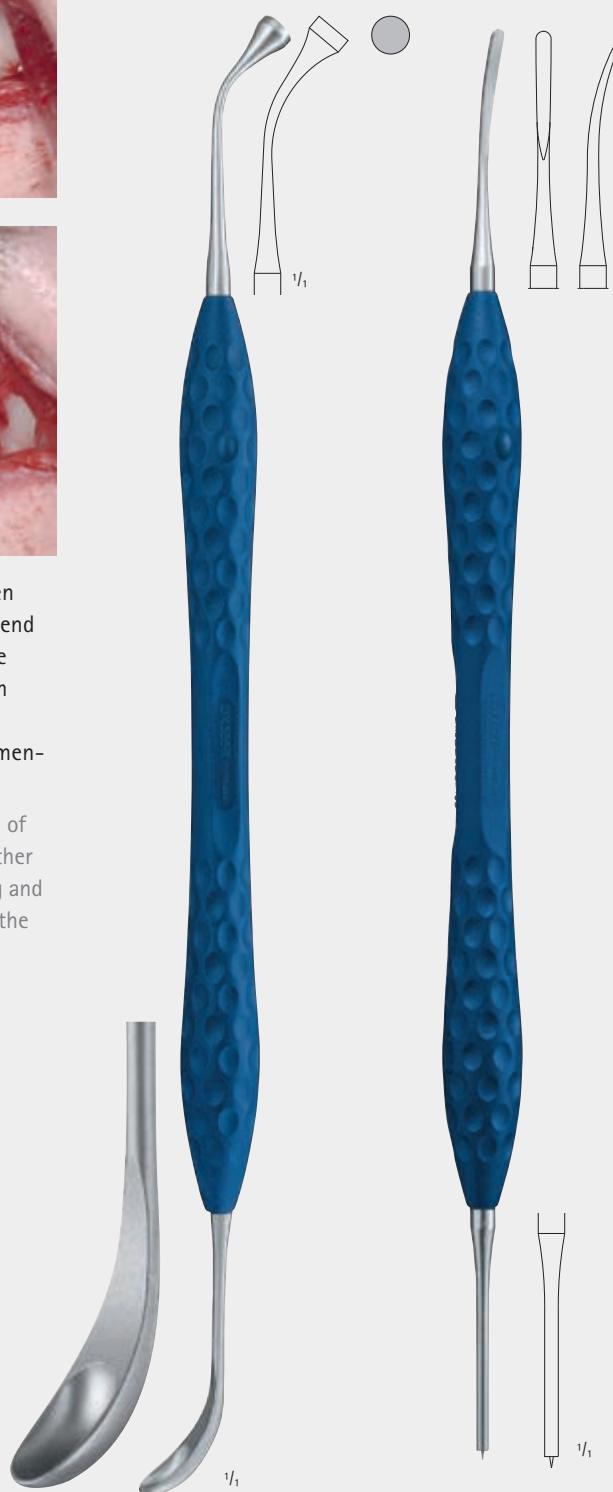
ERGOPLANT
Applicator for
bone material

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The small surface of the pointed tip avoids the sticking of the humid membrane. The small spatula on the other end, helps to take out bone material of, for instance, the bone files DX252R – DX253R, as shown on page 300.

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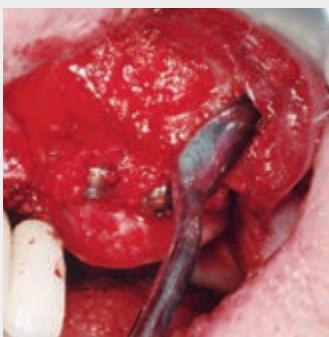
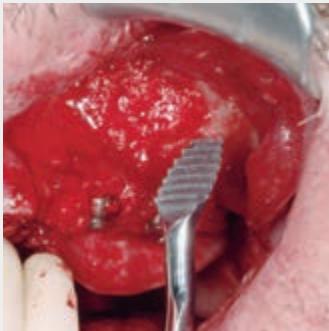
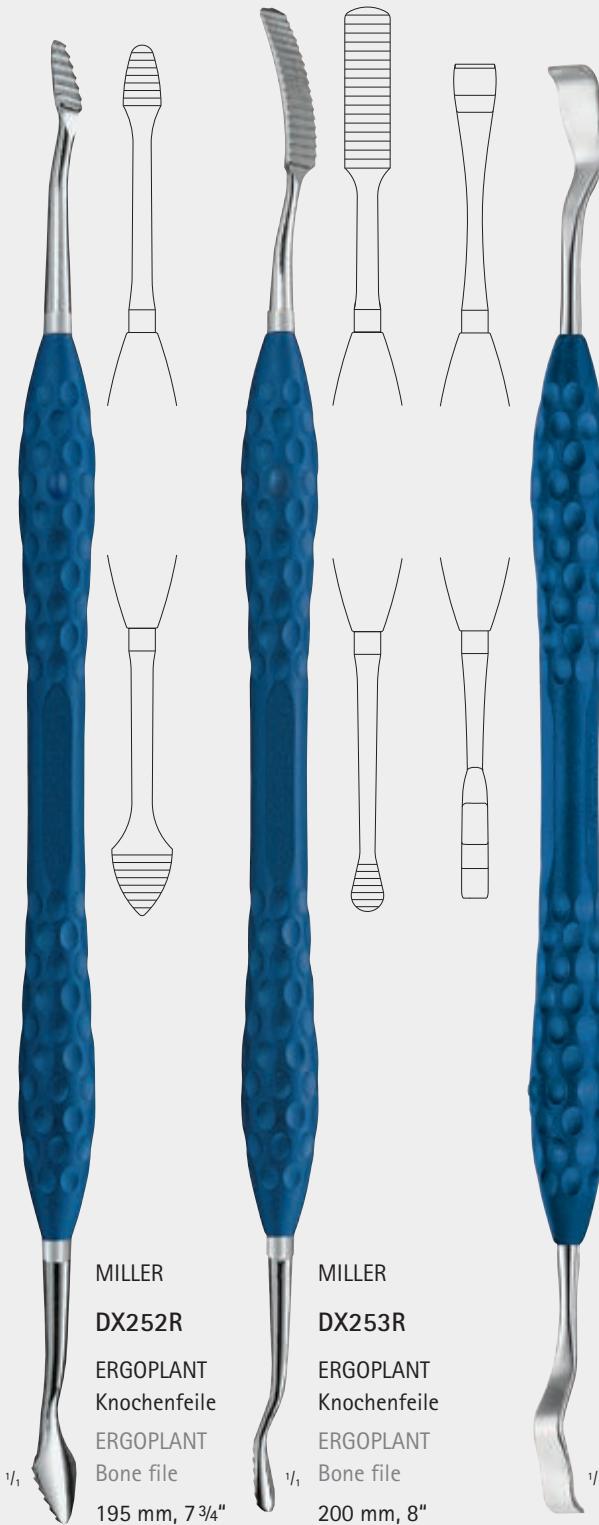
If smaller quantity of bone material is required, with the bone files DX252R – DX253R it can be harvested directly from the jawbone, without additional trauma for the patient. The quantity is often sufficient, for augmentation around the implant-neck.

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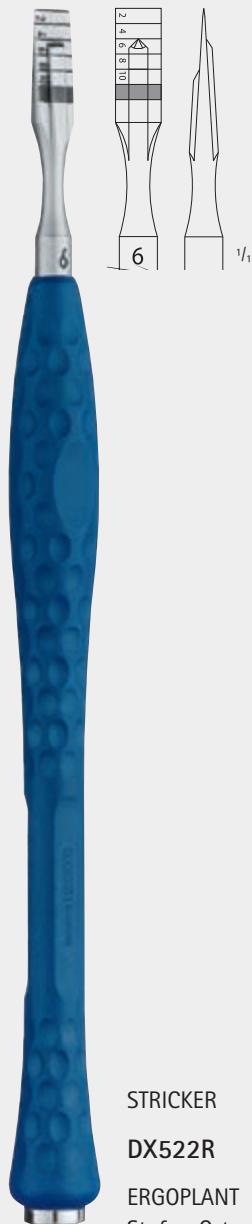
With this raspatory fine bone particles can be harvested by simply scratching over the jawbone.

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The Step-Osteotome was specially developed for reliable bone-splitting with simultaneous stepwise widening. Due to the specially designed working tips, the maxillary crest will be first expanded towards palatal and only afterwards towards vestibular. In the same time spongy bone in the center will be condensed. After removing the Step-Osteotome, a pre-selection point for the implant bur can be clearly seen.

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DX708

AESCLAP® Bone Split Kit for dental implantology,
consisting of:

DX510R 1 ERGOPLANT Osteotome, straight, 3 mm
DX511R 1 ERGOPLANT Osteotome, straight, 4 mm
DX512R 1 ERGOPLANT Osteotome, straight, 6 mm
DX513R 1 ERGOPLANT Osteotome, straight, 8 mm
DX517R 1 ERGOPLANT Osteotome, curved, 4 mm
DX518R 1 ERGOPLANT Osteotome, curved, 6 mm
DX522R 1 ERGOPLANT Step-Osteotome, 6 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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The maxillary crest is splitted transversal by the Step-Osteotome.

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The bone split is kept apart by the retractor.

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The implant can be positioned by using the existing bone.

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After splitting the maxillary crest the bone split is closing within a short period of time. This leads to an unwished and unnecessary bone loss when drilling afterwards. This causes the risk of an implant that is not exactly positioned and additional augmentative treatments could be necessary. The bone split is kept apart by the retractor. Thus, the existing bone can be used better to position the implant.

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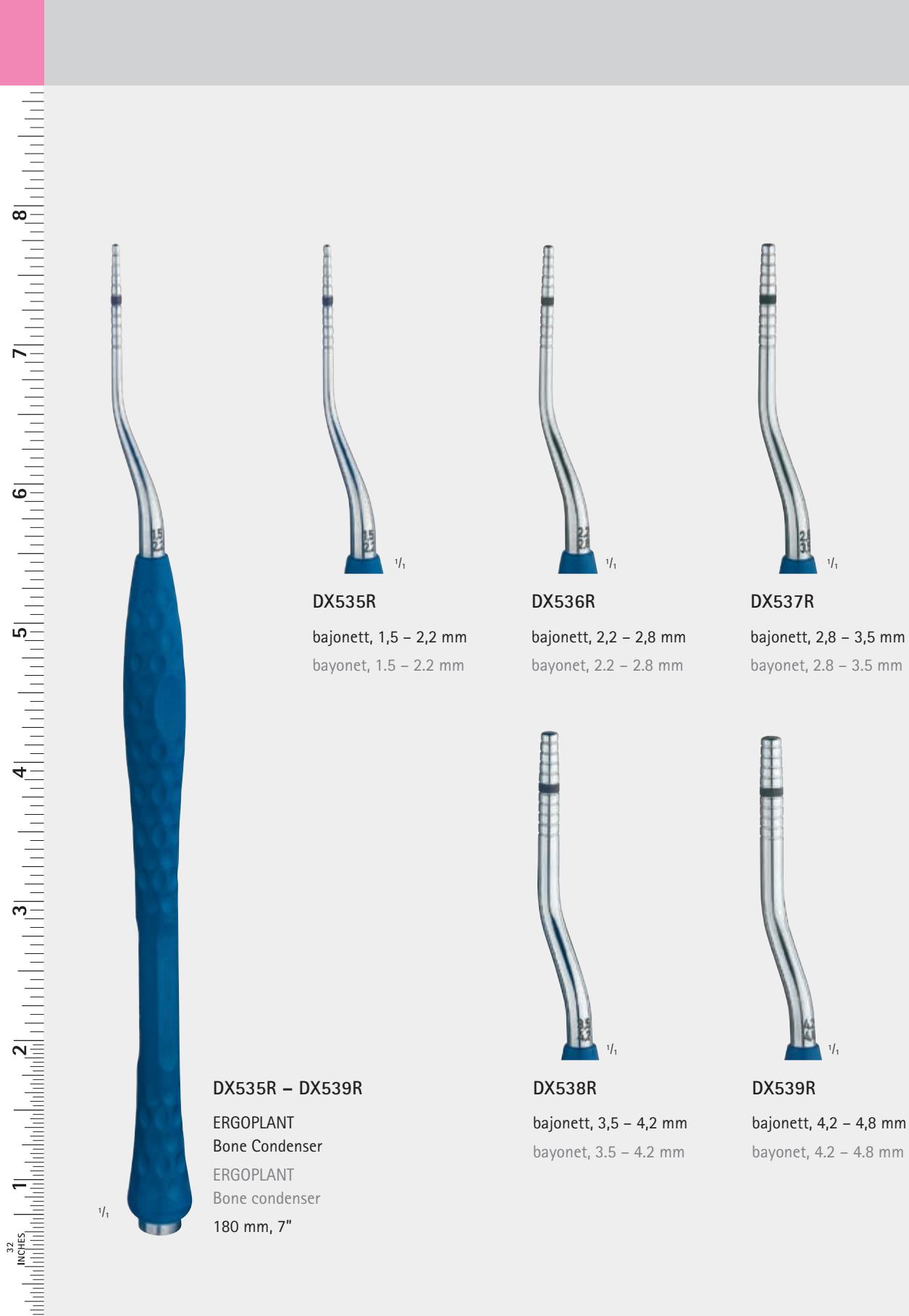
DX709

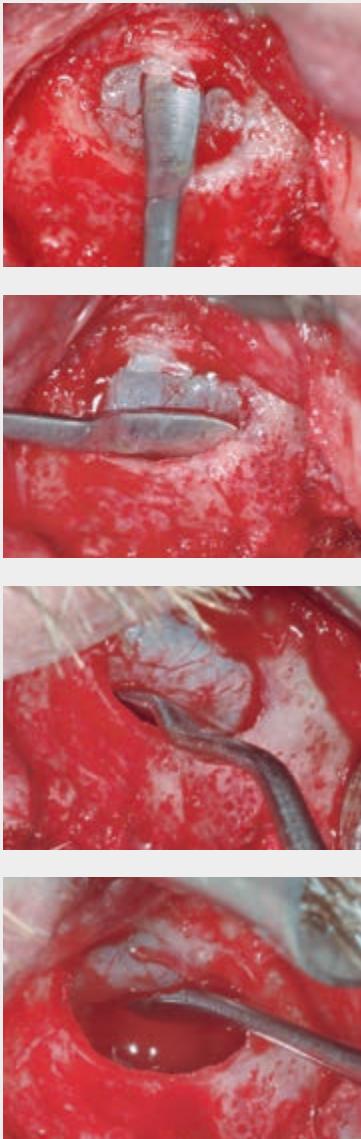
AESCALAP® Bone Condenser Kit for dental implantology,
consisting of:

DX530R 1 ERGOPLANT Bone condenser, straight, 1.5 – 2.2 mm
DX531R 1 ERGOPLANT Bone condenser, straight, 2.2 – 2.8 mm
DX532R 1 ERGOPLANT Bone condenser, straight, 2.8 – 3.5 mm
DX533R 1 ERGOPLANT Bone condenser, straight, 3.5 – 4.2 mm
DX534R 1 ERGOPLANT Bone condenser, straight, 4.2 – 4.8 mm
DX535R 1 ERGOPLANT Bone condenser, bayonet, 1.5 – 2.2 mm
DX536R 1 ERGOPLANT Bone condenser, bayonet, 2.2 – 2.8 mm
DX537R 1 ERGOPLANT Bone condenser, bayonet, 2.8 – 3.5 mm
DX538R 1 ERGOPLANT Bone condenser, bayonet, 3.5 – 4.2 mm
DX539R 1 ERGOPLANT Bone condenser, bayonet, 4.2 – 4.8 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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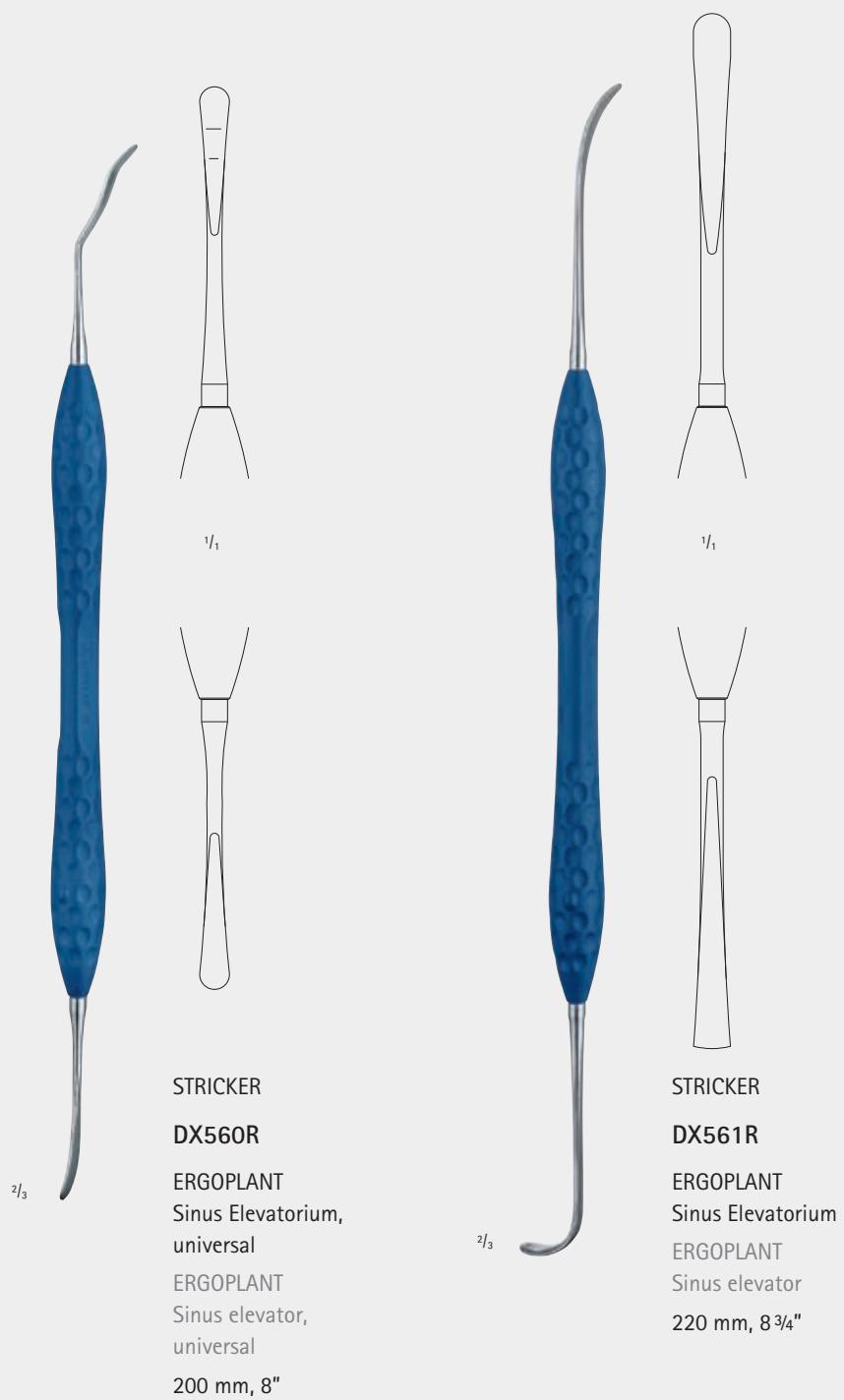
Separating the maxillary sinus membrane carefully from the bony base. Finally the sinus window will be pushed with the other working tip gently into the hollow maxillary cave.

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The Schneiderian membrane of the sinus can be detached with elevator D0571R by 1 – 2 mm along the total cortical bone.

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Elevator D0573R is particularly useful in the lower part for detaching the membrane further. The same goes for the caudal area, especially when the membrane is 4 – 5 mm above the base of the maxillary sinus and the maxillary sinus is built up with granular material.

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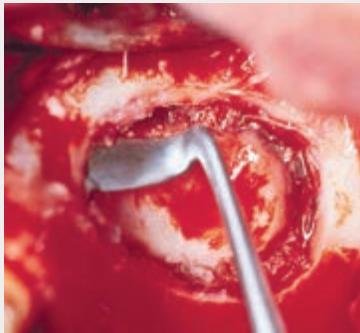
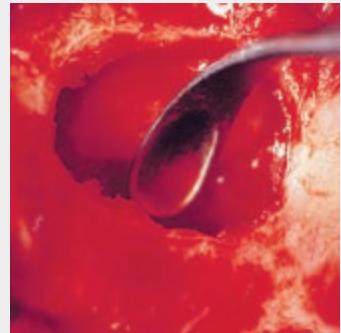
Elevator D0575R makes it possible to lift the membrane further in mesial region. One working tip is specially designed for the right maxillary sinus and the other for the left maxillary sinus.

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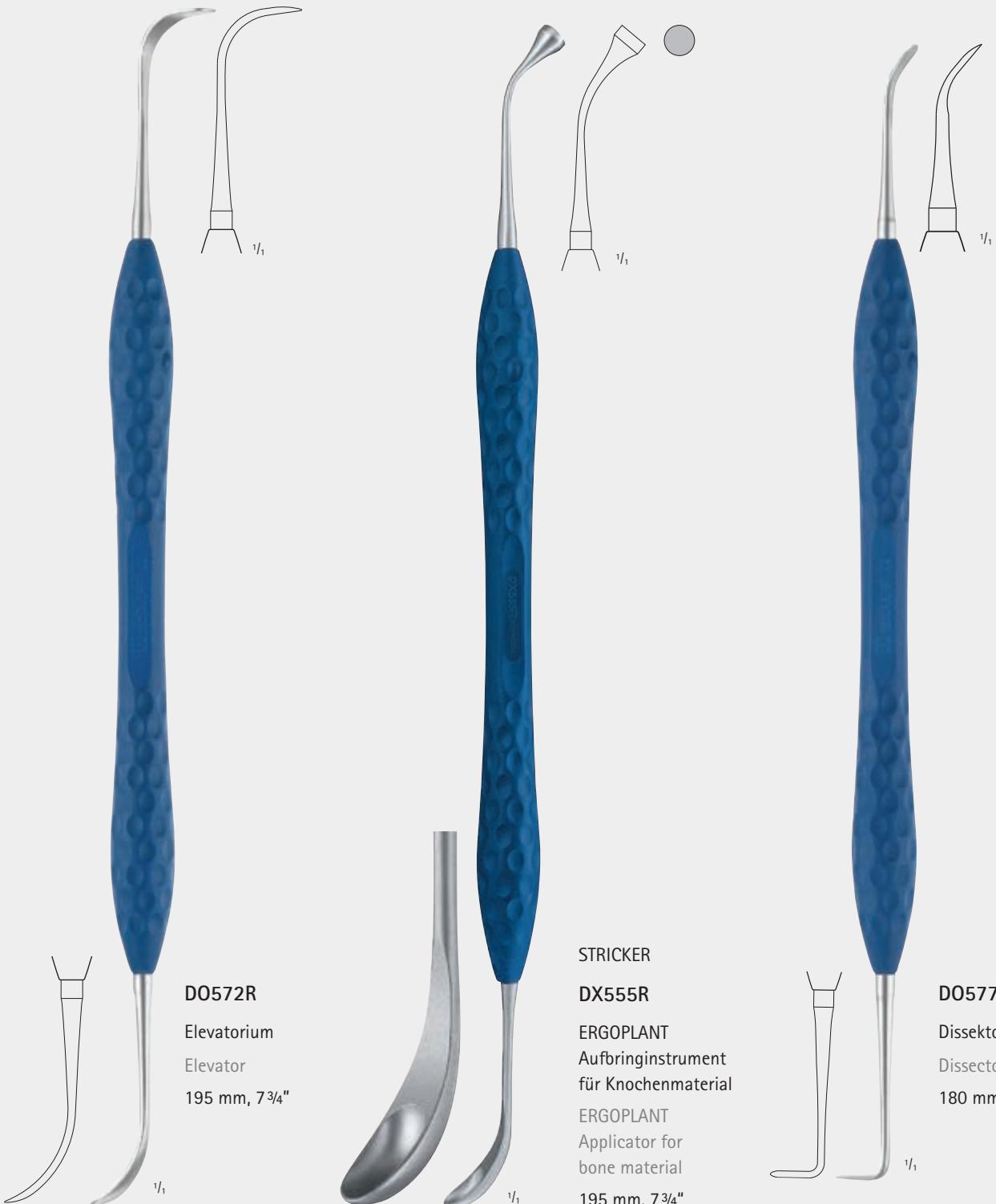
With elevator D0574R, the sinus membrane is further elevated in the upper and posterior part of the maxillary sinus wall.

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With the aid of elevator D0572R, the sinus membrane is further detached over a broad area in the inner region, especially along the roof of the palate. Condenser DX555R permits positioning of the filling material.

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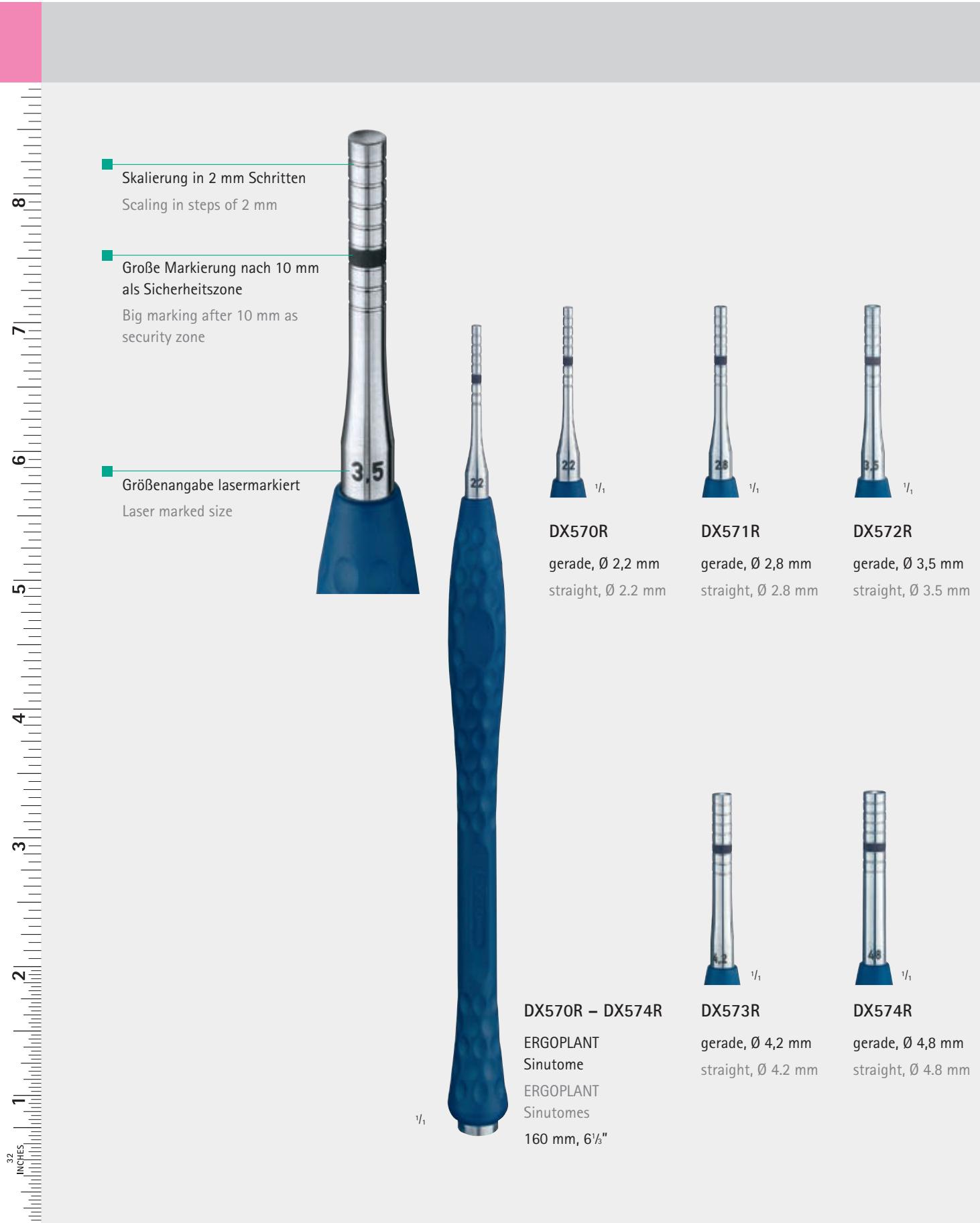
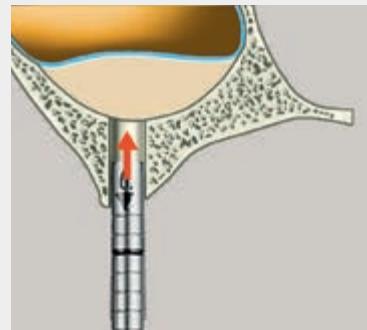
This illustration shows the technique of the percrestale sinus elevation.

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DX710

AESCLAP® Sinutome Kit for dental implantology,
consisting of:

DX570R 1 ERGOPLANT Sinutome, straight, Ø 2.2 mm
DX571R 1 ERGOPLANT Sinutome, straight, Ø 2.8 mm
DX572R 1 ERGOPLANT Sinutome, straight, Ø 3.5 mm
DX573R 1 ERGOPLANT Sinutome, straight, Ø 4.2 mm
DX574R 1 ERGOPLANT Sinutome, straight, Ø 4.8 mm
DX575R 1 ERGOPLANT Sinutome, bayonet, Ø 2.2 mm
DX576R 1 ERGOPLANT Sinutome, bayonet, Ø 2.8 mm
DX577R 1 ERGOPLANT Sinutome, bayonet, Ø 3.5 mm
DX578R 1 ERGOPLANT Sinutome, bayonet, Ø 4.2 mm
DX579R 1 ERGOPLANT Sinutome, bayonet, Ø 4.8 mm
JG381R 1 Tray (273 x 176 x 30 mm),
suitable for dental container JN092

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Plastic identification labels,
white, with 10-digit barcode
for praxiDoc or DIOS software
applications, labeling as specified

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These instruments were developed to hold back lips, cheeks, tongue and lingual mucous membrane. Due to the ergonomic handle design, a more relaxed working with minor physical strain is possible, even in long operations.

DX100R
ERGOPLANT
Wangenhalter
ERGOPLANT
Cheek retractor
215 mm, 8 1/2"

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Sales unit: PAK = Package of 2 pieces
suitable for steam sterilization 134°C (Autoclav)

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ERGOPLANT
Silicone mouth wedge
Size: S

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ERGOPLANT
Silicone mouth wedge with metal core
Size: M

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ERGOPLANT
Silicone mouth wedge with metal core
Size: L

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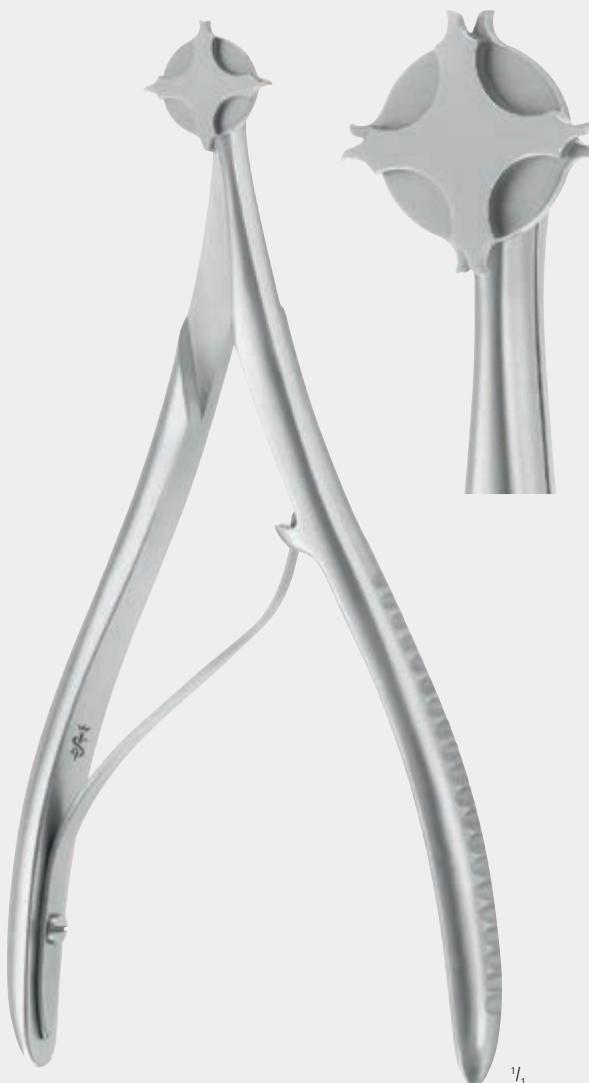
At the beginning the crown is slit parallel with a relatively thin instrument.

To produce parallel slit edges which can be easily gripped with the pliers, a cylindrical instrument with Ø of approx 1.0 mm is guided perpendicular to the tooth center line.

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The teeth of the crown widening pliers are positioned in order to spread the crown.

Finally the crown will be removed in the conventional way.

Our tip:

The slit walls which are produced must be perpendicular. Always cut through the bottom crown edge and, if possible, extend the slit up to the masticatory surface. In the case of bridges, move the slits approximately 1/3 towards the suspension element.

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JESCO®-Form impression trays, impression trays acc. WÖSTMANN perforated pattern. Using plastic or resilient impression compounds, these materials will penetrate the holes and give the necessary support for a correct impression. The impression trays are made of stainless steel.

The impression trays for lower jaw acc. WÖSTMANN in addition have the following characteristics and advantages:

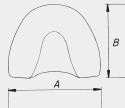
- Modified tray shield and changed tray curve
- Lengthened tray with simultaneous reduction of transversal expansion
- All teeth are seized, incl. the third molars
- Sufficient and uniform thickness of the impression compounds in the area of lingual undercut of the teeth

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* The shapes of the impression trays are based on an analysis of naturally toothed patients of:

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For resilient impression compounds

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Using plastic or resilient impression compounds, these materials
will penetrate the holes and give the necessary support for a
correct impression.

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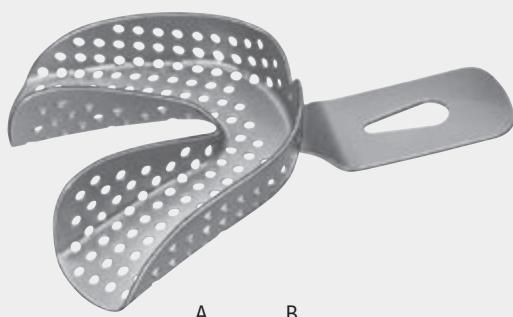
JESCO®-Form impression trays
for toothed upper jaws

DR201R	OB1	66 mm x 52 mm
DR202R	OB2	71 mm x 62 mm
DR203R	OB3	71 mm x 64 mm
DR204R	OB4	80 mm x 70 mm

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Retraction thread packer, flexible

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165 mm, 6 1/2"

DF150R

besonders flexibel
especially flexible
185 mm, 7 1/4"

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Telescope pliers grip the inner crown so that the outer one can be separated easily. To avoid damage to the crown in too stern a pressure, the set screw can be adjusted.

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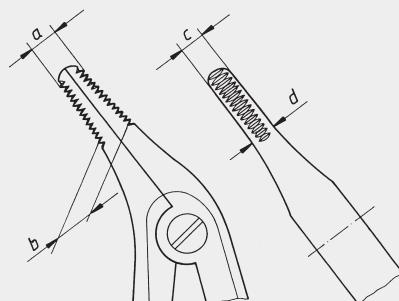
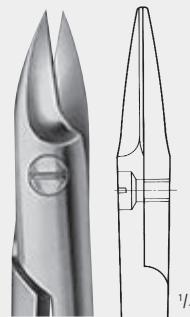
slender pattern
the diamond powder
coated gripping surfaces
allow an especially
firm grip

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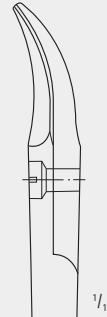
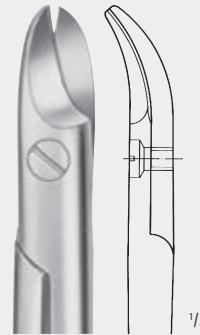
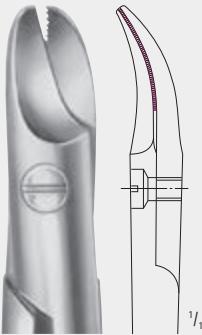
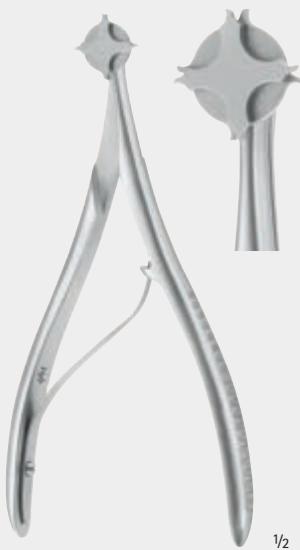
Crown widening pliers, for removing crowns and bridges

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Coperring, temporary measure and crown removing pliers

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Crown and bridge remover, complete, with 3 points

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Light wire pliers

- for shaping and closing loops
- with 3 holding grooves
- max. wire size .028"

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Bird beak pliers

- with tungsten carbide inserts (can be replaced)
- max. wire size .028"

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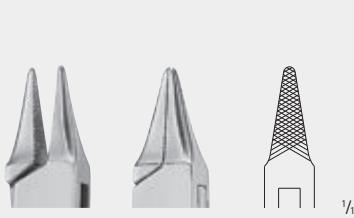
Flat pliers

- mouth parts serrated
- with short mout parts,
for better transmission of force

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Three-finger pliers

- For use with hard wires and spring hard wires in the application of orthodontic appliances, quad-helix appliances, Nance appliances and many other appliances
- max. wire size: 0.7 mm (.028")

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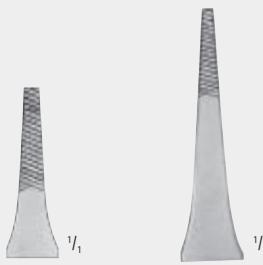
Triangular bending clamp pliers
■ for fast and exact bending of
arrow-head clasps
■ max. wire size: 0.7 mm (.028")

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Loop bending pliers
■ for loops of any dimension
■ particularly suitable for labial bows
■ max. wire size: 0.7 mm (.028") springy

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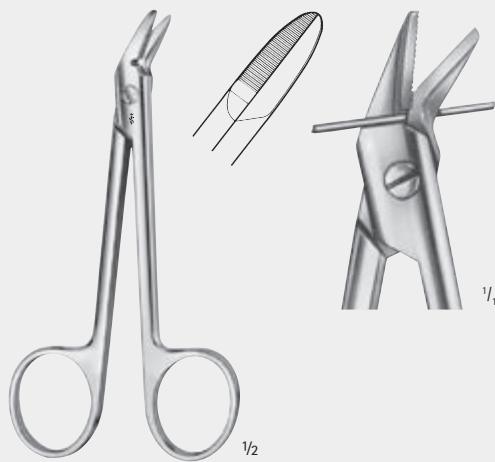
Wire scissors, 1 blade serrated
for soft wire up to Ø 1.2 mm,
for hard wire up to Ø 0.6 mm

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Wire scissors, 1 blade serrated
for soft wire up to Ø 1.2 mm,
for hard wire up to Ø 0.8 mm

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Hard wire cutter

- with tungsten carbide inserts
(can be sharpened/replaced)
- for cutting all orthodontic wires
- wires can be cut without deformation
- max. wire size: .022 x .028"

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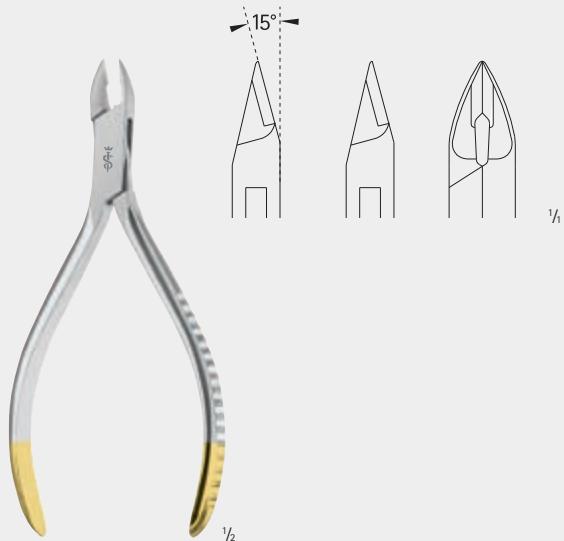
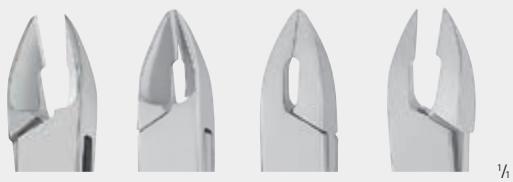
Distal end cutter

- with tungsten carbide inserts
(can be sharpened/replaced)
- for intraoral cutting, with safety hold,
from wire size .014" upwards
- max. wire size: .021 x .025"

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Ligature cutter

- with tungsten carbide inserts
(can be sharpened/replaced)
- fine cutting jaws and their 15° angular offset enable
cutting close to the bracket
- max. wire size: .014"

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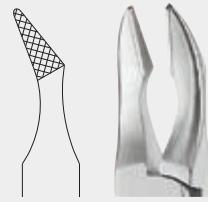
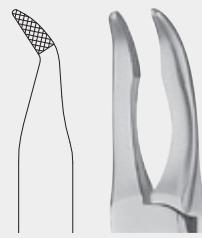
Universal orthodontic pliers

- used as a retaining instrument and to insert archwires in the tube
- enables easy intraoral activation of archwires
- short jaws for better transmission of force

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Torque pliers

- with tungsten carbide inserts (can be replaced)
- for exact, sharp-edged bending thanks to sturdy tungsten carbide inserts
- max. wire size: .028"

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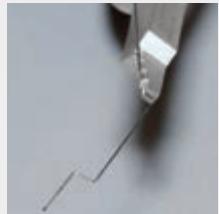
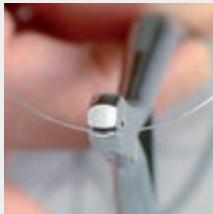
Loop forming pliers

- sturdy, rounded jaw part in size 2 and 2.5 mm (for the inner diameter of the loop)
- flat jaw part serrated for good fixation of the archwire
- sufficient room for the reliable and torque-free formation of loops
- max. wire size: .019 x .025"

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Arch bending pliers

- for forming the desired curvature
- the curved design of the jaws allows the curvature of the wires to be adjusted in small steps
- the spring-loaded handle opens the pliers automatically, which facilitates the work
- suitable for all orthodontic wires

$\frac{1}{2}$

NANCE

DP333R

125 mm, 5"

Stufenbiegezange

- Steps of 3, 4, 5 and 6 mm enable exact, even bending, e. g. for utility arches

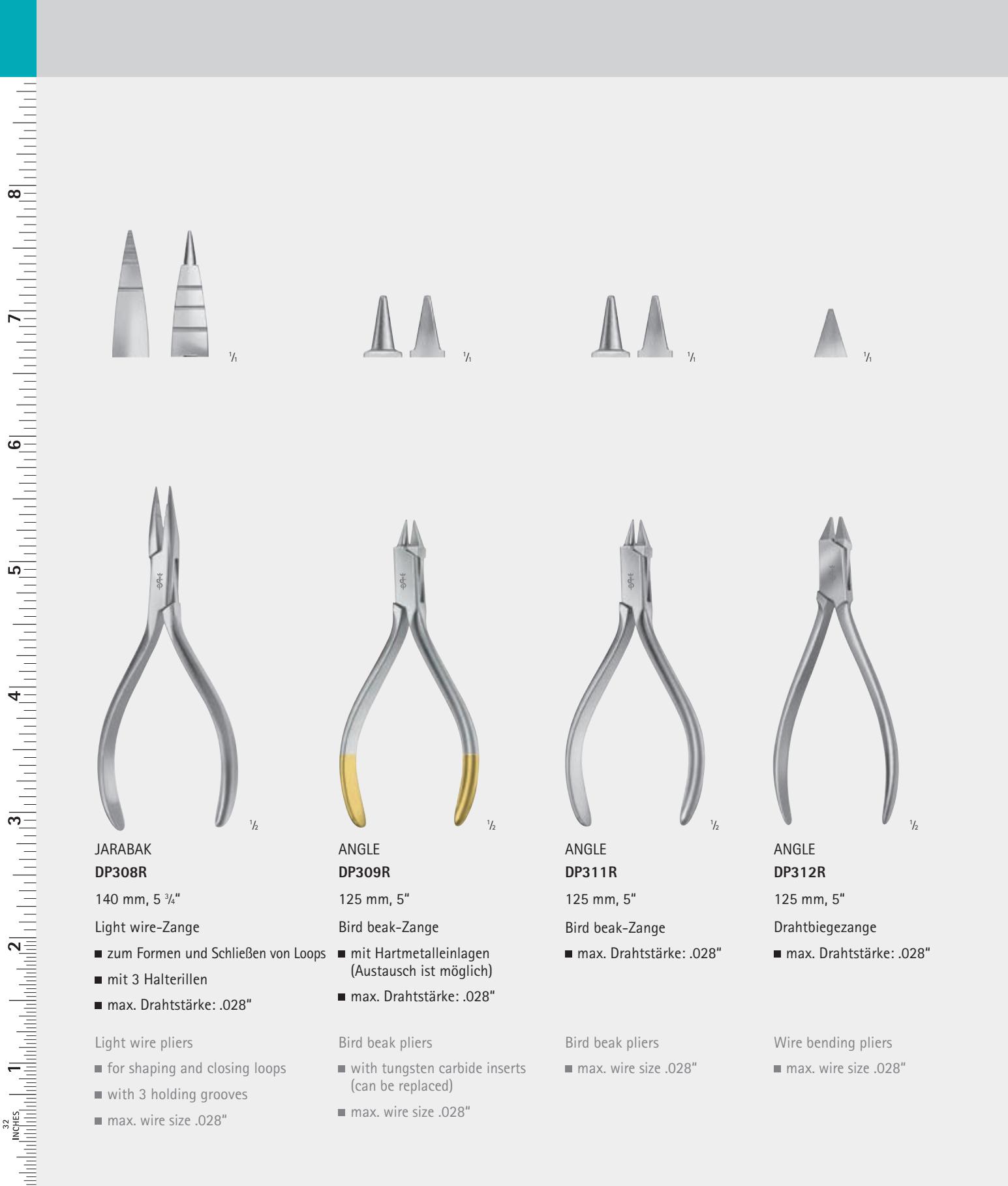
Step loop bending pliers

- Steps of 3, 4, 5 and 6 mm enable exact, even bending, e. g. for utility arches

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Band removing pliers
To remove bands, the plastic tip is
placed on the occlusal surface and the
metal beak is placed under the
tube or cleat, and the bands are
removed in small steps with
alternating forces buccally and
lingually/palatally.

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Band remover

- with tungsten carbide insert (can be replaced)

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Combination clamp forceps

- straight
- mouth parts serrated
- for positioning brackets
- spindle on forceps end is used for adjusting the brackets

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Combination clamp forceps

- angled
- mouth parts serrated
- particularly suitable for positioning brackets in the posterior teeth
- spindle on forceps end is used for adjusting the brackets

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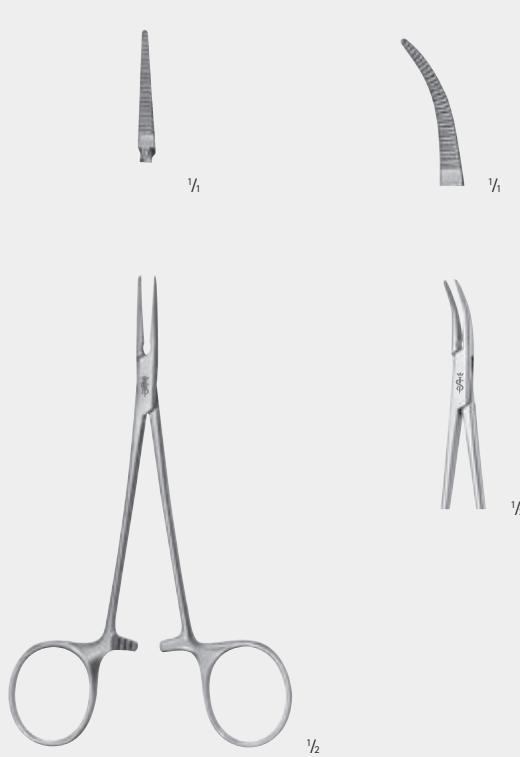
Alastik holder (MATHIEU)

- with delicate, serrated mouth parts
- for applying rubber ligatures
- Spring enables single-handed use
- the single-step locking mechanism facilitates the work and makes it easier to remove the clamp from the patient

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Forceps, straight

- mouth parts serrated
- for applying rubber ligatures

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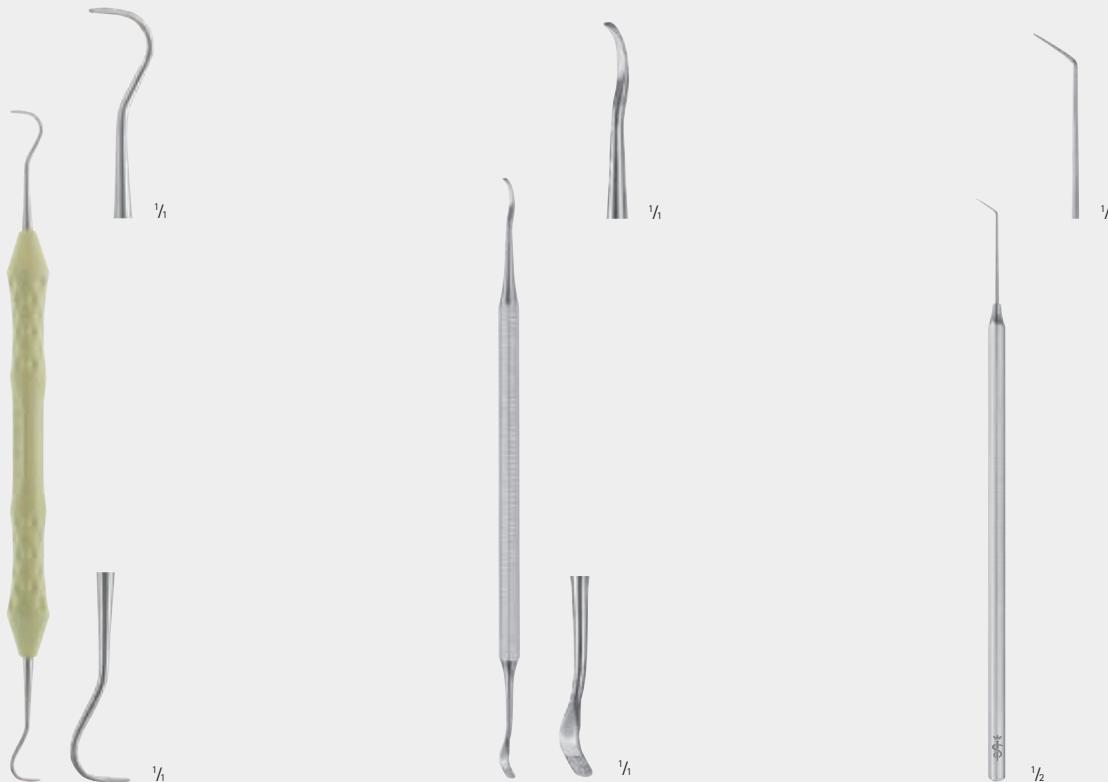
Scaler

- for removing adhesive and cement residues

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Scaler/Pusher

- for positioning and removing bands and for removing cement and adhesive residues

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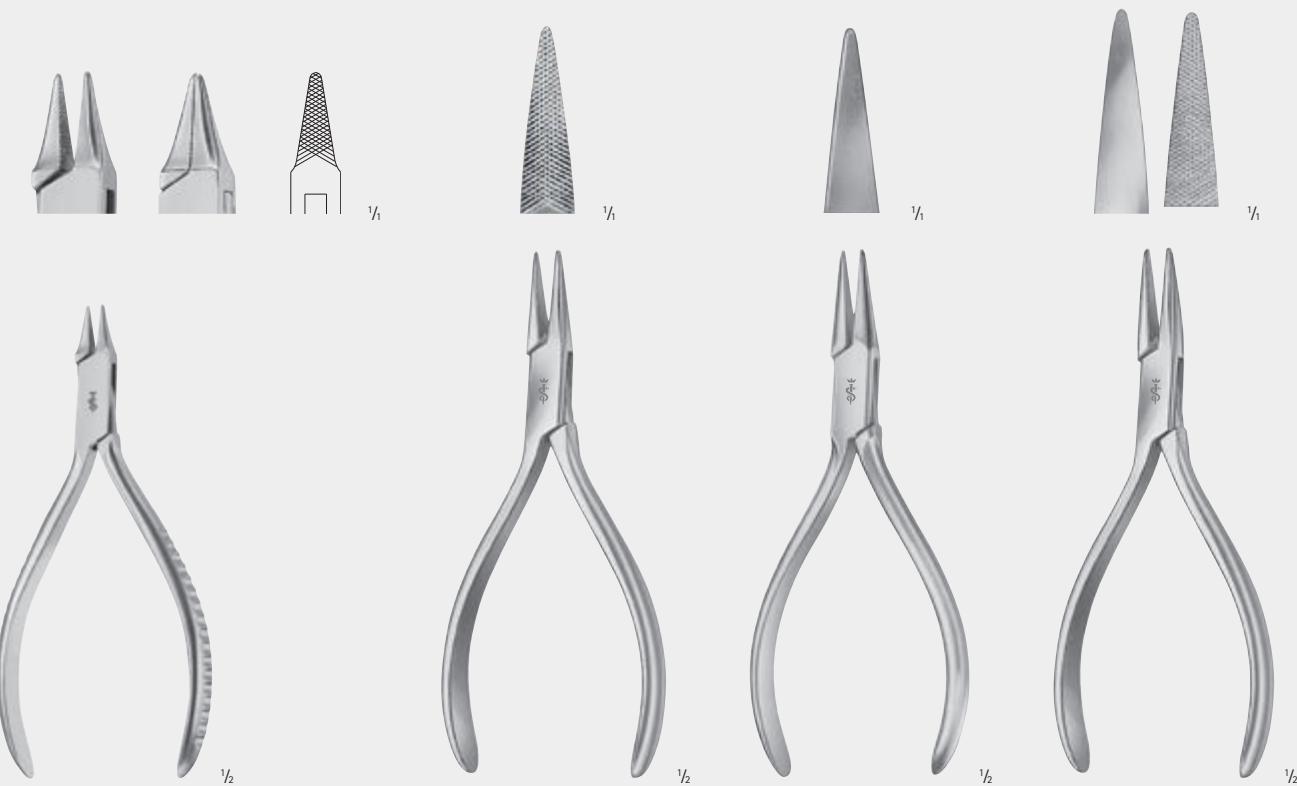
Orthodontic probe

- short, strong tip
- among other things, suitable for removing ligatures and adjusting brackets

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SYSTEMATIC INSTRUMENT REPROCESSING PROCEDURE

The primary intention of hygienic measures in dental practice is to safeguard the health of patients and personnel. To ensure safety it is particularly important to observe the principles of hygiene when cleaning and sterilizing instruments. To maintain a safe, efficient and economical workplace, clear procedures for hygiene must be specified and followed. The AESCULAP® Sterile Container System is a proven and excellent aid in the standardized management of instruments throughout the dental practice workflow, from sterile processing through transport, storage and use.

STERILGUTKREISLAUF – INSTRUMENTENAUFBEREITUNG STERILE GOODS CYCLE – INSTRUMENT REPROCESSING

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FOUR SMART REASONS TO CHOOSE THE AESCLAP® STERILE CONTAINER SYSTEM

■ Prevention of infections

Of patients and staff through preservation of instrument sterility and closed transport of used instruments

■ Economy

Low total cost of ownership due to reusable packaging, minimal processing costs and long service life

■ Comfortable handling

Simple, quick and reliable

■ Transparency

Standardized, effective documentation of your sterile supply workflow

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4 | Kontrollieren, pflegen, verpacken
Inspection, care and maintenance and packing

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INNOVATION FACTORY IN TUTTLINGEN – INNOVATION MEETS TRADITION

The AESCULAP® sterile container system has been manufactured at the Tuttlingen / Germany site for more than 45 years.

With a modern industrial architecture, at the Innovation Factory, which was opened in June 2015, AESCULAP® acts on the B.Braun corporate values: be innovative, increase efficiency and live sustainably.

The goal is an intelligent factory. This is characterized by adaptability, efficient use of resources and ergonomic design.

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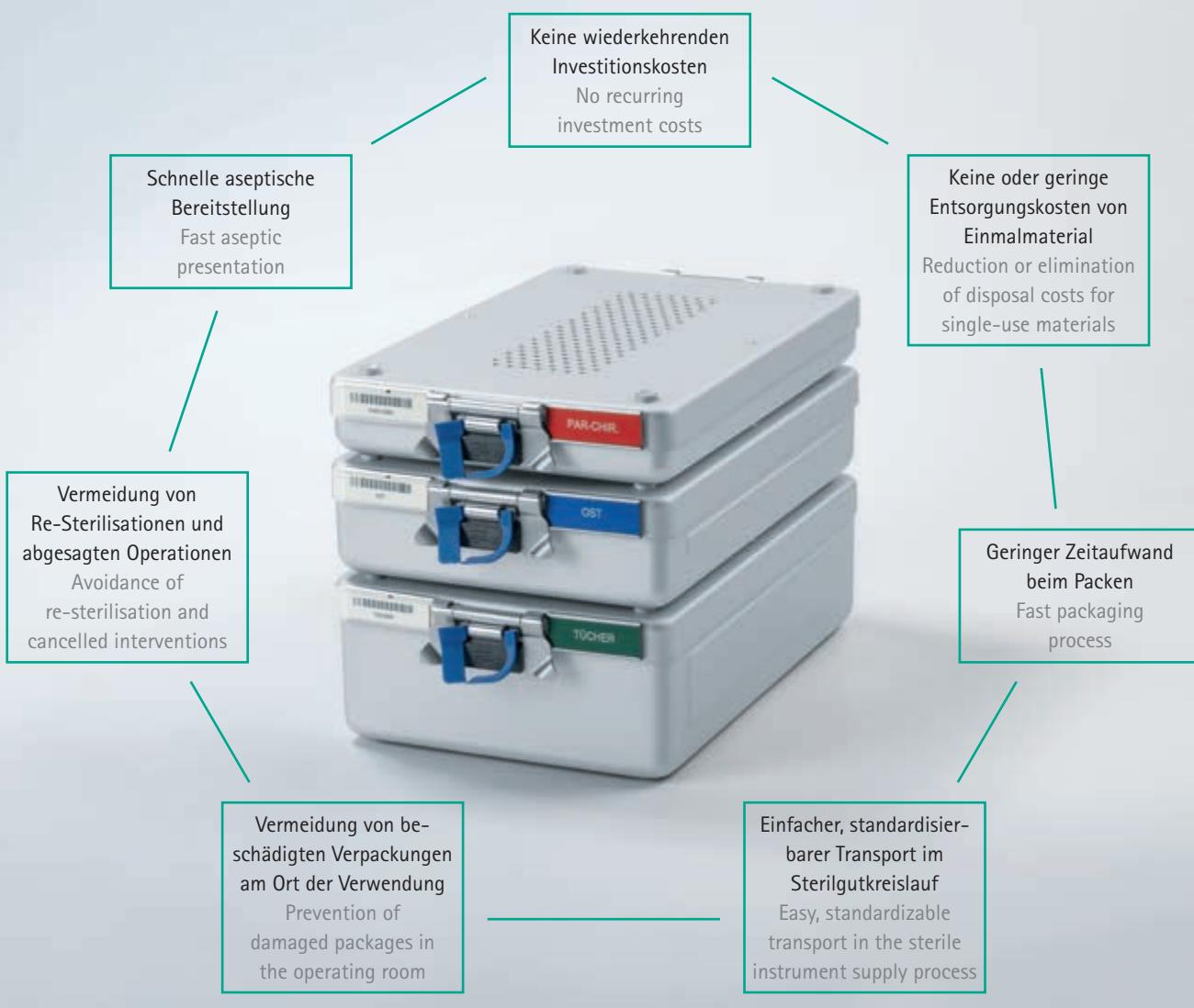
The solid, robust design and the manufacturing technology used by AESCULAP® enable the sterile containers, instrument baskets and all the other components to be manufactured with very precise dimensions and a high level of stability.

All the components of an AESCULAP® sterile container comply with the national and international requirements and standards (e. g. DIN EN ISO 11607-1).

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QUALITY AT A GLANCE

The tension locks are also designed as handles for fast, contamination-free lid removal when sterile goods are presented for use.

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The lock plates can be fixed at any position and turned through 90° into the lid. Springs integrated into the lock plates guarantee that the sterile container always remains consistently closed.

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The containers can be stacked, reducing the amount of space required in the sterile goods cycle.

The seal prevents accidental opening of the container after sterilization.

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AESCALAP® STERILE FILTERS

- The filter material conforms to the requirements of DIN EN ISO 11607
- Reliable: easy to check
- Advantageous: they facilitate good drying results within the container

The AESCULAP® PTFE permanent filter also contributes to efficiency and economy with the following features:

- Extremely resistant and durable
- Validated for 1,000 use cycles

Closure of the dental containers is ensured by the ringshaped silicon seals around the inside of the lid. The filter holder has an extremely durable silicon seal which keeps the filter firmly in place:

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EFFECTIVE AND CLEAN

The wide perforation ensures optimal water jet permeation and avoids rinsing blind spots.

„The microbiological and organic contamination is satisfactorily reduced (1)"

This is the result of extensive testing carried out by the independent laboratory Medical Device Testing.

PAR surgical instrument sets in AESCULAP® trays contaminated with test germs and organic matter were used for testing. Consequently, the mechanical cleaning and thermal disinfection before sterilization were rated as being adequate in accordance with DIN EN ISO 17664.

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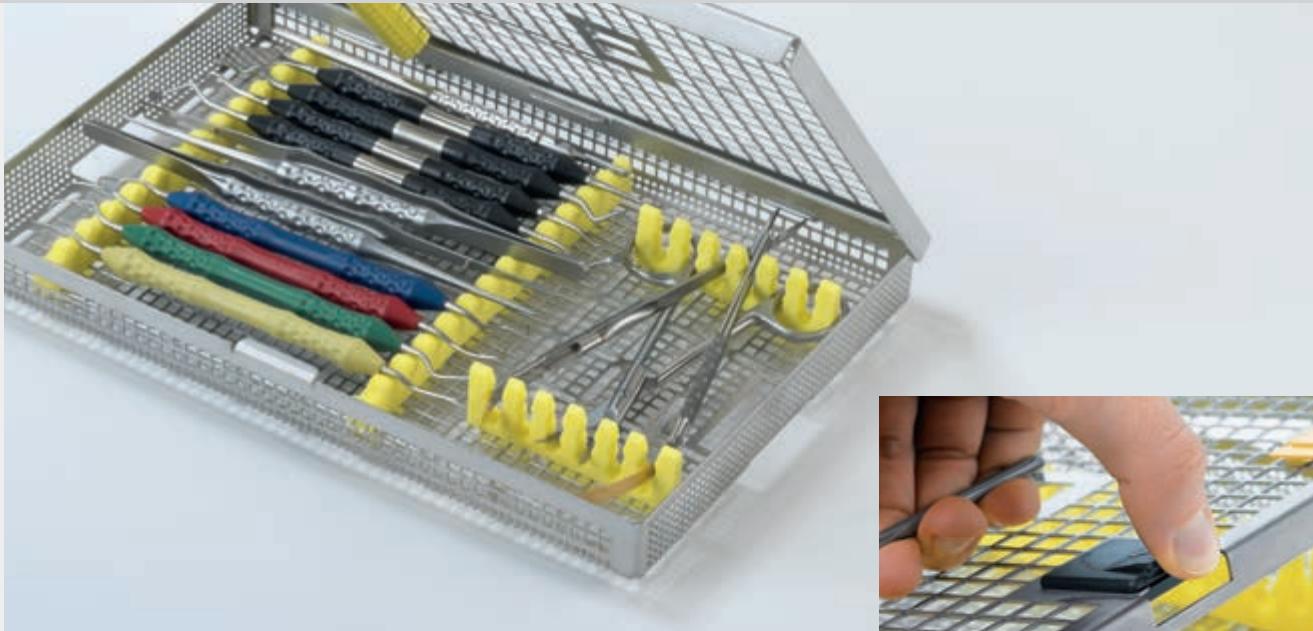
ORGANIZATION

Your instruments are stored in the smallest possible space, safely fixed in their positions.

(1) Final report: Determination of the effectiveness of an Automatic Cleaning Process.
19-February 2009.

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ECONOMICAL

Mechanical processing with the AESCULAP® tray system optimizes your workflow. The stainless steel material makes AESCULAP® trays a robust and durable asset. The simple and robust locking system can be easily opened and closed with one hand.

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FLEXIBILITY

Position the silicone racks in the optimal arrangement for your instruments – in seconds, without any additional tools.

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AESCLAP® reusable filter for dental containers JN092-JN096 and JN292-JN296, service life 1,000 cleaning and sterilization cycles.

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AESCLAP® paper filter with process indicator* for steam sterilization, for dental containers JN092-JN096 and JN292-JN296.

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Filter retainer universal for AESCLAP® reusable filter JK091 and AESCLAP® paper filter JF166. Suitable for lids and bodies of dental containers JN092-JN096 with narrow perforation.

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Tray, contains five instrument racks (2x JG394, 2x JG400, 1x JG401), with lid handle, fitting into Dental Container JN092

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Tray, contains three instrument racks (3x JG392), with lid handle, fitting into Dental Container JN094

273 x 84 x 41 mm (hoch/high)

JG386R

Siebtray, passend in Dental Container JN094

Tray, without inserts, with lid handle, fitting into Dental Container JN094

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Tray, contains three instrument racks (JG393), with lid handle, fitting into Dental Container JN094

273 x 176 x 41 mm (hoch/high)

JG285R

Siebtray, passend in Dental Container JN095

Tray with lid fitting into Dental Container JN095

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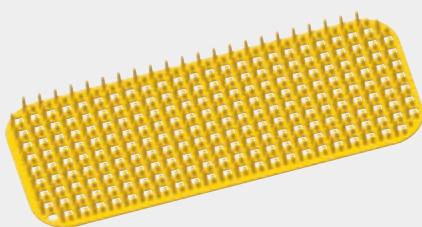
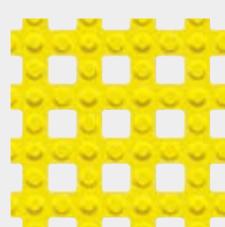
Non-slip storage of surgical instruments, efficient
drying and careful storage during machine cleaning.

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Instrument racks for special trays with 30 mm height

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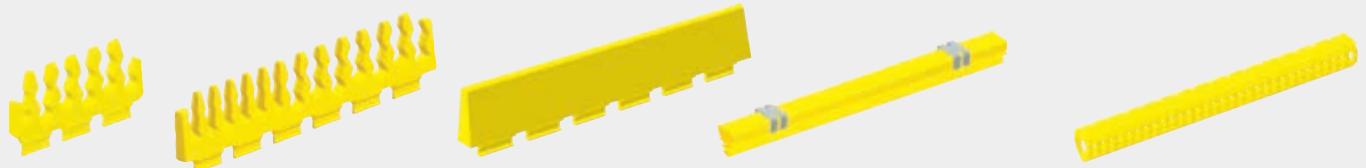
Instrument racks for special trays with 41 mm height

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Press pad, for suppressing
the instrument, to assemble
to the lower side of the lid.

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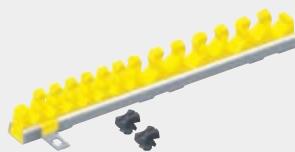
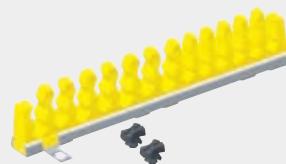
AESCLAP® oil spray, in aerosol can (ozone-friendly), for maintenance of instruments prior to sterilization. Physiologically inert acc. to DAB and § 31 LMBG (German Pharmacopoeia)

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AESCLAP® instrument oil, for box locks, ratchets etc. prior to sterilization. Physiologically inert acc. to DAB and § 31 LMBG (German Pharmacopoeia)

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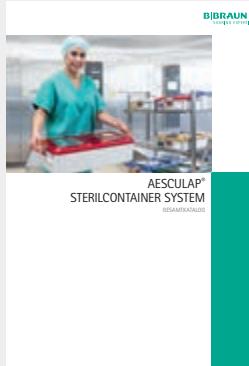
For further details see
brochure No. **C40402**

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For further details see
brochure No. **4004000**

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Seite

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INDEXES

	Page	Page
A		
Abdominal and intestinal spatulas	121	Arkansas grinding stone 54, 273
Accessories, containers	374	Atraumatic micro forceps 101
Eloxal cleaner	377	Atraumatic forceps 159, 160
Filter retainers	375	Atraumatic micro forceps 52
Fixation pins	382	Atraumatic papilla elevators 269
Grinding stones oil	42, 54, 273	Awls 169
Indicator labels	377	for zygomatic arch 169
Instrument oil	383	Mandibular awls 169
Instrument racks	381	Maxillary awls 169
Marking pens	377	
Oil for grinding stones	42, 54, 273	B
Oil spray	383	
Paper filters	375	BABY-CRILE-WOOD, Needle holders 276
Press pads	381	BABY-ALLIS, Tissue grasping forceps 168
Process identification labels	376	BABY-CRILE, Haemostatic forceps 167
Reusable filters	375	BABY-CRILE-WOOD, Needle holders 201
Seals	377	BABY-METZENBAUM, Dissecting scissors 139, 153
Storage, non-slip	380	Surgical scissors 151
Test rods	42	BABY-MOSQUITO (HARTMANN), Haemostatic forceps 166
ADERER , Three-finger pliers	342, 364	BACKHAUS, Towel clamps 165
ADSON , Atraumatic forceps	160	Ball burnisher/spatulas 71
Suction instruments	209	Band removing pliers 359
Surgical forceps	163	Bandage and cloth scissors 156
ADSON-BROWN , Surgical forceps	163	BANE-HARTMANN, Bone rongeurs 196
After-GRACEY , Curettes	34, 35	BARRY, Elevators 257
AINSWORTH , Rubber-dam punches	96	BARTH, Bone curettes 181
Alastik holders	361	Basic kit 283
ALLIS , Tissue grasping forceps	168	BAUMGARTNER, Needle holders 197
ANATOMICA , separating forceps	223	BEEBE, Crown scissors 337
Tooth forceps	217-230	BEERENDONK, Calipers 332
ANDERSEN , Round-nosed pliers	343	BEIN, Elevators 250
ANDRESEN , Wire bending pliers	345	BERGMANN, Retractors 132
ANGLE , Bird beak pliers	340, 358	BEYER, Bone rongeurs 195, 196
Wire bending pliers	340, 358	Bird beak pliers 340, 358
Wire cutters	349	BLACK, Cutting instruments 93
Applicator, for bone split retractors	306	Excavators 95
for bone material	298, 313	Plastic-filling instruments 82
for bone split retractors	306	BLAKESLEY-WILDE, Ethmoidal forceps 119
Arch bending pliers	357	BLUMENTHAL, Bone rongeurs 195

ALPHABETICAL INDEX

	Page		Page
BÖHM , Telescope pliers	336	Bone split, step-osteotomes	301
Bone condenser	307, 308	Bone mill, cleaner	296
Kit	309	Cutting disks	296
Bone curettes	178-182, 270	Micro bone mill	296
Bone cutting forceps	196	Micro bone mill kit	297
Bone files	182, 300	BOWDLER-HENRY , Retractors	127
Bone fixation, bone graft preparation forceps	291	BOWMAN , Probes	207
Box for the twistdrills	293	Box for the twistdrills	293
Dishes	292	Brain spatulas	121
Forceps	193, 291	BREWER , Clamp forceps	97
Osteotomes	292	Bridge and crown remover	339
Permanent filters	293	BROPHY , Surgical forceps	162
Plate cutters	291	BROWN , Surgical forceps	163
Plates	291	BRUENINGS , Tongue spatulas	117
Rack for screw magazines	293	BÜCHS , Plastic-filling instruments	83
Rack for twist drills	293	Raspatories	177
Screw driver blades	289, 292	Retractor for cheeks and soft tissue	128
Screw driver handles	289, 292	Soft tissue hooks	128
Screw magazines	287, 288, 291	Soft tissue retractors	323
Screw safety sleeves	289, 292	Tooth forceps	217, 218, 220, 222, 224, 225, 228-230
Set	293	BUCHWALD , Tongue spatulas	117
Sterile containers	293	Burnisher	88, 90
Twist drills	287, 288, 292	Ball burnisher/spatulas	71
Bone graft preparation forceps	291	BUSER , Bone raspatories	300
Bone harvesting, applicator for bone material	298	Micro-raspatories	171, 268
Bone file	300	Raspatories	57, 171, 269
Bone raspatories	300		
Bone rongeurs	299		
Membrane placing instruments	298		
Bone holding forceps	193	C	
Bone processing forceps	193		
Bone raspatories	300	Calipers	332
Bone rongeurs	194-196, 299	Carver	81
Bone split, applicator for bone split retractors	306	Micro carver	73
Bone split retractors	306	Carving instruments	92
Kit	304	CASTROVIEJO , Needle holders	200
Mallet	301	Cement spatulas	334, 335
Osteotomes	302, 303	CHADWICK , Delicate scissors	145
Rack for bone split retractors	306	Chain with clip lock	115, 325
Retractors	306	Channeled retractors	130
Spare plastic disks	301	Cheek and lip retractors	116

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INDEXES

Page	Page		
Cheek retractors	116, 122, 125, 127, 322	Crown scissors	337
Children tooth forceps	232-235	Crown widening pliers	328, 338
American pattern	243	CRYER, Children tooth forceps, american pattern	243
Chin retractors	130	CRYER, Elevators	254
Chisels	183, 184, 186, 187	CRYER, Tooth forceps, american pattern	236, 241
Haemostatic chisels	186	CRYER-WHITE, Elevators	254
Mini chisels	60, 61	Curettes	23, 39, 66, 319
Chisel-osteotomes	186	After-GRACEY-curettes	34, 35
CHOMPRET, Syndesmotomes	261	Bone curettes	178-182, 270
Clamp forceps	97	Disk-shaped curette	66
Clamps, towel clamps	165	GRACEY-curettes	30-32
Cleaner, eloxal cleaner	377	Mini-GRACEY-curettes	36, 37
Cloth and bandage scissors	156	Recall set	41, 321
COLUMBIA, Universal curettes	28, 29	Rigid GRACEY-curettes	33
Combination clamp forceps	360	Universal curettes	27-29
Condenser, bone condenser	307, 308	CUSHING, Atraumatic forceps	160
Bone condenser kit	309	Cutters, distal end cutters	352
Containers	374	Hard wire cutters	352
Filter retainers	375	Ligature cutters	353
Paper filters	375	Pin- and wire cutters	347
Reusable filters	375	Wire cutters	349, 350
Seals	377	Wire- and pin cutters	347
Contra-angle set, plastic-filling instruments	84, 85	Cutting disks	296
CONVERSE, Needle holders	201	Cutting instruments	93
COOLEY, Atraumatic micro forceps	52, 101	D	
COOPER, Surgical scissors	136		
Coperring, temporary measure and crown removing pliers	338	DARBY-PERRY, Excavators	95
COTTE, Chisel-osteotomes	186	DE BAKEY, Dissecting forceps	159
Mallets	191	Dissecting scissors	141
Osteotomes	186	Needle holders	202
Cotton plugger	134	De La ROSA, Arch bending pliers	357
COUPLAND, Elevators	253	DEAN, Delicate scissors	145
CRANE CAPLAN, Scalers	23	DEAVER, Surgical scissors	138
CRILE, Haemostatic forceps	167	Delicate scissors	142-145, 150
Needle holders	199	Desmotomes	260
CRILE-MURRAY, Needle holders	197	Desmotomes/raspatories	57, 173, 261, 268
CRILE-WOOD, Needle holders	201	DIADUST®, telescope pliers	336
Crown and bridge remover	339	Diagonal cheek retractors	128
Crown depressor	338		
Crown remover	339		
Crown removing, coperring and temporary measure pliers	338		

ALPHABETICAL INDEX

	Page		Page
DINGMANN, Bone holding forceps	193	ERGOPLANT, Atraumatic forceps	160
DINGMANN, modif., Bone holding forceps	193	Forceps	159
Dishes	213, 292	Mouth wedges	115, 325
Round bowls	213	Surgical forceps	163
Disk-shaped curette	66	Tongue retractors	117, 120
Disk-shaped elevator and dissectors	64	Vestibulum hooks	116
Dissecting forceps	157-159	ERGOPLANT, applicator for bone material	298, 313
Dissecting scissors	139-141, 146, 147, 153-155	Applicator for bone split retractors	306
Dissection raspatories	60	Bone condenser	307, 308
Dissector, disk-shaped elevator and dissectors	64	Bone condenser kit	309
Periodontal dissectors	60	Bone files	182, 300
Dissector/raspatories	313	Bone raspatories	300
Distal end cutters	352	Bone split kit	304
DOCKHORN, Retractors	129	Bone split retractors	306
DOYEN-JANSEN, Mouth gags	113	Bone mill cleaner	296
Dressing and sponge forceps	164	Cheek retractors	322
Drill racks	382	Cutting disks	296
Drills, box for the twistdrills	293	Mallets	191, 301
Twist drills	287, 288, 292	Membrane placing instruments	298
DUROGRIP®, atraumatic forceps	160	Micro bone mill	296
Needle holders	53, 200-205, 276	Micro bone mill kit	297
Tooth tweezers	15	Non-traumatic dissecting forceps	277
DUROTIP®, delicate scissors	150	Osteotomes	302, 303
Gum scissors	149	Pair of spare disks	191
Ligature scissors	152	Periotomes	260, 274
Scissors	275	Sinus elevators	310
Surgical scissors	151	Sinutome kit	316
Wire scissors	348	Sinutomes	314, 315
		Soft tissue forceps	277
		Soft tissue retractors	323
		Spare plastic disks	301
		Spatulas	274
		Step-osteotomes	301
E			
EAST GRINSTEAD (WARD), Chisels	186	Tissue forceps	277
Osteotomes	186	Tongue retractors	324
Elevators, disk-shaped elevator and dissectors	64	Vestibulum hooks	323
Sinus elevators	310	Ethmoidal forceps	118, 119
Elevators	174, 311-313, 248-258	Excavators	81, 93-95
Eloxal cleaner	377	Explorers	6-9
ENGL. MOD., Towel clamps	165	Micro explorer	67
Ergoperio forceps	47-49, 103-105, 279-281	Exterior box	206

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INDEXES

	Page		Page
Extraction instruments	245-247	Forceps, Ergoperio forceps	47-49, 103-105, 279-281
ExtraLux, extraction instruments	245-247	Ethmoidal forceps	118, 119
		Forceps for articulating paper	92, 332
		Haemostatic forceps	166-168, 275
		Ligature and wire forceps	169
		Micro dissecting forceps	52, 101
		Micro tissue forceps	52, 67, 101
F		Non-traumatic dissecting forceps	277
FARABEUF, Retractors	124	Pin holding forceps	74, 208
FEINER, Tooth forceps	223, 230	Plate cutters	291
FELSCH, Tooth forceps	220	Reposition forceps	192
FERGUSSON, Suction instruments	210	Root splinter forceps	231
FERGUSSON-ACKLAND, Mouth gags	113	Separating forceps	223
FICKLING, Haemostatic forceps	168	Soft tissue forceps	277
Files, bone files	182, 300	Splinter forceps	208, 231
Filling remover	92	Sponge and dressing forceps	164
Filters, paper filters	375	Surgical forceps	161-163
Permanent filters	293	Suture forceps	158, 200
Reusable filters	375	Tissue forceps	277
Retainers	375	Tissue grasping forceps	168
Finishing instruments	90, 91	Tooth forceps	217-230
Fixation pins	382	Tooth forceps for children	232-235
Flat pliers	341, 363	Tooth forceps, american pattern	236-242
FLOHR, Elevators	252	Wire and ligature forceps	169
FOMON, Dissecting scissors	146	FOSTER FLAGG , Plastic-filling instruments	87
Retractors	132	FRAHM , Finishing instruments	91
Forceps	75, 159, 361	FRAZIER , Retractors	132
Articulating paper	92, 332	FREER , Chisels	184
Atraumatic micro forceps	101	Raspatories	59, 173, 176, 267
Atraumatic forceps	159, 160	Retractors	132
Atraumatic micro forceps	52	FREER-YASARGIL , Raspatories	174
Bone cutting forceps	196	FRETWURST , Desmotomes/raspatories	57, 173, 261, 268
Bone fixation forceps	193, 291	Furcation probes	9
Bone graft preparation forceps	291	FURRER ,	
Bone holding forceps	193	Coperring, temporary measure and crown removing pliers	338
Bone processing forceps	193		
Children tooth forceps	232-235		
Children tooth forceps, american pattern	243		
Clamp forceps	97	G	
Combination clamp forceps	360		
Dissecting forceps	157-159		
Dressing and sponge forceps	164	GABKA , Elevators	256

ALPHABETICAL INDEX

Page	Page		
Gauze cases	134	HARTMANN, Tongue spatulas	120
GERALD, Surgical forceps	162	HAUER, Medicine applicators	135, 208
GILLIES, Needle holders	198, 203	Periodontometers	21
Retractors	132	HEGAR-MAYO, Needle holders	201
Surgical forceps	162	HEGAR-OLSEN, Needle holders	198, 203
Gingival margin trimmer	92	HEGAR-OLSEN modif. LANGE, Needle holders	203
Gingivectomy knives	54, 62, 271	HEIDBRINK, Elevators	251
GLICKMAN, Raspatories	60, 171	HEIDEMANN, Spatulas	80, 88, 274
GOLDMAN-FOX, Dissecting scissors	147	HEISTER, Mouth gags	113
Gum scissors	149	HEMINGWAY, Bone curettes	179, 180
Scalers	27	Holder, alastik holders	361
Universal curettes	28	Holding device for identification labels	41, 321, 382
GOSLEE, Flat pliers	341, 363	HOLLENBACK, Finishing instruments	90
Gouges	183, 184	Plastic-filling instruments	86
GRACEY, Curettes	30-32	Hooks	177
GRAEFE, Retractors	132	Soft tissue hooks	128
GRAY, Tooth forceps for children	233	Vestibulum hooks	116, 323
Grinding Stones	42	HOPSON, Burnisher	88
Arkansas grinding stone	54, 273	Plastic-filling instruments	86
Oil	42, 54, 273	HÖSEL, Needle holders	205
GROSS, Dressing and sponge forceps	164	HOWARTH, Raspatories	176
Gum scissors	148, 149, 275	HULL, Children tooth forceps, american pattern	243
GUY, Tooth forceps	218	Tooth forceps, american pattern	236
		HYLIN, Elevators	252

H

Page	Page		
HAASE, Reposition forceps	192		
Haemostatic chisels	186	Identification labels	41, 321, 376
Haemostatic forceps	166-168, 275	Holding device for identification labels	41, 321, 382
HAJEK, Mallets	191	Impression trays	331
HALLE, Elevators	174	Indicator labels	377
HALSEY, Needle holders	197, 201	Instrument oil	383
HALSTED, Haemostatic forceps	168	Instrument racks	381, 382
HALSTED-MOSQUITO, Forceps	361	Interior box	206
Haemostatic forceps	166, 275	Intestinal and abdominal spatulas	121
Handle for micro mirrors and microsurgery blades	69	IVORY, Rubber-dam punches	96
Hard wire cutters	352		
HARRIS, Tooth forceps, american pattern	237		
HARTMANN, Nasal speculum	114		

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INDEXES

	Page		Page
J		L	
JACOBSON, Bone curettes	181	LA GRANGE, Delicate scissors	145
JAMESON, Dissecting scissors	141	Labels, identification labels	376
JAMESON-WERBER, Dissecting scissors	140	Indicator labels	377
JANSEN, Bone curettes	180, 181	LADMORE, Plastic-filling instruments	86
JARABAK, Light wire pliers	340, 358	LAMBOTTE, Osteotomes	183
JESCO®, Impression trays	331	LANGE, Raspatories	58, 172
JONES, Towel clamps	165	LANGENBECK, Needle holders	204
JOSEPH, Delicate scissors	144	Retractors	124, 125
Retractors	132	LANGENBECK-GREEN, Retractors	133
		LANGER, Curettes	39, 319
		Universal curettes	27
		LASTER, Cheek retractors	127
K		LEXER, Dissecting scissors	146
KAISER, Tooth forceps	221, 223, 230	LICHTENBERG, Needle holders	205
KAY (AUSTIN), Retractors	127	Lid for trays	380
KELLY, Delicate scissors	144	Ligature and wire forceps	169
Haemostatic forceps	167	Ligature cutters	353
KELSEY FRY, Awls	169	Ligature scissors	152, 156
KILNER, Dissecting scissors	140	Light wire pliers	340, 358
Needle holders	202	LINDO-LEVIAN, Elevators	253
Retractors	132	Lip and cheek retractors	116
KIRKLAND, Gingivectomy knives	62, 271	LISTER, Bandage and cloth scissors	156
Mini chisels	61, 185	Sponge and dressing forceps	164
KLEIN, Children tooth forceps	234, 235	LITTAUER, Ligature scissors	156
Tooth forceps for children	234, 235	LOCKLIN, Dissecting scissors	147
Knives, gingivectomy knives	54, 62, 271	Gum scissors	275
KOCHER, Haemostatic forceps	167	LONDON HOSPITAL COLEMAN, Elevators	256
KOCHER-LANGENBECK, Retractors	124	LONDON-COLLEGE, Tooth tweezers	13, 14
KÖHLER, Triangular bending clamp pliers	346	LONDON-COLLEGE, mod., Tooth tweezers	14
KÖNIG, Elevators	174	Loop bending pliers	346, 356
Raspatories	175	LORNA, Towel clamps	165
KORKHAUS, Wire and ligature forceps	169	LUCAS, Bone curettes	178, 270
KRAMER-NEVINS, Mini chisels	61, 185	Chisels	184
		Gouges	184
		LUER, Bone rongeurs	194
		LUER-FRIEDMANN, Bone rongeurs	194, 299
		LUNIATSCHEK, Cotton plugger	134
		LUNIATSCHEK, modif., Cotton plugger	134
		Luxators	259

ALPHABETICAL INDEX

	Page		Page
M			
Mallets	190, 191, 301	MICRO-ADSON, Surgical forceps	163
Pair of spare disks	190	MICRO-HALSTED, Haemostatic forceps	166
Spare plastic disks	301	Micro-Raspatories	171, 268
Mandibular awls	169	Microblade holders	265
Marking pens	377	Microsurgery blades	45, 107
MARTINI, Bone curettes	181	Microsurgical blades	265
MATHIEU, Needle holders	199, 205	MIDDLEDORPF, Cheek retractors	122, 125
MATHIEU-STANDARD, Needle holders	205	MILLER, Bone curettes	182
Matrix retainers	98	Bone files	182, 300
Maxillary awls	169	Mini chisels	60, 61, 185
Maxillary mobilizers	130	Mini-GRAEY, Curettes	36, 37
MAYO-HEGAR, Needle holders	198	MINNESOTA, modif., Cheek retractors	116
MC INDOE, Surgical forceps	162	Mirror, handle for micro mirrors and microsurgery blades	69
McCALL, Scalers	25	Micro mirrors	68
Universal curettes	29	Mouth mirrors	3, 266
MEAD, Bone rongeurs	196	Mouth mirror handles	3, 4, 266
Mallets	190	MOLT, Mouth gags	113
Pair of spare disks	190	Raspatories	59, 172, 267
Raspatories	176	Moulding instruments	78
Medicine applicators	135, 208	Mouth gags	113, 114
Membrane placing instruments	298	Mouth mirrors	3, 266
MERIAM, Tooth tweezers	12, 13	Handles	3, 4, 266
MERSHON, Crown depressor	338	Mouth wedges	115, 325
METZENBAUM, Dissecting scissors	139, 153-155	Chain with clip lock	115, 325
Surgical scissors	151	MÜHLING, Raspatories	177
Micro bone mill	296		
Micro bone mill kit	297		
Micro carver	73	N	
Micro dissecting forceps	52, 101		
Micro explorer	67	NABERS, Furcation probes	9
Micro ligature scissors	51, 100, 282	NANCE, Step loop bending pliers	357
Micro mirrors	68	Nasal speculum	114
Micro needle holders	53, 101, 200, 282	Needle attachments	211, 212
Micro scalpels	106	Needle cases	206
Micro scissors	100	Exterior box	206
Spring type	51	Interior box	206
Micro soft tissue set	285	Needle holders	53, 197-205, 276
Micro tissue forceps	52, 67, 101	Micro needle holders	53, 101, 200
MICRO-ADSON, Dissecting forceps	157	Nerve instruments	208, 231

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INDEXES

	Page	Page	
Nerve instruments, forceps	75	Pair of spare disks	190, 191
NEVIUS, Tooth forceps, american pattern	238	PALMER, Clamp forceps	97
NIEDERDELLMANN, Osteotomes	186	Paper filters	375
Noir®, Dissecting scissors	153-155	Papilla elevators	58
Non-traumatic dissecting forceps	277	Atraumatic papilla elevators	269
NORTHBENT, Ligature scissors	156	PARMLY, Tooth forceps, american pattern	239
NYSTRÖM, Matrix retainers	98	PARTSCH, Chisels	183
		Gouges	183
		Raspatories	177
		PASSOW, Haemostatic chisels	186
		Tampers	186
O		PEAN, Haemostatic forceps	166, 167
OBWEGESER, Awl for zygomatic arch	169	PEESO, Wire bending pliers	344
Channeled retractors	130	PEET, Splinter forceps	231
Chin retractors	130	Periodontal dissectors	60
Ligature and wire forceps	169	Periodontal probes	320
Mandibular awls	169	Periodontometers	10, 11, 18-21, 40
Maxillary awls	169	Periotomes	260, 274
Pterygomaxillary osteotomes	186	Permanent filters	293
Raspatories	175	PERRY, Tooth tweezers	12
Retractors	131	PHW, Atraumatic papilla elevators	269
Soft tissue retractors	126	Papilla elevators	58
Spina-nasalis retractors	129	PHYSICK, Tooth forceps, american pattern	242
Wire and ligature forceps	169	Pin holding forceps	74, 208
OBWEGESER-FREER, Elevators	174	Pin- and wire cutters	347
OCHSENBEIN, Mini chisels	61, 185	PLANERT, Crown widening pliers	328, 338
Oil, instrument oil	383	Plastic-filling instruments	82-87
for grinding stones	42, 54, 273	Burnisher	88
Spray	383	Spatulas	88
ORBAN, Gingivectomy knives	54, 62, 271	Plate cutters	291
Orthodontic probes	362	Plates	291
Osteotomes	183, 186, 188, 189, 292, 302, 303	PLESTER, Suction instruments	210
Chisel-osteotomes	186	Pliers, arch bending pliers	357
Pterygomaxillary osteotomes	186	Band removing pliers	359
Step-osteotomes	301	Bird beak pliers	340, 358
		Coperring, temporary measure and crown removing pliers	338
		Crown widening pliers	328, 338
		Flat pliers	341, 363
		Light wire pliers	340, 358
		Loop bending pliers	346
Packer, retraction thread packers	135, 333	Loop forming pliers	356

ALPHABETICAL INDEX

	Page		Page
Pliers, round pliers	343	Raspatories, dissection raspatories	60
Round-nosed pliers	343	Micro-raspatories	171, 268
Separating pliers	359	Raspatories/desmotomes	57, 173, 261, 268
Step loop bending pliers	357	Raspatories/dissectors	313
Telescope pliers	336	READ, Tooth forceps	221
Temporary measure, crown removing and coperring pliers	338	Recall set	41, 321
Three-finger pliers	342, 364	REILL, Reposition forceps	192
Torque pliers	355	Wire cutters	350
Triangular bending clamp pliers	346	REINWALD, Hooks	177
Universal orthodontic pliers	354	Reposition forceps	192
Wire bending pliers	340, 344, 345, 358	Retraction thread packers	135, 333
Pluggers	70, 72, 79	Retractors	65, 123–125, 127, 129, 131–133
Cotton pluggers	134	Bone split retractors	306
Press pads	381	Channeled retractors	130
PRICHARD, Raspatories	56, 170, 266, 267	Cheek and lip retractors	116
Probes	207	Cheek and soft tissue retractors	128
Furcation probes	9	Cheek retractors	116, 122, 125, 127, 322
Orthodontic probes	362	Chin retractors	130
Process identification labels	376	Diagonal cheek retractors	128
Pterygomaxillary osteotomes	186	Lip and cheek retractors	116
Punches, rubber-dam punches	96	Ramus retractors	129
Pusher/scaler	362	Retractors for cheeks and soft tissue	128
		Soft tissue retractors	126, 323
		Spina-nasalis retractors	129
		Tongue retractors	117, 120, 324
Q		Reusable filters	375
		Marking pens	377
QUINBY, Dissecting scissors	147	REYNOLDS, Dissecting scissors	141
		RHODES, Mini chisels	60, 61, 185
		RICHTER, Retractors	65
		Rigid GRACEY, Curettes	33
R		ROBERTS, Raspatories	176
		Rongeurs, bone rongeurs	194–196, 299
Rack, box for the twistdrills	293	Root splinter forceps	231
for bone split retractors	306	ROSER-KÖNIG, Mouth gags	114
for drills	382	ROSIN, Luxators	259
for screw magazines	293	Round bowls	213
for twist drills	293	Round pliers	343
Ramus retractors	129	Round-nosed pliers	343
Raspatories	56–60, 170–177, 266, 267, 269	ROUTURIER, Tooth forceps	227
Bone raspatories	300	ROUX, Cheek retractors	122

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INDEXES

S

	Page		Page
Rubber-dam punches	96	Screw magazines	287, 288, 291
Rulers	332	Screw safety sleeves	289, 292
RYDER, Needle holders	202	Seals	377
		SELDIN, Elevators	255
		Raspatories	176
		SEMKEN, Dissecting forceps	157
		Surgical forceps	162
		Separating forceps	223
Safety scalpels	110, 111	Separating pliers	359
SAROT, Needle holders	201	Set, basic kit	283
Scaler	23-27, 362	Bone condenser kit	309
Scaler/pusher	362	Bone fixation set	293
Scalpels, blades	44, 108, 109, 265	Bone split kit	304
Blade holders	265	Micro bone mill kit	297
Handles	44, 45, 107-109	Micro soft tissue set	285
Micro scalpels	106	Recall set	41, 321
Microsurgery blades	45, 107	Sinutome kit	316
Safety scalpels	110, 111	Soft tissue kit	284
Sterile scalpels	112	Sinus elevation, elevators	311
SCHAEDEL, Towel clamps	165	Sinus elevators	310
SCHMIDT, Children tooth forceps	234	Sinutome kit	316
Tooth forceps for children	234	Sinutomes	314, 315
SCHNITZER, Bone graft preparation forceps	291	Sinus lift, applicators for bone material	313
Bone processing forceps	193	Dissectors/raspatories	313
SCHURE, Pusher/scaler	362	Elevators	312, 313
SCHWARZ, Bone rongeurs	195	Sinutome kit	316
Wire bending pliers	345	Sinutomes	314, 315
Scissors	275	Soft tissue forceps	277
Bandage and cloth scissors	156	Soft tissue hooks	128
Crown scissors	337	Soft tissue kit	284
Delicate scissors	142-145, 150	Soft tissue retractors	126, 323
Dissecting scissors	139-141, 146-147, 153-155	Spare plastic disks	301
Gum scissors	148, 149	Spatulas	80, 88, 274
Ligature scissors	152, 156	Abdominal and intestinal spatulas	121
Micro ligature scissors	51, 100, 282	Brain spatulas	121
Micro scissors	100	Cement spatulas	334, 335
Micro scissors (spring type)	51	Tongue spatulas	117, 120
Surgical scissors	136-138, 151	Spatula/ball burnisher	71
Wire scissors	348	SPENCER, Ligature scissors	156
Screw driver blades	289, 292	Spina-nasalis retractors	129
Screw driver handles	289, 292	Splinter forceps	208, 231

ALPHABETICAL INDEX

	Page		Page
Sponge and dressing forceps	164	Telescope pliers	336
Spray, oil spray	383	Temporary measure, crown removing and coperring pliers	338
STAMM, Bone cutting forceps	196	TESSIER, Maxillary mobilizers	130
STANDARD, Atraumatic forceps	160	Osteotomes	189
Dissecting forceps	158	Test rods	42
Surgical forceps	161	Three-finger pliers	342, 364
STELLBRINK, Bone rongeurs	195	Tissue forceps	277
Step loop bending pliers	357	Tissue grasping forceps	168
Step-osteotomes	301	TOBOLD, Tongue spatulas	120
Sterile containers	293	TOENNIS, Needle holders	199, 204
Sterile scalpels	112	TOENNIS-ADSON, Dissecting scissors	146
Sterilite®, instrument oil	383	TOFFLEMIRE, Matrix retainers	98
Oil for grinding stones	42, 54, 273	TOMES, Tooth forceps, american pattern	239
Oil spray	383	Tongue retractors	117, 120, 324
STERNBERG, Cheek and lip retractors	116	Tongue spatulas	117, 120
STIEGLITZ, Splinter forceps	208, 231	Tooth forceps	217-230
Storages, non-slip	380	for children	232-235
STRICKER, Applicator for bone material	298, 313	american pattern	236-242
Applicator for bone split retractors	306	Tooth tweezers	12-15
Bone fixation forceps	193, 291	Torque pliers	355
Membrane placing instruments	298	Towel clamps	165
Sinus elevators	310	Trays	41, 321, 378-380
Spatulas	274	Fixation pins	382
Step-osteotomes	301	Holding device for identification labels	382
Tunnelling instruments	55, 272, 273	Instrument racks	381, 382
Suction instruments	209, 210	Lid for trays	380
Surgery blades,		Mini trays	380
handle for micro mirrors and microsurgery blades	69	Impression trays	331
Surgical forceps	161-163	TREYMANN, Crown remover	339
Surgical scissors	136-138, 151	Triangular bending clamp pliers	346
Suture forceps	158, 200	TROTTER, Tooth forceps	217, 225
SYME, Raspatories	174	Tunnelling instruments	55, 272, 273
Syndesmotomes	261	TWEED, Torque pliers	355
Syringes	211, 212	Tweezers, tooth tweezers	12-15
Needle attachments	211, 212	Twist drills	287, 288, 292

T

Tampers

U

UFFENORDE, Bone curettes

181

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INDEXES

Page	Page		
Universal curettes	27-29	WITZEL modif., Root splinter forceps	231
Universal orthodontic pliers	354	WOOD, Tooth forceps	226
		WOODSON, Plastic-filling instruments	86
		WÖSTMANN, impression trays	331
V			
Vestibulum hooks	116, 323	Y	
VOLKMANN, Bone curettes	181	YOUNG, Loop bending pliers	346
W			
WALLIS, Burnisher	88	Z	
WARD, Cheek retractors	127	ZAUFAJ-JANSEN, Bone rongeurs	196
Finishing instruments	91	ZIELINSKY, Calipers	332
Raspatories	177		
WARWICK-JAMES, Elevators	256		
WASSMUND, Retractors	129		
WAUGH, Surgical forceps	162		
WEIL BLAKESLEY, Ethmoidal forceps	118		
WEINGART, Universal orthodontic pliers	354		
WESTON, Explorers	9		
WHITE, Burnisher	90		
Elevators	250		
Excavators	95		
Plastic-filling instruments	87		
WHO, Periodontal probes	320		
Periodontometers	10, 11, 18, 19, 40		
WILAND, Carver	81		
WILLIAMS-FOX, Periodontometers	21		
WILLIGER, Bone curettes	181		
Elevators	174		
Raspatories	175		
WINTER, Elevators	255, 257		
Wire and ligature forceps	169		
Wire bending pliers	340, 344, 345, 358		
Wire cutters	349, 350		
Wire scissors	348		
Wire- and pin cutters	347		

ALPHABETICAL INDEX

Page

Page

NOTIZEN
NOTICE

AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-2600 | www.aesculap-dental.com

The main product trademark "Aesculap" and the product trademarks "DIADUST", "DUROGRIP", "DUROTIP", "JESCO", "Noir", "Sterican" and "Sterilit" are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.